

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

SAVE THESE INSTRUCTIONS

⚠ WARNING

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Each individual should always consult with their physician or therapist to determine proper adjustment and usage.
- Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately.
- **ALWAYS** observe the weight limit on the labeling of your cane. Check that all labels are present and legible. Replace labels as necessary.
- The canes are **NOT** designed to support the total weight of an individual.
- Snap button or double button **MUST** be fully engaged and protruding through the height adjustment hole on the cane to ensure a positive lock. An audible "click" will be heard. Make sure that the can handle is properly and securely locked in place **BEFORE** using.
- When using a quad cane, all four legs must be in contact with and perpendicular to the walking surface. Make sure that the longest/or flattest portion of base is closest to foot - otherwise, injury or damage may occur. It may be necessary to adjust the cane for a right or left hand user. If the cane is used in any other way, cane damage and/or personal injury may occur. Extra care should be exercised when walking on a slippery or wet surface. After adjustment and before use, make sure that the cam lever is locked in place.
- If the cane is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips **DO NOT** twist on cane handle - otherwise damage or injury may occur.
- **BEFORE** using a folding cane, make sure that it is opened properly and securely locked or a fall could result, causing bodily injury and/or damage.
- **Wooden canes MUST** be cut by Invacare Dealers **ONLY**. Use common sense when cutting wooden canes with hand or power tools. Safety glasses should be worn and other precautions may be necessary. Read and follow all instructions before using hand or power tools.

Cutting Canes

NOTE: This procedure **MUST** be performed by an Invacare Dealer **ONLY**.

NOTE: This procedure applies to wooden canes **ONLY**.

⚠ WARNING

The cane tip **MUST** be removed **BEFORE** cutting a wood cane. If the cane is cut without removing the tip, it is difficult to remove the tip from the cut portion of the cane. Injury or damage may occur.

1. Remove the cane tip.
2. Measure and cut the cane to the proper length for the user.

NOTE: The cut **MUST** be level and smooth. Avoid cutting the cane at an angle, otherwise the cane will become unstable.

3. Install the cane tip.


NOTE: It may be necessary to tap the cane tip on the ground to fully seat it onto the end of the cane.

Adjusting Cane Height

1. Depending on the style of cane, turn the anti-rattle collar to loosen or lift up on the cam lever.
2. Press the snap button or double button in to release it from the height adjustment hole in the cane.
3. Slide the extension up or down until the snap button or double button fully protrudes through a height adjustment hole in the cane.

NOTE: An audible "click" will be heard.

4. Check the height of the cane to the user. Repeat STEPS 2 and 3 as necessary.
5. Turn the anti-rattle collar in the opposite direction from STEP 1 to tighten, or press down on the cam lever to lock.

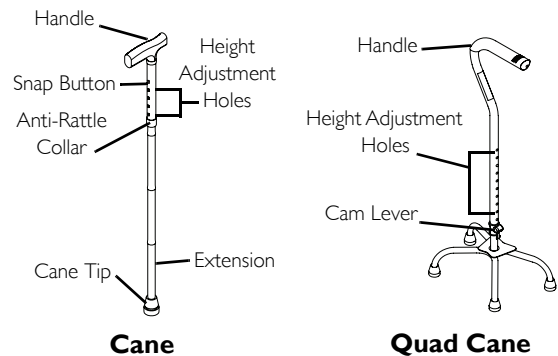

Adjusting a Quad Cane for Left or Right Hand Use

1. Lift up on the cam lever.
2. Press the double button in to release it from the height adjustment hole in the cane.
3. Turn the cane handle 180°. Make sure the double button protrudes through the height adjustment hole on the opposite side of the cane.

NOTE: An audible "click" will be heard.

4. Press down on cam lever to lock.

Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants all this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.