

One-Step Cleaner/Disinfectant/Sanitizer

Premoistened disinfecting wipes

- **Kills 99.9% of bacteria in just 15 seconds**
- **Effective in the presence of 5% serum contamination**
- **Hospital Disinfectant**
- **Eliminates Pseudomonas, MRSA, CA-MRSA, E.coli, Campylobacter in 4 minutes**

Protex Ultra Disinfectant Wipes Are Effective Against:

- **Staphylococcus aureus-Multi-Drug Resistant**
- **Staphylococcus aureus-Vancomycin Intermediate Resistant [VISA]**
- **Pseudomonas aeruginosa**
- **Burkholderia cepacia**
- ***Hepatitis B Virus [HBV]**
- ***Hepatitis C Virus [HCV]**
- ***HIV-1 [AIDS Virus]**
- ***SARS Associated Coronavirus [SARS]**

Active Ingredients:

Octyl decyl dimethyl ammonium chloride.....	0.0909%
Diocetyl dimethyl ammonium chloride.....	0.0364%
Didecyl dimethyl ammonium chloride.....	0.0545%
Alkyl (C ₁₄ , 50%; C ₁₂ , 40%; C ₁₆ , 10%)	
dimethyl benzyl ammonium chloride.....	0.1212%
Other Ingredients:	99.6970%
Total.....	100.0000% ¹

¹Includes weight of absorbent wipe

KEEP OUT OF REACH OF CHILDREN CAUTION

See back panel for Precautionary Statements and First Aid
EPA Registration No. 6836-340-82613
EPA Establishment No. 50757-WI-001, 50757-WI-003



Net Contents: 75 pre-moistened wipes, 7" x 9.5"
Minimum Wt.: 1 lb.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OR HBV OR HCV ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS.

Personal protection: Clean-up must always be done wearing protective gloves, gowns, masks and eye protection.

Cleaning procedure: Blood and other body fluids containing HIV or HBV or HCV must be thoroughly cleaned from surfaces and objects before application of **Protex Ultra Disinfectant Wipes**. **Protex Ultra Disinfectant Wipes** can be used for this purpose.

Contact time: Leave surface wet for 1 minute (60 seconds) for HIV-1. Use appropriate contact time as listed on label for all other viruses and bacteria claimed.

Disposal of infectious material: Blood, body fluids, cleaning materials and clothing must be disposed of according to local regulations for infectious waste disposal.

SANITIZING:

To sanitize hard, non-porous, non-food contact surfaces; wipe surface using enough wipes to allow treated surface to remain wet for 15 seconds for Staphylococcus aureus, Klebsiella pneumoniae, and Campylobacter jejuni. Let surface dry. For highly soiled surfaces, clean excess dirt first.

This product is not to be used as a terminal sterilant / high level disinfectant on any surface or instrument that is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Protex Ultra Disinfectant Wipes Kills:

Bacteria
Acinetobacter baumannii, Corynebacterium ammoniagenes, Campylobacter jejuni, Staphylococcus aureus - Community Associated Methicillin-Resistant [CA-MRSA] [NRS384] [USA300], Enterobacter aerogenes, Enterococcus faecalis-Vancomycin Resistant [VRE], ESBL Escherichia coli, Klebsiella pneumoniae, Klebsiella pneumoniae (NDM-1), Legionella pneumophila, Pseudomonas aeruginosa, Burkholderia cepacia, Salmonella enterica, Salmonella schottmuelleri, Salmonella typhi, Serratia marcescens, Shigella dysenteriae, Staphylococcus aureus - Methicillin-Resistant [MRSA],

Staphylococcus aureus-Multi-Drug Resistant, Staphylococcus aureus-Vancomycin Intermediate Resistant – [VISA], Streptococcus pyogenes, Vibrio cholerae

Viruses

*Hepatitis B Virus [HBV], *Hepatitis C Virus [HCV], *Herpes Simplex Virus Type 1, *Herpes Simplex Virus Type 2, *HIV-1 [AIDS Virus], *Human Coronavirus, *Influenza Type A/Brazil Flu, *Respiratory Syncytial Virus [RSV], *SARS Associated Coronavirus [SARS], *Vaccinia Virus

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

Caution. Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

FIRST AID

IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling poison control center or doctor or going for treatment.

STORAGE AND DISPOSAL

Store in original container in areas inaccessible to children. Keep securely closed. Nonrefillable container. Do not reuse or refill this container. Discard empty container in trash or recycle (if available).

Questions? Comments?
Call 1-800-631-8888



MANUFACTURED FOR:



Parker Laboratories, Inc.
286 Eldridge Road
Fairfield, NJ 07004

www.parkerlabs.com

LA-048-0001D

protex disinfectant wipes
Ultra