### USE PROTEX™ DISINFECTANT SPRAY

- · As a disinfectant on hard, nonporous surfaces
- For disinfecting of ultrasound transducers, probes, mammography compressor plates and other hard nonporous surfaces
- To clean and disinfect tables, chairs and countertops
- Use on nonporous athletic mats, wrestling mats, gymnastic mats, exercise equipment and training tables

### USE IN:

- Hospitals Medical Related Facilities Athletic Facilities
- Exercise Facilities

### **USE ON**

Washable, hard, nonporous surfaces of: walls - counters - ultrasonic baths Other hard nonporous surfaces made of: laminated surfaces - metal - plastic - stainless steel - vinyl and plastic upholstery - finished woodwork

Protex Disinfectant Spray is a one-step hospital use disinfectant, Bactericidal according to the current AOAC Use-Dilution Test Method, Fungicidal according to the AOAC Fungicidal Test and Virucidal\* according to the Virucidal qualification modified in the presence of 5% organic serum against:

Bacteria: Acinetobacter baumannii - Corynebacterium ammoniagenes - Burkholderia cepacia - Campylobacter jejuni - Enterobacter aerogenes - Enterobacter cloacae NDM-1 - Enterococcus faecalis-Vancomycin resistant-VRE - ESBL Escherichia coli - Escherichia coli - Escherichia coli, ew Delhi Metallo-Beta Lactamase (NDM-1) - Klebsiella pneumoniae - Klebsiella pneumoniae-NDM-1 positive - Legionella pneumophilia - Pseudomonas aeruginosa - Salmonella enterica - Salmonella schottmuelleri - Salmonella typhi - Serratia marcescens

Shigella dysenteriae • Staphylococcus aureus-Community
 Associated Methicillin-Resistant ICA-MRSAIINRS384I[USA300]

Staphylococcus aureus-Methicillin Resistant-MRSA

• Staphylococcus aureus Multi-Drug Resistant

Staphylococcus aureus-Vancomycin Intermediate
 Resistant-VISA • Streptococcus pyogenes • Vibrio cholerae

Viruses: \*Hepatitis B Virus (HBV) • \*Hepatitis C Virus (HCV) • \*Herpes Simplex Virus Type 1 • \*Herpes Simplex Virus Type 2

• \*HIV-1 (AIDS Virus) • \*Human Coronavirus

• \*Influenza Virus Type A / Brazil • \*2013 Influenza A Virus (H7N9)

\*SARS Associated Coronavirus • \*Vaccinia

### Fungi:

Trichophyton mentagrophytes • Aspergillus niger

### DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

### **Refill Directions:**

Remove top from empty spray bottle.

Unscrew cap on refill and pour contents directly into empty bottle until full. Replace trigger sprayer. If liquid remains in refill bottle, replace cap. Store for future use. Use product according to label directions.

### DISINFECTION / \*VIRUCIDAL / FUNGICIDAL DIRECTIONS:

Apply Protex Disinfectant Spray undiluted to hard, nonporous surfaces, thoroughly wetting the surfaces with a cloth, mop, sponge, sprayer or by immersion. Treated surfaces must remain wet for 10 minutes. For Influenza Virus Type A and Human Coronavirus, treated surfaces must remain wet for 1 minute. Wipe dry with a cloth, sponge or mop or allow to air dry. For heavily soiled areas, a preliminary cleaning is required.

For sprayer applications, use a coarse spray device. Spray 6 - 8 inches from surface; rub with brush, sponge or cloth. Avoid breathing spray.

Rinse all surfaces that come in contact with food such as countertops, appliances, tables and stovetops with potable water before reuse. Do not use on utensils, glassware and dishes.

For fungicidal activity: Protex Disinfectant Spray is an effective one-step fungicide against Trichophyton mentagrophytes when used on hard nonporous surfaces in areas such as locker rooms, dressing rooms, shower and bath areas and exercise facilities. Allow surface to remain wet for 10 minutes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

### ULTRASONIC BATH DISINFECTANT / \*VIRUCIDAL DIRECTIONS:

Use Protex Disinfectant Spray to disinfect hard nonporous non-critical objects compatible with Ultrasonic cleaning units. Pour Protex Disinfectant Spray directly into bath chamber, Pre-clean heavily soiled objects. Place objects into unit and operate for a minimum of 10 minutes, according to the manufacturer's directions. Remove objects and wash with sterile water (sterile water for injection) or allow to air dry. Use fresh product for each use.

REF 42-28



# **One-Step Disinfectant**

For Hospitals and Institutional Use

Effective in the presence of 5% serum contamination

- Kills Pandemic 2009 H1N1 Influenza A Virus

• Salmonellacidal • Fungicidal • Virucidal\*

Active Ingredients:

# KEEP OUT OF REACH OF CHILDREN CAUTION

See back panel for Precautionary Statements and First Aid

EPA Reg. No. 6836-152-82613 EPA Establishment No. 82613-NJ-001



### \*KILLS HIV-1 AND HBV AND HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS

in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids and in which the surfaces / objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1(HIV-1) (associated with AIDS) or Hepatitis B Virus (HBV) or Hepatitis C Virus (HCV).

# SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OR HBV OR HCV ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS.

**Personal Protection:** Clean-up should always be done wearing protective gloves, gowns, masks, and eye protection.

Cleaning Procedure: Blood and other body fluids containing HIV or HBV or HCV must be thoroughly cleaned from surfaces and objects before application of Protex Disinfectant Spray.

**Contact Time:** Leave surface wet for 1 minute (60 seconds) for HIV-1 and 10 minutes for HBV and HCV. Use a 10 minute contact time for disinfection against all other viruses, bacteria and fungi claimed.

**Disposal of Infectious Material:** Blood, body fluids, cleaning materials and clothing should be autoclaved and disposed of according to local regulations for infectious waste disposal.

### PROTEX™ DISINFECTANT SPRAY

Is a one-step Hospital Use disinfectant cleaner that is effective against a broad spectrum of bacteria, is virucidal\* including HIV-1, the AIDS Virus and Hepatitis B Virus and Hepatitis C Virus, and fungicidal when used as directed.

Meets AOAC efficacy standards for hospital Disinfectants

## PRECAUTIONARY STATEMENTS HAZARDS TO HUMANS AND DOMESTIC ANIMALS

Caution. Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing qum, or using tobacco.

## FIRST AID

**IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing the eye.

Call a poison control center or doctor for treatment advice. Have the container or label with you when calling a poison control center or doctor, or going for treatment.

### STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

#### Pesticide Storage:

Open dumping is prohibited. Store in original container in areas inaccessible to children.

### Container Disposal:

Nonrefillable container. Do not reuse or refill this container.

Triple rinse (or equivalent). Offer for recycling or reconditioning or puncture and dispose of in a sanitary landfill, or incineration, or if allowed by state and local authorities, by burning. If burned, stay out of smoke.

Parker Laboratories Inc. 286 Eldridge Road Fairfield, NJ 07004

parkerlabs.com



Net Contents: ONE U.S. GALLON (3.8 liter)

LA-1GL-0010A