


## IMPORTANT INFORMATION

The instructions in this document refer to the Table Pad Cover with the following identification:

- **Manufacturer:** Domico Med-Device, LLC
- **Part number:** 9121
- **Product name:** Table Pad Cover
- **Type designation:** Patient Positioning
- **Lot Number:** See Identification Tag
- **Manufacture date:** See Identification Tag

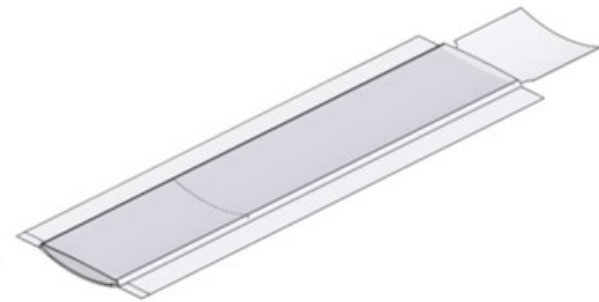


## 9121 Table Pad Cover Instructions For Use

Fits Part Numbers 9106F, 9107, 9108 and 9116.


**Step 1:**

Insert table pad into Table Pad Cover from foot end. Ensure table pad is fully inside of cover and Velcro on table pad as well as mating holes are on

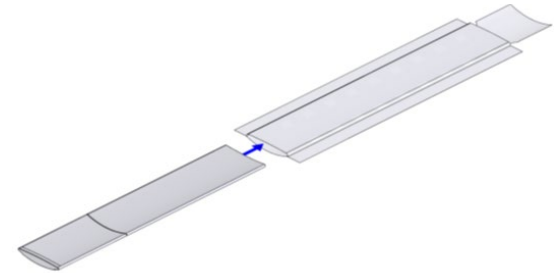

**Step 2:**

Locate end of table pad inside of cover and trim off excess cover about 5 inches from end of table pad. Dispose of extra material. \*Note that removal of

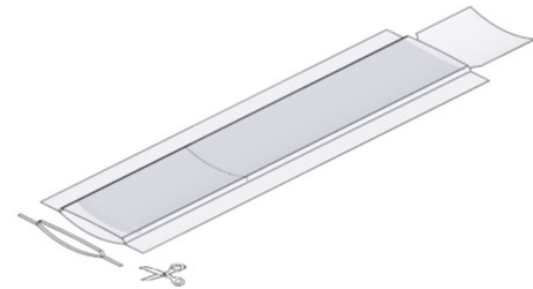

**Step 3:**

Fold Table Pad Cover under table pad to close end. \*Note that this will interfere with the first to second Velcro placement.

