



Instructions for Use

Elbow Arthroscopy Support 800-0168
&
Replacement Pad:
Universal Prepper Pad 508-0159

Copyright Notice

All rights reserved. Any duplication, adaption or translation without prior written consent is prohibited unless otherwise provided in relevant copyright laws.



Initial Release: 12/2/2011



Manufacturer
SchureMed (081001460)



Authorized Representative

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

GENERAL SPECIFICATIONS

Device Dimensions (maximum)

- *Maximum Support Height: 21" +/- 0.5"*
- *Minimum Support Height: 8.5" +/- 0.5"*
- *Maximum Height: 25.5" +/- 0.5"*
- *Overall Width: 11" +/- 0.5"*
- *Depth: 4.25" +/- 0.5"*
- *Net Weight: 3.2 +/- 0.5 lbs.*
- *Gross Weight: 8.3 +/- 0.5 lbs.*
- *5/8" diameter mounting post insert into socket*
- *Single-person installation*

INTENDED USE

Intended use is to hold patient's upper extremity comfortably during a surgical procedure.

INSTRUCTIONS

Become familiar with patient positioning device's features before use with patient. Always practice on a nurse, physician or appropriate volunteer prior to using clinically.

Attaching Elbow Arthroscopy Support to Surgical Table

1. *Attach Schure Socket XL P/N 800-0134 (sold separately) onto side rail*
2. *Insert mounting post into socket and lock tight*

Detaching Elbow Arthroscopy Support from Surgical Table

1. *Turn clamp handle counterclockwise to loosen and remove*

COMPONENT OVERVIEW

ELBOW ARTHROSCOPY SUPPORT is a surgical table accessory. With its padded horseshoe design, it is used to hold patient's upper extremity comfortably during a surgical procedure.

Other required products for use:

800-0134 Schure Socket XL (sold separately)

US: 0.374" x 1.122" (9.5 mm x 28.5 mm) PN# 800-0134

Denyer: 0.236" x 1.496" (6 mm x 38 mm) PN# 800-0134-DEN

Europe: 0.394" x 0.984" (10 mm x 25 mm) PN# 800-0134-EU

Eschmann (UK): 0.236" x 1.260" (6 mm x 32 mm) PN# 800-0134-UK

Japan: 0.354" x 1.260" (9 mm x 32 mm) PN# 800-0134-JPN

Swiss: 0.394" x 1.181" (10 mm x 30 mm) PN# 800-0134-SWISS

Replacement Pad:

508-0159 Universal Prepper Pad

GENERAL INFORMATION

- *Product not made with Natural Rubber Latex*
- *Device supports 500 lb. proportional patient load (6'4" tall patient per 99% human body model)*
- *Product warranty covers product from manufacturing defects for period of 2 years*
- *Product is maintenance-free, check product condition before next use*
- *Life of device is 5 years under normal use*
- *Store device between -4°F to +86°F*

DISPOSAL

- **General** - *Prevent infection by cleaning and disinfecting product before disposal*
- **Packaging** - *Dispose packaging material via household waste according to national requirements*
- *SchureMed accepts back used or retired products - or dispose of product in accordance with national requirements*



Product Use Warnings

WARNING!

Maximum load should not exceed appropriate proportion of a patient weighing 500 lbs. Use care with low-maximum load capacity surgical tables that accessory rails are not overloaded.

**WARNING!**

Hazard resulting from incorrect use. Strictly follow Instructions for Use with your Operating Table system.

**WARNING!**

Surgical table load capacities may be less. Never overload surgical table. Device is intended for mounting on side rail of surgical table only.

CLEANING RECOMMENDATION

Follow current Association of periOperative Registered Nurses Journal Guidelines for proper cleaning and disinfection procedures.

**WARNING!**

Adhere to standards for blood-borne pathogens from the Occupational Safety and Health Administration. Use recommended protective clothing, gloves, masks and eye protection to clean accessory.

CAUTION

Strictly read/follow manufacturer's directions for cleaning fluids. DO NOT use cleaners containing phenolics.

- 1. Remove major contaminants from accessory with disposable materials. Follow appropriate bio-hazard waste disposal procedures.*
- 2. Apply cleaning fluid liberally to entire accessory and wipe with clean, lint-free cloth until all moisture and cleaning fluid is removed from accessory*
- 3. Let accessory dry*



Symbol Explanation

Date of Manufacture



Conformity Mark



Manufacturer



Batch Code/Serial Number



Authorized Representative
in European Community

IFU-800-0168 REV 3.04