

# SONO<sup>®</sup>

HEALTHCARE

Trusted by Hospitals. Loved by Everyone.

## Manufacturer Information for Use

### SONO Disinfecting Spray

## 32oz Hydrogen Peroxide Spray

---

**THIS PRODUCT IS A ONE-STEP DISINFECTANT, CLEANER** that is effective against a broad spectrum of bacteria and is virucidal\* including HIV-1, when used as directed.

**Use on washable hard, nonporous surfaces in:**

homes, healthcare facilities, restaurants, exercise facilities, hotels, playground equipment, airports, airplanes, boats, cars, trains and emergency vehicles.

**FOR USE ON WASHABLE HARD, NON-POROUS, NON-FOOD CONTACT SURFACES OF** exteriors of appliances, bathroom fixtures and tubs, countertops, desks, doorknobs, floors, highchairs, outdoor furniture, showers, sinks and telephones.

**Non-Critical Medical Equipment Surfaces:**

Beds and bedframes, crutches, defibrillators, gurneys, examining tables, rescue tools, resuscitators, scales, stretchers, stethoscopes, walkers and wheel chairs.

This product is a Hospital Disinfectant, tested using the current AOAC Germicidal Spray Test method and to the ASTM Virucidal test method for disinfection on inanimate surfaces.

**This Product is Effective as a Disinfectant Against:**

**BACTERIA:** Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella enterica, Enterobacter aerogenes, Escherichia coli, Klebsiella pneumoniae, Methicillin Resistant Staphylococcus Aureus [MRSA],

**VIRUSES\*:** \*Herpes Simplex Type 2, \*HIV-1, \*Influenza A/Hong Kong, \*Norovirus

**ANIMAL VIRUSES\*:** Feline Calicivirus.

**DIRECTIONS FOR USE:** It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

**TO OPERATE:** Turn nozzle to on position and spray.

**GENERAL CLEANING / DEODORIZING DIRECTIONS:** Apply to soiled area then wipe clean. Repeat for heavily soiled areas.

**DISINFECTION / VIRUCIDAL\* DIRECTIONS:** Spray 6 - 8 inches from surface, until surfaces are thoroughly wet. Do not breathe spray. Treated surfaces must remain wet for 3 minutes. Wipe dry with a clean cloth, sponge, mop or towel, or allow to air dry.

**MOLD AND MILDEW CONTROL DIRECTIONS:** This product effectively inhibits the growth of mold and mildew, and the odors caused by them, when applied to hard, non-porous surfaces. Spray 6 - 8 inches from surface, until surfaces are thoroughly wet. Do not breathe spray. Allow to air dry on surface. Repeat every 7 days, or when mold or mildew growth returns. This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. (This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.)

---

---

**NON-ACID TOILET BOWL AND URINAL CLEANER DIRECTIONS:** Apply this product to exposed surfaces including under the rim with a sprayer. Use a toilet brush to scrub the bowl. Repeat and allow to stand for 1 minute, then flush.

**\*KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES<sup>1</sup> / OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings, or other settings, in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids for transmission of Human Immunodeficiency Virus Type 1 (HIV-1).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS.**

**PERSONAL PROTECTION:** Clean-up must always be done wearing protective gloves, gowns, masks and eye protection.

**CLEANING PROCEDURE:** Blood and other body fluids containing HIV must be thoroughly cleaned from surfaces and objects before application of this product.

**CONTACT TIME:** Leave surface wet for 3 minutes.

**DISPOSAL OF INFECTIOUS MATERIALS:** Blood, body fluids, cleaning materials and clothing must be

**STORAGE AND DISPOSAL**

**STORAGE AND DISPOSAL** Do not contaminate water, food or feed by storage or disposal. Pesticide Storage: Open dumping is prohibited. Store in original container in areas inaccessible to children. Container Disposal: Nonrefillable container. Do not reuse or refill this container. Triple rinse (or equivalent). Offer for recycling or reconditioning or puncture and dispose of in a sanitary landfill, or incineration, or if allowed by state and local authorities, by burning. If burned, stay out of smoke.



Trusted by Hospitals. Loved by Everyone.

---