

160-1101 FlexiStore™ Flex Fill Earloop Face Mask Dispenser - 1 Box
160-1103 FlexiStore™ Flex Fill Earloop Face Mask Dispenser - 3 Box
160-1106 FlexiStore™ Side Fill Earloop Face Mask Dispenser - 6 Box
160-1107 FlexiStore™ Side Fill Earloop Face Mask Dispenser - 6 Box + 1 Tissue

Our Flex/Side Fill Earloop Face Mask Dispensers are PETG dispensers that hold from 1 to 6 boxes of earloop face masks; one model also holds either 1 standard or 2 cube boxes of facial tissues. These dispensers are designed to be mounted on a flat, structurally solid surface using screws secured into at least 2 pre-cut keyholes of the rear panel. We supply screws and anchors in the package that are suitable for 1/2" thick drywall. Be sure to use a screw and anchor appropriate for the surface where you plan to install the dispenser.

Recommended Tools: 3/16" brad point drill bit, cordless drill, Phillips head screwdriver, level

Installation Instructions

1. Choose a convenient location for the dispenser and clean it if necessary.
2. Review the included Alignment Guide and note the location of the keyholes.
3. On the solid surface where you intend to place the dispenser, draw a level or a plumb line based on the dispenser you're mounting. Refer to the diagrams on the Alignment Guide to determine the length of the line you need to draw. Based on the distance shown on the Alignment Guide, mark 2 points along the line that matches the dispenser you're mounting.
4. Press your brad point drill bit or screw tip into the center of the points you just marked.
5. **Taking care not to hit wires, pipes, or other obstructions**, drill a pilot hole in each of your marked locations sufficient to pre-drill the screws or to install the anchors. If you're using the supplied anchors, press 2 anchors fully into these holes until they are flush with the surface.
6. Completely remove the clear protective outer film from the dispenser.
7. Insert 1 screw into each of 2 keyholes and screw the dispenser into your pre-drilled holes or anchors. Let the shafts of the screws slip into the upper notches of the keyholes. Tighten the screws to draw the dispenser flush with your surface – but don't fully tighten the screws at this point.
8. Check that the dispenser is level and plumb; adjust as necessary.
9. Once you've confirmed the dispenser is level, fully tighten all screws. **Be careful not to over-tighten the screws so as not to crack the PETG material.** Take pride in a job well done!

Cleaning Instructions

To clean our PETG-based products, the best approach is to use warm, soapy water. A 3 - 5% solution of hydrogen peroxide or a 70% solution of isopropyl alcohol can also be used to clean PETG without risk of damage to the dispenser. Diluted household bleach (1/3 cup of bleach per 1 gallon of water) from a supply made weekly is also safe; diluted bleach is not stable for long periods of time. Do **NOT** use harsh disinfectants such as Lysol, Shockwave, or ZEP. Do **NOT** use cleaning fluids with acetone, ammonia, carbon tetrachloride, denatured alcohol, or gasoline. To reduce the risk of scratching the surface of the dispenser, only use a soft cloth for cleaning. Like most plastics, PETG can be easily scratched by steel wool or abrasive sponges; therefore, do **NOT** use abrasive wipes or sponges on your dispenser.

Product Warranty

Subject to normal use consistent with the installation and cleaning instructions supplied with this product, we warrant that it shall be free from defects for a period of one (1) year from the date of purchase. Should the product exhibit a defect during this one (1) year period, upon confirmation of the defect, we in our sole and absolute discretion will repair or replace it at our expense. Purchaser is responsible for all return shipping costs, if any. Our liability is strictly limited to the cost of repairing or replacing the product with a similar product in new condition. Our liability does **NOT** include consequential, special, incidental, or indirect damages in any form or amount.


To file a warranty claim, please visit <https://www.brandedsafety.com/warranty/>

Alignment Guide


The diagrams below represent guides to help you place screws to mount your dispenser to a solid surface.

IMPORTANT: Mount your dispenser using at least 2 keyholes in a straight line. The 160-1101 & 160-1103 include universal keyholes that enable you to mount the dispenser so that it fills from the right, top, or left. The diagrams below reflect the dispenser as it would be filled from the right.

**160-1101 FlexiStore™
Flex Fill Earloop
Face Mask
Dispenser - 1 Box**

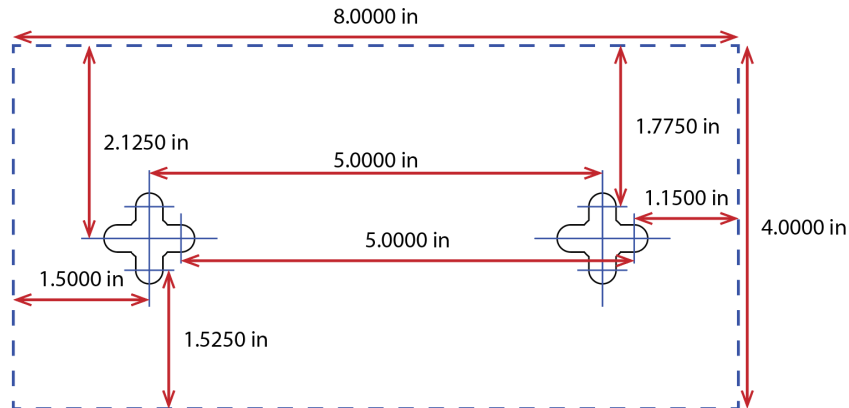

**160-1103 FlexiStore™
Flex Fill Earloop
Face Mask
Dispenser - 3 Box**

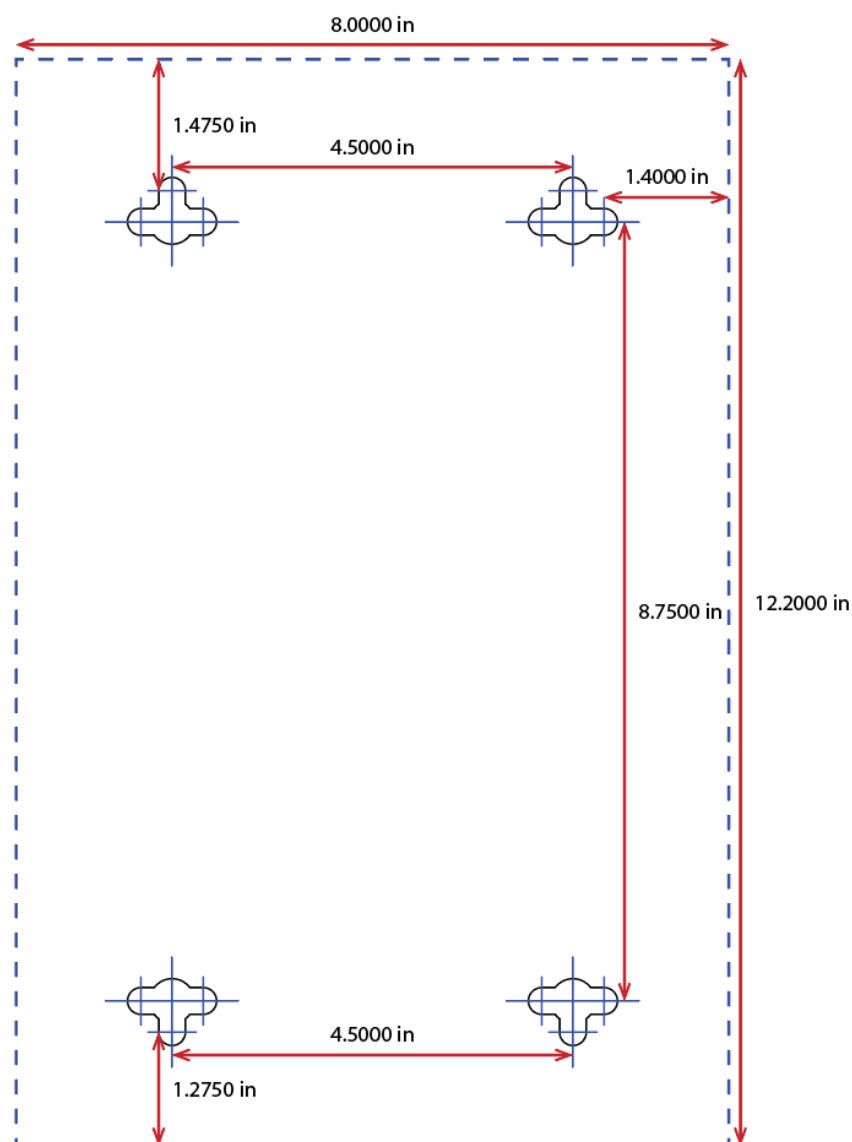


Alignment Guide

The diagrams below represent guides to help you place screws to mount your dispenser to a solid surface.

IMPORTANT: Mount your dispenser using at least 2 keyholes in a straight line. The 160-1106 & 160-1107 fill from both the right and left sides. Additionally, on the 160-1107, facial tissues fill from the top.

160-1106 FlexiStore™
Side Fill Earloop
Face Mask
Dispenser - 6 Box

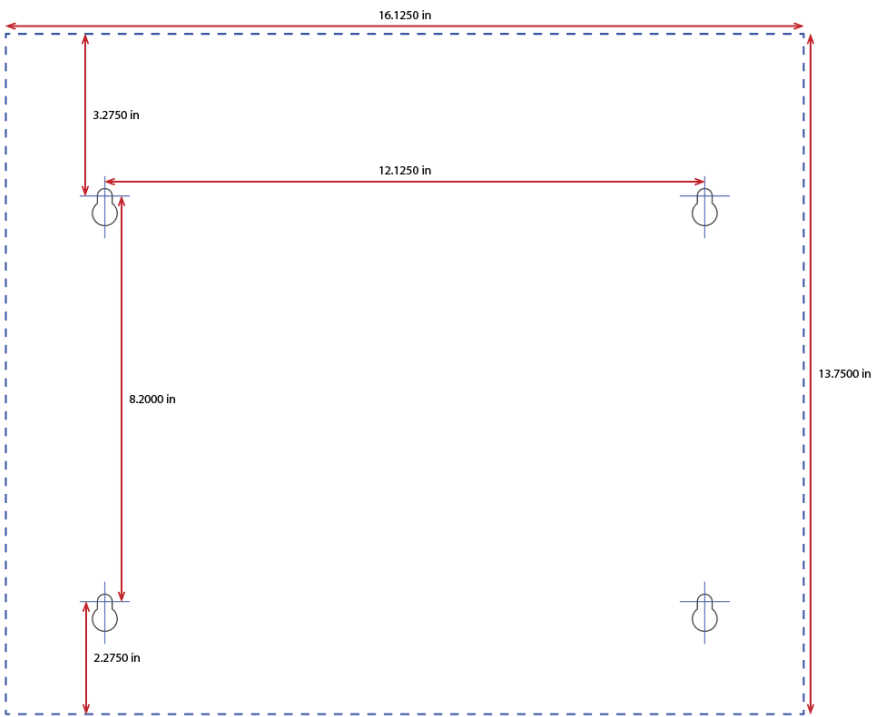

160-1107 FlexiStore™
Side Fill Earloop
Face Mask
Dispenser - 6 Box + 1
Tissue

