

NOTE: Reference the *Cleaning Process* section in the Product Manuals for further instruction. Cleaning and disinfecting an OAKWORKS product is a three part process.

1. Clean any visible soil off the surface.
2. Disinfect surface per manufacturer's recommended contact time.
3. Rinse off disinfectant with clean water and thoroughly dry with a clean cloth or towel.

CLEANERS:

- Formula 409 (EPA No.: 67619-10)
- Fantastik (EPA No.: 89900-3)
- Windex Multisurface (EPA No.: 4622-593)

DISINFECTANTS:

- Formula 409 (EPA No.:67619-10)
- Parker Labs Protex (EPA No.: 6836-152)
- Madacide-FD (EPA No.: 1130-15)
- Clorox EZ Kill Wipes (EPA No.: 5984-10)
- Clorox Disinfecting Wipes (EPA No.: 67619-9)
- Clorox Healthcare VersaSure Wipes (EPA No.: 67619-37)
- Clorox Healthcare Fuzion (EPA No.: 67619-30)
- AntiGone Wipes (SC 291919-10)
- PureGreen 24 (EPA No.: 72977-3)*
- Rejuvenate Disinfectant Wipes (EPA No.: 74559-3)
- Kurland Fog It IQ

*This list supersedes all previous versions
* Recommended for upholstery only*

A NOTE ON BLEACH:

- 10%, or 1:10 dilution ratio, of sodium hypochlorite (household bleach) is an effective disinfectant but not an effective cleaner.
- Additional cleaner must be used for initial cleaning.
- Bleach must be rinsed with clean water after disinfection.

NOT RECOMMENDED:

- Citrus based cleaners
- Acetone
- Hydrogen Peroxide based cleaners/disinfectants
- Acetic Acid
- Phenolics
- Higher concentrations of bleach
- Unapproved quaternary ammonias ("quats")

All cleaners and disinfectants have the ability to degrade and damage the upholstery. This information is not guaranteed, however, following the recommended cleaner and disinfectant list and cleaning process will provided the best care for your OAKWORKS products.

**FAILURE TO FOLLOW RECOMMENDATIONS WILL VOID
THE WARRANTY ON THE PRODUCT.**