

(EN) Start-Up Guide for your qubeAVgo software.

Before you can start your training with your new SKILLQUBE devices you need to take these following steps to finish setting up your new hardware

Login data for the qubeCLOUD:

As a qubeSERIES customer you'll receive an e-mail to start you off with your qubeCLOUD login data. Please choose a password for your qubeCLOUD account before you start to set up your devices.

Software set up on your hardware

1. open settings and log in with your Apple-ID or create a new one to continue setting up your devices.
Scan the QR-code for detailed instructions →
2. After logging in to all your devices go to the App-Store and search for SKILLQUBE.
3. Download the qubeAVgo-App for your debriefing-iPad.

