

Large Printer Shelf

Dimensional Illustrations and Installation Instructions

SKU Number: FG9M06040501

Product Description

The Large Printer Shelf provides a secure and easily accessible way to mount label and 8.5" x 11" sheet printers.


Features and Benefits

- Integration of a printer on mobile computing cart improves work-flow efficiency by allowing printing directly at the bedside.
- Printer height remains constant during work surface height adjustment.
- Ability to hold up to 20 lbs provides a multi-functional surface area.

Specifications

Accepted on all M40, M38, M38 XP, and M38 RX Carts.

Dimensions: 3.90" h x 15.04" w x 9.60" d
Weight: 4.6 lbs

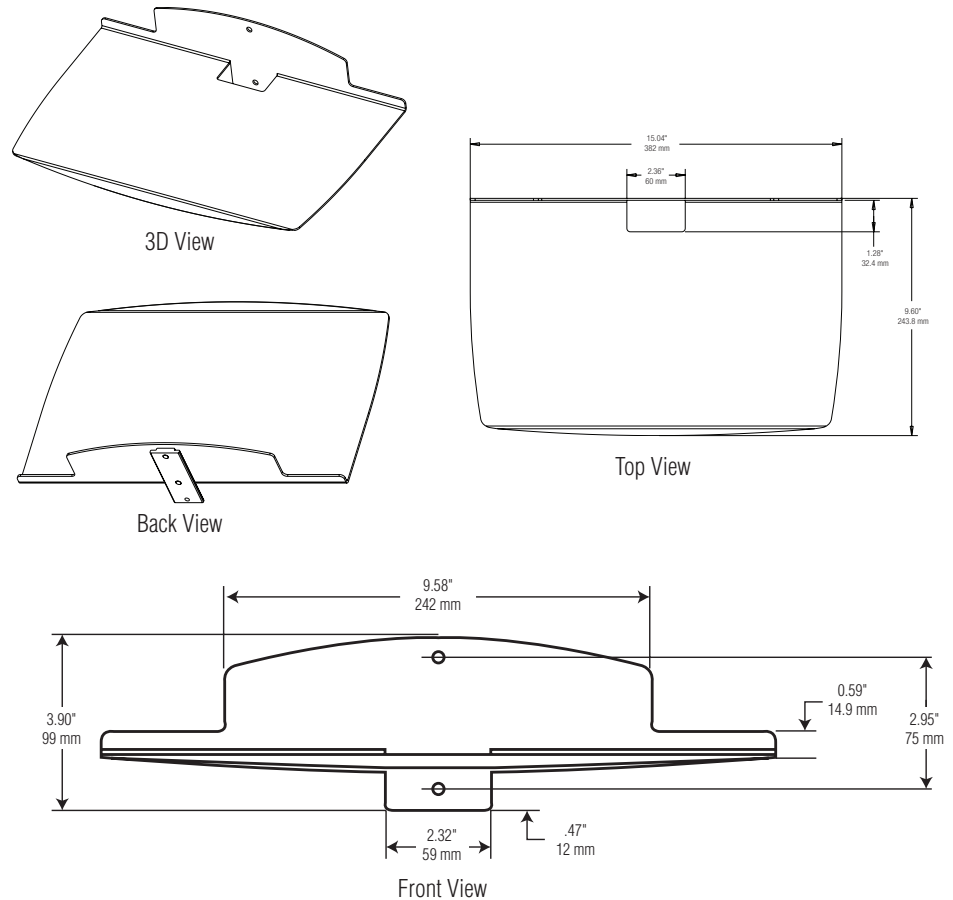

Installation Instructions

Note: (R) refers to Rear t-track mounting and (F) refers to Front t-track mounting.

1. (R) Carefully remove the shuttle cap, loosen the screws holding the rear handle on the t-track. Slide the rear handle assembly up the t-track and set aside.
2. (R) Slide the t-nut down the t-track until the shelf is in the desired location.
(F) Slide the t-nut up the t-track until the printer shelf is in the desired location.
3. Tighten the screws using a 4mm hex driver.
4. (R) Replace the rear handle and shuttle cap.

Printer Integration

1. Secure the printer to the shelf using Velco or double-sided tape.
2. Run the cables into the technology cavity. Ensure enough excess cable is left out of the tech tray to accommodate work-surface height adjustment.
3. Using the integrated power strip, plug in the printer and test for power.
4. Use optional Cable Clips to conceal any excess cables not left for height adjustment.

Mounting Location

- M38 XP and M38 RX: Back t-track
 - M38: Option of front (under the work surface) or back t-tracks
- Note:** Cart will maintain original footprint.

