



2020

We, at the Harloff Company, strive to provide the highest quality products and instruction available. Below you will find a list of standard cleaning solutions that may be used on our line of powder coat painted and stainless steel carts and cabinets, as indicated below.

Harloff carts are not designed for autoclave or other high temperature machines.

***Metal Powder Coated Surface:*** Sani-Cloth Plus Wipe, Glance, Oxivir Five, Hydrogen Peroxide (3.0%), Virex II, Lysol Wipes, Alcohol Wipes

***Glass Surface:*** Windex, Glance

***Polystyrene, Polypropylene, High Density Polyethylene:*** Sani-Cloth Plus Wipe, Alcohol Wipes, Lysol Wipes

***PVC Trim/Rubber Extrusion:*** Windex, Glance, Sani-Cloth Plus Wipe  
**(NO BLEACH PRODUCTS)**

***Acrylic/Polycarbonate:*** Soap and Water Only  
**(NO BLEACH PRODUCTS)**

Please feel free to contact us if you have any further needs or require information on a cleaner not listed above. We will be happy to look into a product with a copy of the MSDS sheet you provide.