



## **P-6000 P-7000 Surgeons stool operating instructions:**

**Armrest height adjustment:** Rotate knob located on “outside” of arm until armrest is at desired height. Armrest will stay locked at that height until readjusted.

**Armrest swivel adjustment:** While seated in the stool, swivel arms forward to desired position. Armrests swivel independently, and will NOT lock at desired position.

**Back adjustment:** Stand behind the stool, loosen knob located at the base of the seat, push or pull the back to adjust it to desired position & tighten knob securely. (Height & angle of back cannot be adjusted)

**Seat height adjustment:** While seated in the stool, step down on the foot pedal until it stops, release foot pressure & the pedal will return automatically. Repeat “pumping” the foot pedal until seat raises to desired height. To lower the seat, use the heel of the foot to “lift” the pedal (pivot the pedal in the opposite direction) toward the vertical cylinder.