




# Safety Needle Holder *Instructions for Use*

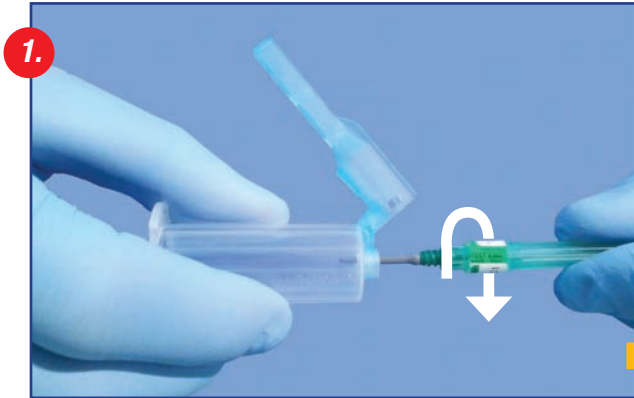


1. Screw the needle completely onto the holder. Remove cap from needle.

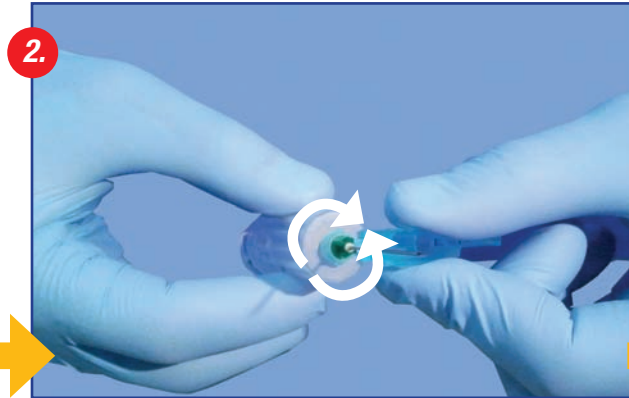


2. The safety shield can be rotated for comfortable blood collection. Simply grasp the end of the shield and rotate.

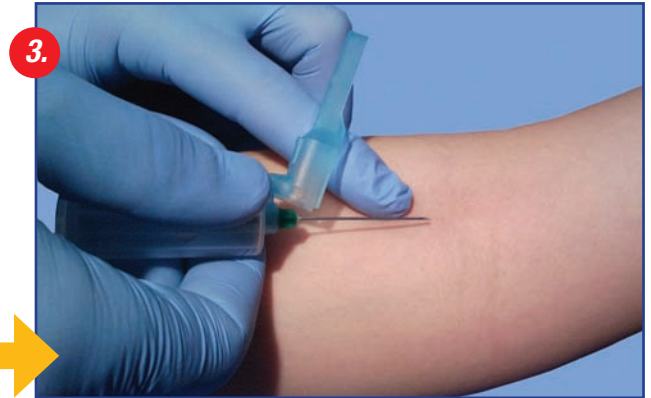


3. Perform blood collection according to established safety procedures.

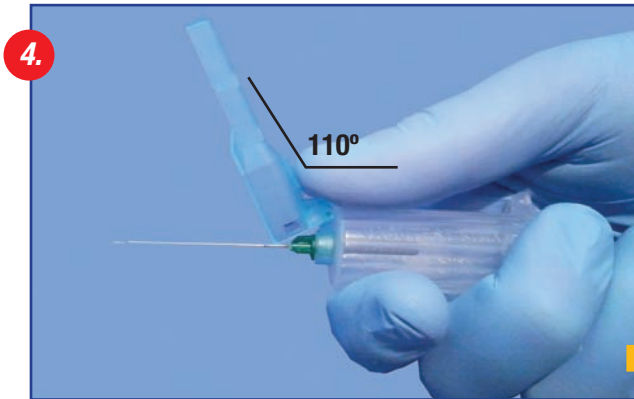

4. Immediately after blood collection, swivel the safety shield over the needle to prepare for closing.

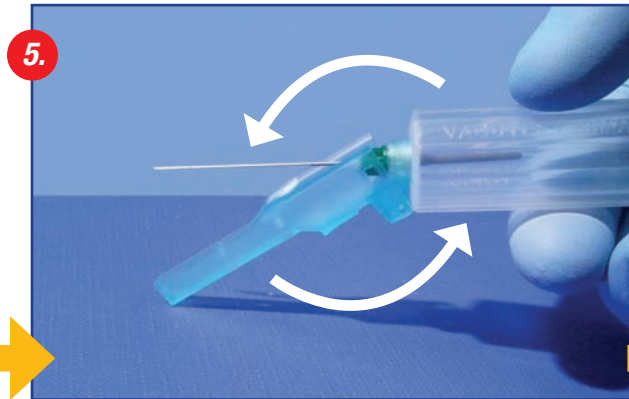

5. Close by pressing the safety shield against a hard surface.

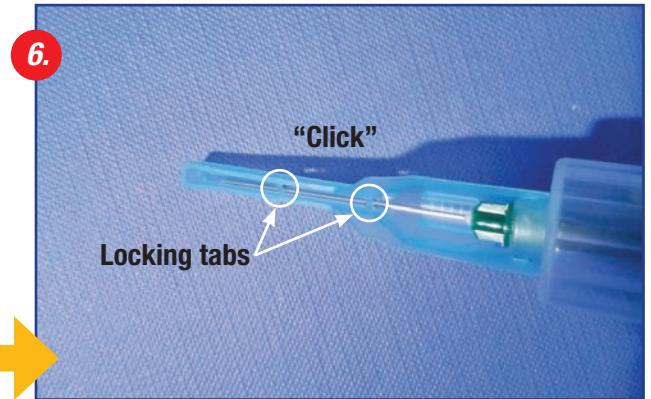

6. When a "click" is heard the needle is secured in place by the locking tabs inside the safety shield.



7. Dispose of the blood collection device into a Sharps container.  
**NEVER reuse the device after initial use.**

- *Complies with OSHA directives by preventing removal of needle and re-use of holder*
- *Fits both 13mm and 16mm blood collection tubes and all common needle units*

**globe**<sup>®</sup>  
SCIENTIFIC INC.