

# CALIBRATION

**This scale has been factory calibrated, and does not require calibration prior to use.** If required, the scale can be calibrated. Calibration of your scale is performed using kilograms (KG). Twenty (20) kilograms are required for calibration. Calibration is performed in kilograms even if the scale is set in weigh in pounds (LB).

**Note: Only weights certified and traceable to national standards should be used for calibration procedures.**

ACTION	DISPLAY SHOWS
Press <b>ON • TARE • OFF</b> to turn on the scale. “88:8:8.8” will appear on the display, quickly tap <b>ON • TARE • OFF</b> twice, then hold down the button until “CAL” appears in the display. Release the button when “CAL” appears.	“88:8:8.8” followed by “CAL”
Press and hold <b>ON • TARE • OFF</b> until 5 digits appear in the display. Release the button when the 5 digits appear.	“XXXXX”
Ensure there is no weight on the scale, then press <b>ON • TARE • OFF</b> again. “20.00” will appear in the display.	“20.00”
Place 20 kg on the scale, then press <b>ON • TARE • OFF</b> . “PASS” will appear in the display. Remove the 20 kg. The scale will restart and return to “0.0”	“PASS”, then the scale will restart.