


1 REMOVE ITEMS FROM THE BOX

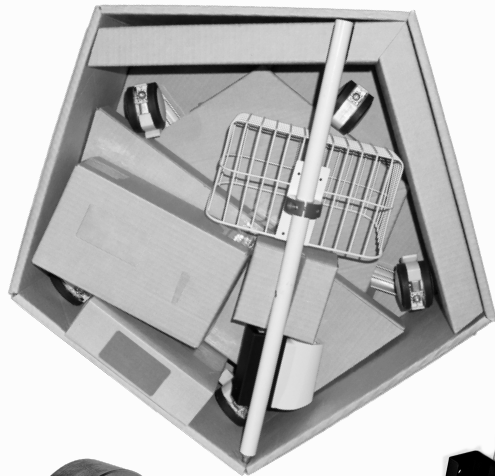

2. Pole



3. Arm



4. Base



5. Tools (not shown)

THE NEXTVEIN STAND ARRIVES SUBSTANTIALLY PRE-ASSEMBLED

2 INSERT THE POLE FIRMLY INTO THE BASE



3 PLACE BOLT THROUGH THE WEIGHT AND THE BOTTOM OF THE BASE AND CONNECT TO THE POLE. A HEX WRENCH IS PROVIDED



4 CONNECT THE FLEX ARM TO THE TOP OF THE POLE



IMPORTANT

PULL THE BLACK CONNECTOR UP BEFORE CONNECTING THE FLEX ARM TO THE TOP OF THE POLE



ONCE THE FLEX ARM IS SEATED ON THE POLE, MAKE CERTAIN TO MOVE THE BLACK CONNECTOR DOWN TO LOCK IN PLACE



S800NV ASSEMBLY INSTRUCTIONS

NEXTVEIN