


## Sterisystem® closed Stericart



Our Sterisystem® closed, washing-tunnel-resistant stainless steel Stericarts are manufactured based on the double DIN standard dimensions of 12 x 24 inch, with the idea of being able to use 2 DIN positions one after the other. The 2 inch extra depth at the back means that this transport cart can also be used for sterile transport of slightly longer sets. These Belintra Sterisystem® closed Stericarts are characterized by yellow runners, plastic bumpers, handlebars and latch.

The low model (43 inch) can double as a sterile field in the operating theatre. This case cart is a universal space-saving transport device. It is suitable for 2 DIN wire baskets or shelves or 1 ISO wire basket or shelf per level. The new "Extra-Wide" model can serve to transport packages up to 37 x 24 inch.

The Sterisystem® washing-tunnel-resistant and shock-resistant closed Stericarts with their unique space-saving design are perfect for safely transporting sterile instruments from and to the operating theatre and contaminated sets to the sterilizing unit.

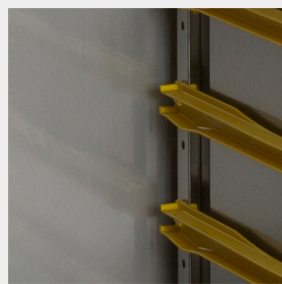

### » Wash tunnel resistant


The Sterisystem® closed case carts are designed in such a way to prevent any residual moisture after drying. The synthetic runners are also wash tunnel resistant\*. Unlike stainless steel runners, the high quality plastic ensures smooth running and substantial noise reduction when sliding stainless steel transport baskets or trays or plastic modules in and out. Moreover, the synthetic runners protect wrapped instrument trays against perforations and absorb noise during transport.



### » Ergonomic

The doors of these stainless steel transport carts swivel open to 270° and click onto the side walls with magnets when in the open position. This ensures that the cart is easy to access.

The high model still allows the user to see over the top of the cart during transport. The low model can also be used as a case cart with a sterile field in the operating theatre.

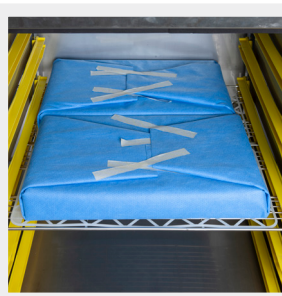

### » Optimum loading and space-saving

The high case carts are equipped with 9 pairs of runners and the low case carts with 6 pairs. Thanks to the high number of positions, the loading is optimised without losing out on space.

Transporting the instruments on DIN Perfo-Safe® wire shelves or in DIN Perfo-Safe® wire baskets allows you to double the capacity per level.



## » Safety

The cart can be sealed at the front with a breakable seal, to guarantee sterility from the sterilization unit to the operating theatre and also avoid contamination during the transport back.

It is recommended that the packed instrument baskets are placed on silicone anti-slip mats to prevent movement during transport and to reduce noise levels.



## » Identification

The closed Stericarts feature a document holder on the side (A4) as standard to ensure that the carts and their contents can be identified from the outside.



## » Castors

The transport carts are equipped with four double Tente® swivel castors with a diameter of 5 inch. Both wheels on the front have a brake and directional lock. The castors are equipped with a wheel hub on stainless steel bearings.

The carts are equipped with four bumpers to prevent damage to doors and walls.



## » Cart washer resistant magnets

The Sterisystem® stainless steel transport cart is equipped with 4 sets (2 sets per door) of cart washer resistant neodymium magnets which are embedded in a silicone housing. Neodymium magnets are the strongest magnet on earth.

The design of the silicone housing allows for a perfect alignment of the magnets resulting in optimal connection power between the magnets and guaranteeing that the doors will remain open during the cleaning process in the cart washer.



## » Handlebars

The low Sterisystem® transport carts are equipped with 1 horizontal yellow handlebar along each side, whereas the high carts are equipped with 2 vertical handlebars on each side to ensure smooth manoeuvrability.





## » Status display

These transport trolleys are equipped with a status display where "green" indicates that the trolley contains clean, sterile goods. "Red" indicates that the cart is returning after use in the O.R.

Drying instructions: 8 minutes at 185°F in dry air. Dry air is air with a relative humidity of maximum 60% at an ambient temperature of 70°F. Converted this is a relative humidity of maximum 1,1% at 185°F.

## » Sterisystem® closed Stericarts



### » Sterisystem® closed Stericart - Low

Article number	Runners	Weight	Ext. Dimensions
19031 00039	Equipped with 6 pair of nylon runners	128 lbs	28 <sup>31/32</sup> x 28 <sup>31/32</sup> x 43 <sup>5/16</sup> inch



### » Sterisystem® closed Stericart - High


Article number	Runners	Weight	Ext. Dimensions
19031 00038	Equipped with 9 pair of nylon runners	172 lbs	28 <sup>31/32</sup> x 28 <sup>31/32</sup> x 62 <sup>63/64</sup> inch




### » Sterisystem® closed Stericart - XW

Article number	Columns	Weight	Ext. Dimensions
19031 00048	Equipped with 9 pair of nylon runners for XW Wire Shelves	209 lbs	42 <sup>43/64</sup> x 28 <sup>31/32</sup> x 62 <sup>63/64</sup> inch


## » Possible filling (model 24 x 24 inch) per pair of runners

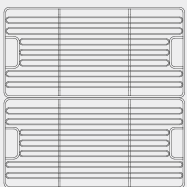


**2 x DIN wire shelves**  
24 x 12 inch  
or  
**1 x ISO wire shelf**  
24 x 16 inch

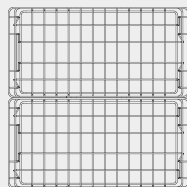


**2 x DIN wire baskets**  
24 x 12 inch  
or  
**1 x ISO wire basket**  
24 x 16 inch

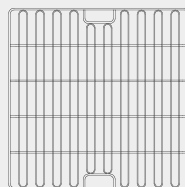

**1 x double DIN wire shelf**  
24 x 24 inch

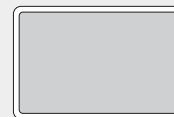

**1 x PC or ABS module or basket**  
24 x 16 inch

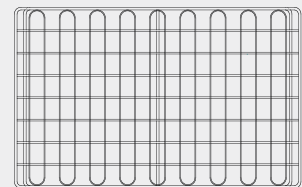

**XW wire shelf**  
24 x 37 inch

(Only for use in the top compartment of the Sterisystem® transport cart - XW)