



How to use the **SafetySure® Universal Thigh Strap**:



Place gait or transfer belt around the waist of the patient. Slide the Universal Thigh Strap under the patient's thigh.

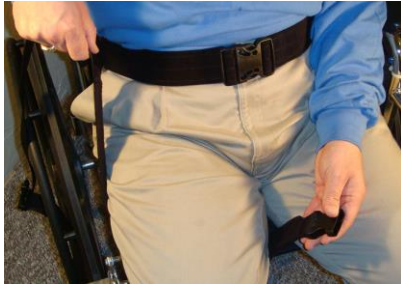

Take the end from the outside of the patients thigh and thread between the gait / transfer belt and the patients body.

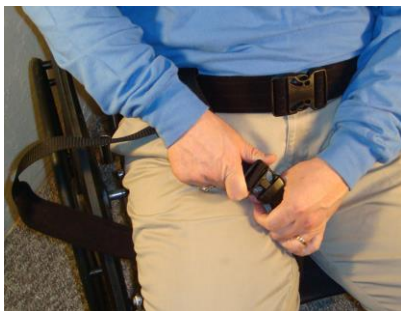

Bring the strap from the patients inner thigh and secure with the plastic buckle.



Adjust the position and tension of the **SafetySure® Universal Thigh Strap** as needed.



Repeat the procedure on the other thigh.

You are now ready to lift and transfer the patient. The gait / transfer belt and the **SafetySure® Universal Thigh Straps** can be left in place if the patient requires frequent transfers.