

### Case Cart Cleaning Instructions

<b>Disinfection:</b>	<ul style="list-style-type: none"> <li>Disinfection may be achieved by thermal means as described in the standard wash cycle below, or by use of hospital grade chemical disinfectants. If chemical disinfectants are used, it is recommended that all surfaces be thoroughly rinsed or wiped with a damp cloth after the required exposure time. Follow the disinfectant manufacturer's instructions for use.</li> </ul>
<b>Cleaning (Automated):</b>	<ul style="list-style-type: none"> <li>Machine cleaning is preferred and using only a washer disinfector cleared by the FDA.</li> <li>Be sure to follow the IFU's of the washer-disinfector and detergent manufacturers.</li> <li>Products should be positioned in the washer to allow maximum water penetration and drainage.</li> <li>The standard program to include, always refer to the washer manufacturer's IFU:             <ol style="list-style-type: none"> <li>Visual inspection is required to ensure complete removal of soil.</li> <li>If product still shows soil repeat program.</li> </ol> </li> </ul>
<b>Cleaning (Manual):</b>	<ul style="list-style-type: none"> <li>Manual cleaning is required only if cart is heavily soiled and cannot be cleaned by standard disinfection.</li> <li>Equipment: detergent, soft bristle brush, running water.</li> <li>Be sure to follow the IFU of the detergent manufacturer.</li> <li>Rinse excess soil from all surfaces.</li> <li>Apply detergent to all surface and use brush to clean all surfaces.</li> <li>Thoroughly rinse all surfaces of residual detergent and soil.</li> <li>Dry with lint free cloth.</li> </ul>
<b>Drying:</b>	<p>Drying may be accomplished during the dry-cycle of the automated washer and/or with the use of a lint-free, disposable towel.</p>