



POLY

User Manual

Read before use!



Please retain this document for the warranty and care of your Poly Cart. This serves as an official record for warranty purposes and as a certificate of authenticity

CERTIFICATE OF AUTHENTICITY

You have in your possession an original Poly from Pegasus Medical. All carts have patented designs crafted following extensive R&D and rigorous quality control processes. This document contains your cart's unique serial number, registered in our servers, showing our commitment to quality and reliability.

CLEANING GUIDE*

1. Clean using warm water and neutral detergent with a disposable cloth
2. Avoid using concentrated bleach, synthetic detergents, solvents, wax polishes, or aerosol sprays. [If you have any questions, please contact us]
3. Use diluted Bleach/ NaDCC (1000 ppm) or Hydrogen Peroxide solutions
4. Do not saturate — use a damp cloth only
5. Dry with a lint-free cloth after wiping with clean water
6. Avoid rubbing with abrasive cloths or tools
7. Do not wash in an automatic washing machine!
8. **Follow the hospital's Infection Control Guidelines for proper cleaning procedures**

* Related to polymer surfaces

SAFETY

1. Do not open more than one drawer at the same time
2. Move the cart carefully over obstacles to avoid tipping
3. Do not use the cart on stairs or outdoor surfaces
4. Do not force the cart to turn when the caster brake is locked
5. Push the cart using only the designated handle
6. Do not move the cart when any drawers are open
7. Do not move the cart by pushing or pulling accessories
8. Do not move the cart without unlocking the caster brakes
9. Attention! Remember to lock the caster brakes when you stop/park the cart
10. Attention! Remember to tighten screws when setting up parts and accessories
11. Do not overload the top of the cart
12. Do not overload the drawers
13. Attention! Always close the drawers
14. Do not park the cart on a ramp
15. Do not lean against any open drawers
16. Do not use excessive force to open drawers
17. Do not pour cleaning or hot liquids directly on the cart or drawers

ANTIMICROBIAL COMPONENTS

PP infused with a biocidal product containing Zinc pyrithione as an antimicrobial additive that acts against a wide range of bacteria, virus, fungi, molds, mildew and algae that may affect the product.

BIOCIDAL ZINC-BASED AGENTS

Antimicrobial efficacy	✓ ✓	After 24h
Antifungal efficacy	✓ ✓	
Antiviral efficacy	✓	After 2.5h
Safety	✓ ✓	

