



*Recommended Use & Care of Light Cloud™ Resposable Positioners*

Light Cloud™ positioners are Latex-Free

*CLEANING:* Use of standard operating room cleaning agents. Topical Cleansing only. A diluted cleanser should be wiped upon the surface and then wiped off after period instructed by cleanser's manufacturer. We recommend removing any cleanser residue by wiping down with water.

*SHARPS:* Avoid using sharp instruments that may puncture the outer skin. Do not drag product across sharp edges.

DO NOT AUTOCLAVE or ATTEMPT TO STERILIZE the products.

Light Clouds™ should not be exposed to high heat or flame.

*STORAGE:* Store pads flat whenever possible. Do NOT store in cold places, if unavoidable please allow product to return to room temperature for at least 30 minutes.

*WARRANTY:* All Light Cloud™ products are covered by a ninety day manufacturer's defective replacement policy. If pad is deemed defective due to manufacturing process it will be replaced at no cost.

*QUESTIONS:* Contact the distributor from whom the product was purchased or:

MANUFACTURER:  
DAVID SCOTT CO.

CE