



USER INSTRUCTIONS

Health o meter® Professional 894KLT

Health o meter® Professional 895KLT

Cautions and Warnings

To prevent injury and damage to your scale, please follow these instructions very carefully.

- Do not transport the scale with anything on the platform.
- For accurate weighing, the scale must be placed on a flat, stable surface.
- For accurate weighing, verify proper operation according to the procedure described in this manual before use.
- Do not use in the presence of flammable materials.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

Specifications


This Talking Digital Floor Scale announces the weight measurement in English or Spanish. It also features a weight hold that allows the weight measurement to stay displayed for 20 seconds after stepping off scale. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements and provides features that make the weighing process simple, fast and convenient. This scale is set up to determine the weight of a stationary patient. Weight is displayed in pounds or kilograms (kg).

Capacity and Resolution	894KLT Capacity: 400 lb / 180 kg; 895KLT Capacity: 550 lb / 250 kg; 894KLT/895KLT Resolution: 0.2 lb / 0.1 kg	
Environmental	Operating temperatures: 5°C to 40°C (41°F to 104°F); Storage temperatures: 0°C to 50°C (32°F to 125°F) Atmospheric Pressure: 500hPa to 1060hPa; Maximum Humidity: 85% RH in use or in storage	
Physical Dimensions	Platform Size: Width: 13 3/4" (349 mm) Depth: 10 3/4" (273 mm) Height: 1 1/2" (38 mm)	Product Footprint: Width: 14 1/4" (362 mm) Depth: 11 7/8" (302 mm) Height: 1 1/2" (38 mm) Weight: 4 lb (1.8 kg)

Inserting the Batteries / Replacing the Batteries

The scale is shipped without the batteries installed. The batteries are included in a small plastic bag. Follow these instructions to insert the batteries on a new scale or to replace depleted batteries on your current scale. The battery compartment is located on the underside of the scale.

1. To open the battery compartment cover, push the battery compartment tab to the right and lift upwards.

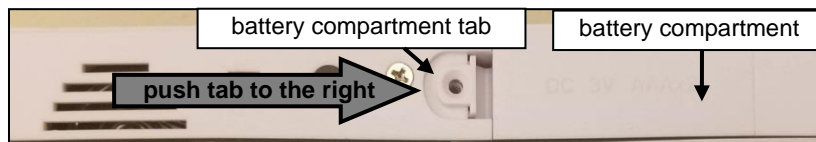

2. Install two new AAA batteries. Health o meter® Professional recommends using an alkaline battery rated for 130°F (54°C) operation. Do not use zinc carbon batteries. Follow the battery illustration to ensure the batteries are inserted in the proper direction.




3. To attach the battery compartment cover, slide the end of the cover into the battery compartment. Then push down on the tab and compartment to ensure the battery cover is firmly attached. You do NOT need the screw to close the battery compartment. You may use the screw to further secure the battery compartment if desired. Do not use the scale without the battery compartment cover attached.



To Set Weight Mode (KG or LB)

Slide the switch to the desired weight mode.

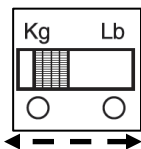


To Change the Language or Mute

Push the language mode button. The display will show which language is selected. Select the mute symbol for no sound.


EN = English
ES = Spanish
🔇 = No Sound

To Start


To turn on the scale, tap firmly with your foot on the center of the platform.

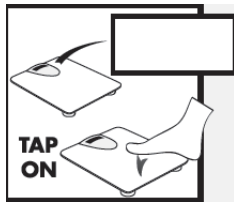

Four digits will appear on the display.

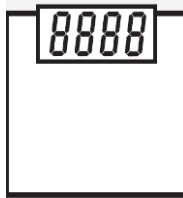


Wait for 0.0 to appear on the display.

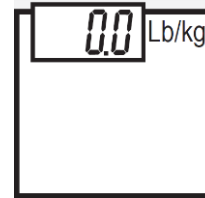

To Weigh


When ready, the scale will direct patient to step onto the scale. Stand still until weight appears.

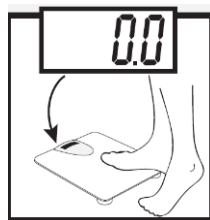

The measurement will flash on the display and scale will announce the weight.




Weight will remain displayed for 20 seconds after stepping off the scale.

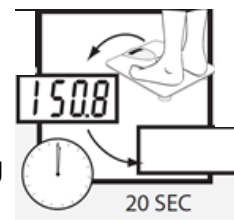


Troubleshooting

If the display shows "Lo bAtt" message, replace batteries per the included instructions.

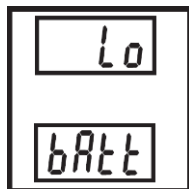

If the display shows "Err E" the load on the scale exceeds the capacity.
894KLT Capacity: 400 lb / 180 kg
895KLT Capacity: 550 lb / 250 kg

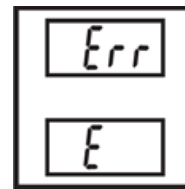

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
 - mild soap and water solution
 - 70% isopropyl alcohol
 - solution with 1-5% hydrogen peroxide concentration
2. After cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.
3. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
4. Do not submerge the scale in water or any other liquid.
5. Do not pour or spray fluids directly on the scale.

1-Year Limited Warranty

This Health o meter® Professional product is warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

For information or support, please contact Customer Service at 1 (800) 815-6615 or HomProCS@homscales.com

For complete warranty terms or the full user manual visit www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

© Pelstar, LLC 2020