



Thank you for purchasing Scan-Bands® Radiology Straps. Please read all instructions before using the product.

INDICATIONS FOR USE: Scan-Bands® Radiology Straps are intended to aid in the reduction of patient motion during a procedure on a medical imaging table or in other situations deemed appropriate by a healthcare professional.

GENERAL WARNINGS: Scan-Bands® Radiology Straps should never be used as a restraint. The strap should not be placed over the patient's head or neck. All patients should be monitored by staff while using Scan-Bands Radiology Straps during any medical imaging procedure. In the event of tears or damage, discontinue use.

NO LIABILITY FOR CONSEQUENTIAL DAMAGE: In no event will our liability for any damages to you, or any other person or entity, exceed the replacement cost of the product.

DEVICE DESCRIPTION: Scan-Bands® Radiology Straps are made of various widths and lengths, constructed from nylon anti-fungal materials and intended to help reduce patient motion during a medical imaging procedure. Scan-Bands® utilizes hook and loop or corded attachment methods.

CLEANING INSTRUCTIONS

GENERAL GUIDANCE: Healthcare professionals should test cleaning procedures on a small area before cleaning the entire product. Abrasive cleaning agents should not be used.

WASHING AND DISINFECTION: The cover is not sterile upon arrival. Clean thoroughly before each use with a bleach/water solution, as recommended by the Center for Disease Control or other controlling authority.

ROUTINE CARE: Wipe surface with neutral detergent and lukewarm water or a hospital grade disinfection. Rinse with water and air dry thoroughly.

DISINFECTION: Wipe surface with a quaternary solution at the recommended dilution ratios. Neutral solutions less than 10 pH are preferred; however, as previously noted, you should first test the fabric with the solution to guarantee fabric holds its high-quality standard.

STORAGE: Scan-Bands® Radiology Straps may be rolled or folded once thoroughly dry.

TROUBLESHOOTING: Do not use if torn or damaged. If you have questions about the product, please email us at info@Scan-Bands.com or call our Customer Service line at (877) 824-7510.



WARNING: Cancer and Reproductive Harm
www.P65Warnings.ca.gov

STEP BY STEP INSTRUCTIONS FOR USE

Our 2 set Scan-Bands Patient Band attach to the table with either a hook and loop attachment or a corded attachment.




1. Attach the 2 set Scan-Bands Patient Strap to table at desired location along the table using **hook and loop attachment** under the table pad.



3. Place the hook (hard) strap on the one side of the table with the hook facing out on the exposed side and the label visible.
4. Repeat with the strap that has loop (soft) so the plain side of the Scan-Bands Patient Strap hangs down from the edge of the table. The Scan-Bands Label should be facing out and be seen when strap is attached.



6. The hook (hard) strap attaches on top making sure at least 8 inches of hook overlaps the loop (soft) strap.
7. Straps should be snug and secure without binding the patient.
8. **Patients should be monitored by healthcare professional while using the Scan-Bands® Patient Strap.**

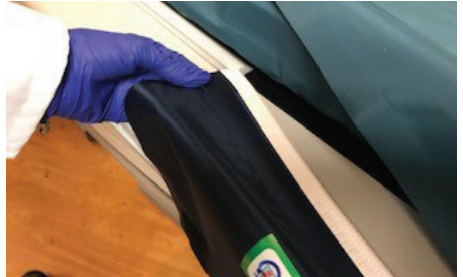


2. If the Scan-Bands Patient Strap has a **corded attachment** slide the corded strap into the channel under the table pad on the cradle



5. Position the patient on the table. Lay the smooth surface strap over the patient.

Universal Scan-bands is a single band that wraps under the table cradle to help support the patient on the Narrow imaging table.

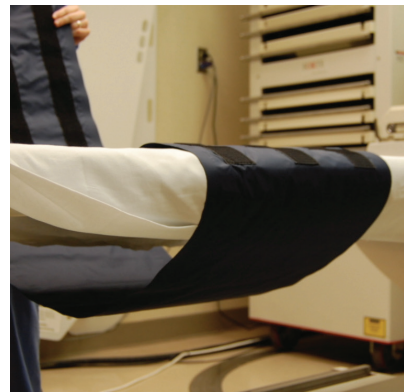

FOR SAFER, CLEANER, MORE COMFORTABLE MEDICAL IMAGING