

# CLEANING & CARE GUIDELINES

## Toboggan Arm Boards

TAB-CLR-S | TAB-CLR-M | TAB-CLR-L

INSTRUCTIONS OF USE



**PHILLIPS SAFETY**  
PRODUCTS INC.

COPYRIGHT ©2025. ALL RIGHTS RESERVED

Toboggan Arm Boards protect a patient's arms/legs during interventional procedures while securing them alongside the Operating table. The Arm Boards are designed to reduce the chance of having the arm pinched or crushed as well as eliminating arm drops during surgical procedures.

## USE

---

Slide the toboggan under the Operating table pad and weight of the patient will hold the arm board in place.

Contoured top offers stabilizing support for clinician's arms while shielding against sharp edges during procedures.

Protects the patient and the lines connected during procedures

## ATTENTION

---

Toboggan Arm Boards should be cleaned and disinfected after each use.

## CLEANING

---

Wash arm board with a mild detergent and water. Do not use abrasive cleaners or scouring pad because this will scratch the arm board.

## DISINFECTING

---

Use only disinfectants containing 70% Isopropyl alcohol (IPA) or 3-5% Hydrogen peroxide.

70% concentration of IPA is CDC approved for Covid-19 disinfection.

**Do not use** any disinfectant that contains dimethyl or diethyl benzyl ammonium chloride. Any form of ammonia (or -amines), in any amount, will cause cracks in the arm boards in a short amount of time.

## MAINTENANCE

---

Before each use, arm boards should be inspected for damage (i.e. cracks).

If cracks are noted, the arm board should not be used.