

Model 536510 / 537010



**Suction Canister Systems Tandem Connection  
1/4" (6.35 mm) x 18" (457 mm) Connecting Tube**

# Suction Canister Systems (Tandem Connection) 1/4 " (6.35 mm) x 18 " (457 mm) Connecting Tube

## INSTRUCTIONS FOR USE

1. Cap vacuum ports and pour spouts of first three canisters.
2. Attach patient tube to patient port of first canister.
3. Connect tandem port of first canister to patient port of next canister using a short length of tubing (18").
4. Repeat Step 3 until all canisters are attached.
5. Cap tandem port and attach vacuum source to vacuum port of last canister in series.

**Note:** When using a tandem setup, dispose of all used and unused canisters after the procedure.

## ⚠ CAUTION

Check to ensure that tubing is not bent or kinked.

P = PATIENT  
V = VACUUM  
T = TANDEM  
S = POUR SPOUT

