

BEMIS CANISTER STAND CLEANING INSTRUCTIONS



533910



534910

CLEANING

1. All surfaces may be washed with soap or mild detergent.
2. A soft brush or cloth may be used to remove stubborn dirt or blood.
3. Follow with a clean water rinse.

DISINFECTING

Use disinfection products that contain Quaternary Ammonium Chloride compounds.

- For example, PDI Super Sani-Cloth®

1. For the disinfection agent your facility chooses, follow the manufacturer's use and safety instructions to wipe all surfaces of canister stand.
2. Quaternary Ammonium Chloride can leave a buildup of residue; it is recommended to wipe off residue with the use of a clean, dry or water dampened cloth or paper towel.

CAUTIONS

- Do not use abrasive cloths or pads as this may alter the finish of the stand materials.
- Do not use a cart washing machine as this may damage wheel bearings or other materials.
- It is not recommended to use disinfectants that contain sodium hypochlorite (chlorine bleach).