

PORTA SHIELDS LEADED SHIELDS

CARE & CLEANING GUIDELINES

COPYRIGHT ©2025. ALL RIGHTS RESERVED



PHILLIPS SAFETY
PRODUCTS INC.

CLEANING

Clean your Porta Shield with a damp cloth and mild detergent. If you need to scrub the shield, use a soft toothbrush.

AVOID immersing the shield in water or putting it under a faucet.

AVOID harsh cleaning agents like bleach, IPA/ethanol.

DO NOT machine wash.

DISINFECTING

The following disinfecting agents are acceptable for application with a damp cloth, then allow to dry:

Hydrogen peroxide, 6% solution
Water based QAT's - or quaternary ammonium disinfectants - are acceptable provided they are substantially alcohol-free and made up in dilute water-based forms.

Follow manufacturer's instructions for dilution.

DO NOT autoclave.

STORAGE

Shields should be stored flat when not on base.

DO NOT FOLD!

Folding can lead to cracks in the protective material.