

HOW TO PROPERLY CLEAN RICE LAKE HEALTH SCALES

Knowing how to clean your Rice Lake health scale is essential to its longevity. Proper care and cleaning will help ensure accurate and effective operation.

Follow these tips to ensure proper cleaning of your Rice Lake health scale.



STEP 1

Turn the power off. Disconnect your scale from the AC power source.



STEP 2

Clean all external surfaces with a clean, damp cloth or tissue.

NOTE

Use of alcohol based products could potentially lead to the display and keypad drying out and ultimately failing.

Mild soap (any liquid and/or antimicrobial) and water solution may be used.

Do not immerse the scale into cleaning or other liquid solutions.

An EPA-registered product can be used on the scale, but use sparingly and avoid the display if possible.




STEP 3

Dry with a clean, soft cloth.



STEP 4

Reconnect your scale to the AC power source and resume usage.

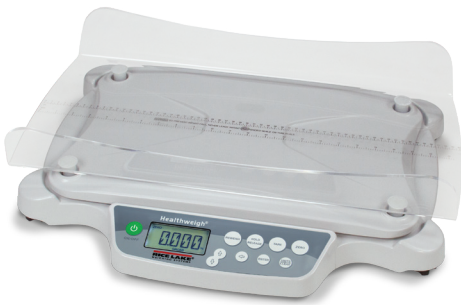

Rice Lake knows how important dependable weights and measurements are in healthcare, fitness and medical settings.

RICE LAKE
WEIGHING SYSTEMS