



PRESTAN®

# Prestan Professional Manikins

## Replacing the Chest Foam on the Infant Manikin

Tools required: Cordless Screwdriver (recommended) or Phillips Screwdriver

1. Pull chest skin down from the shoulders to expose foam.
2. Unscrew and remove (lift off) chest plate.




3. Remove (lift out) old chest foam.



4. Replace with new chest foam.
5. Reposition chest plate and screw for reattachment.
6. With one hand, compress down on chest plate and foam.



7. While still holding down, screw the chest plate back into place.

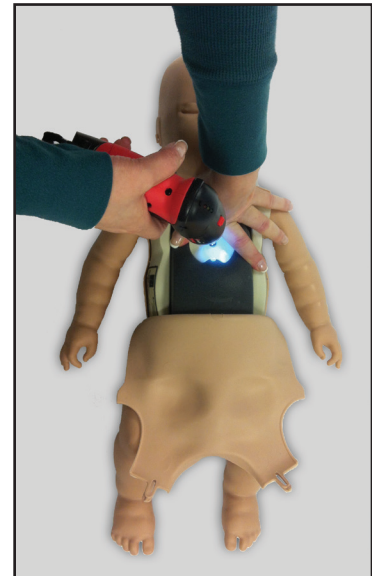

8. Re-cover chest with skin, securing tabs at shoulders.

