



Sentinel by Bemis™ Wall Bracket and Key Set

Soporte de pared y juego de llaves / Support mural et jeu de clés

Model 445 020 - Bracket & Key Set

Model 435 020 - Bracket Only



INCLUDED IN PACKAGE

- 10 Wall Brackets
- 44 Plastic Wall Plugs
- 44 1-1/4" Pan Head, 10-Gauge Screws
- 10 Keys (Model 445 020)

EN EL PAQUETE SE INCLUYEN

- 10 Soportes para montaje en la pared
- 44 Taquetes de plástico para pared
- 44 Tornillos de cabeza troncocónica de calibre 10 de 1-1/4 pulgadas
- 10 Llaves (Modelo 445 020)

COMPRIS DANS L'ENSEMBLE

- 10 supports muraux
- 44 capuchons muraux en plastique
- 44 vis à tête cylindrique de 3,2 cm (1-1/4 po), de calibre 10
- 10 Clés (Modèle 445 020)

Mounting Instructions

A safely accessible height for personnel is generally considered 52-56" from the floor to the top edge of the sharps container. With this as a guideline, mount the top of the bracket approximately 50" from the floor. Using wall bracket as template, mark location of four holes.

MASONRY WALLS

Drill 1/4" diameter holes 1-1/4" deep. Tap wall plugs into hole until flush to the wall. Using provided screws, mount the Wall Bracket.

WALLBOARD

Typical recommended fasteners are:

Molly fastener	CD6S	Metal fastener for 3/8" to 5/8" wallboard
Rawl anchor	2112	Metal fastener for 3/8" to 5/8" wallboard
Poly-Toggle anchor	2304	Plastic toggle fastener for 1/2" wallboard
Poly-Toggle anchor	2305	Plastic toggle fastener for 5/8" wallboard

Use Bracket Key to remove sharps container from bracket.