







High-Capacity Battery Installation

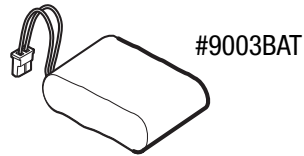

1

Press On/Off switch at back of unit to power off the unit.





Unplug power adapter socket from the back of the unit.

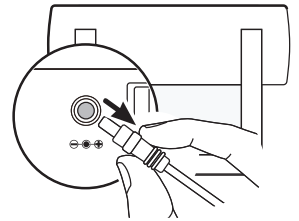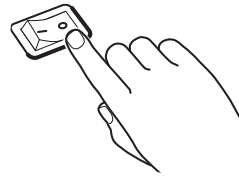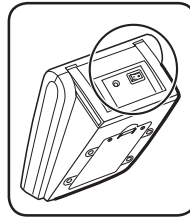

2

Remove both screws with a Philips screw driver.


Slide out back cover.

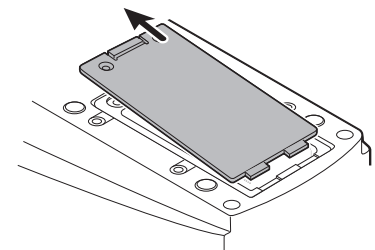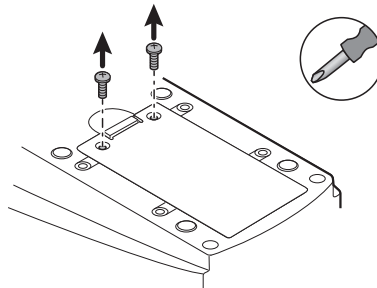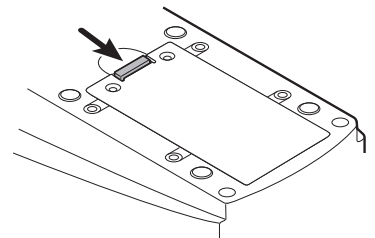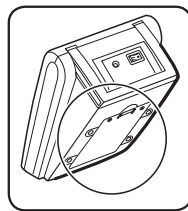

3



Remove the existing battery power pack.

Please do not dispose of battery in municipal waste.


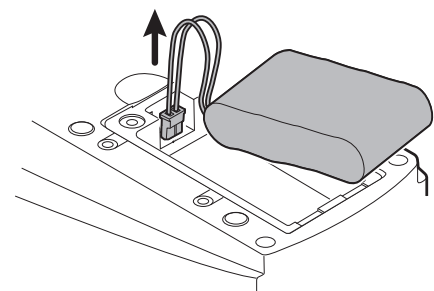


4


Insert new battery power pack. Make sure exposed battery wires are tucked underneath the battery power pack so that the battery cover can slide back on.

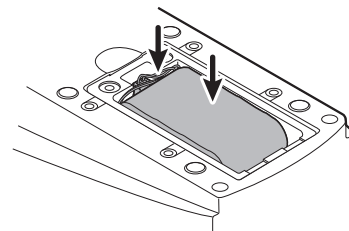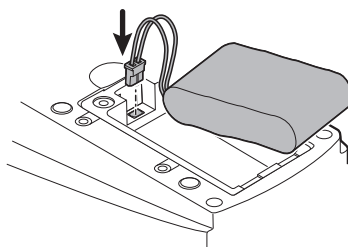**5**

Slide in the battery cover back into position and replace screws.

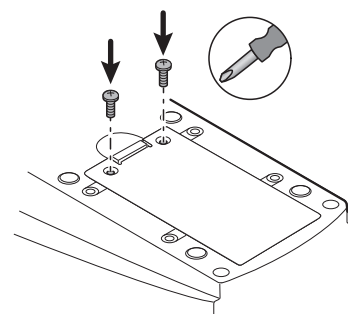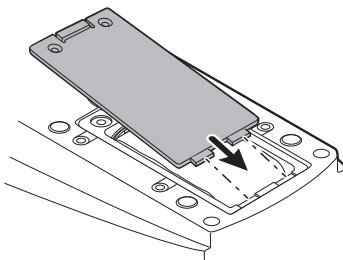**6**

Plug in power adapter socket at back of device and plug into wall outlet. Charge battery until re-charge indicator on main unit turns **green**.

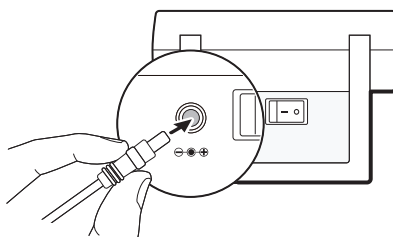

Download full manual at www.adctoday.com/esphyg3

