


PRESTAN®

Ultralite®12

with CPR Feedback
- User Guide -

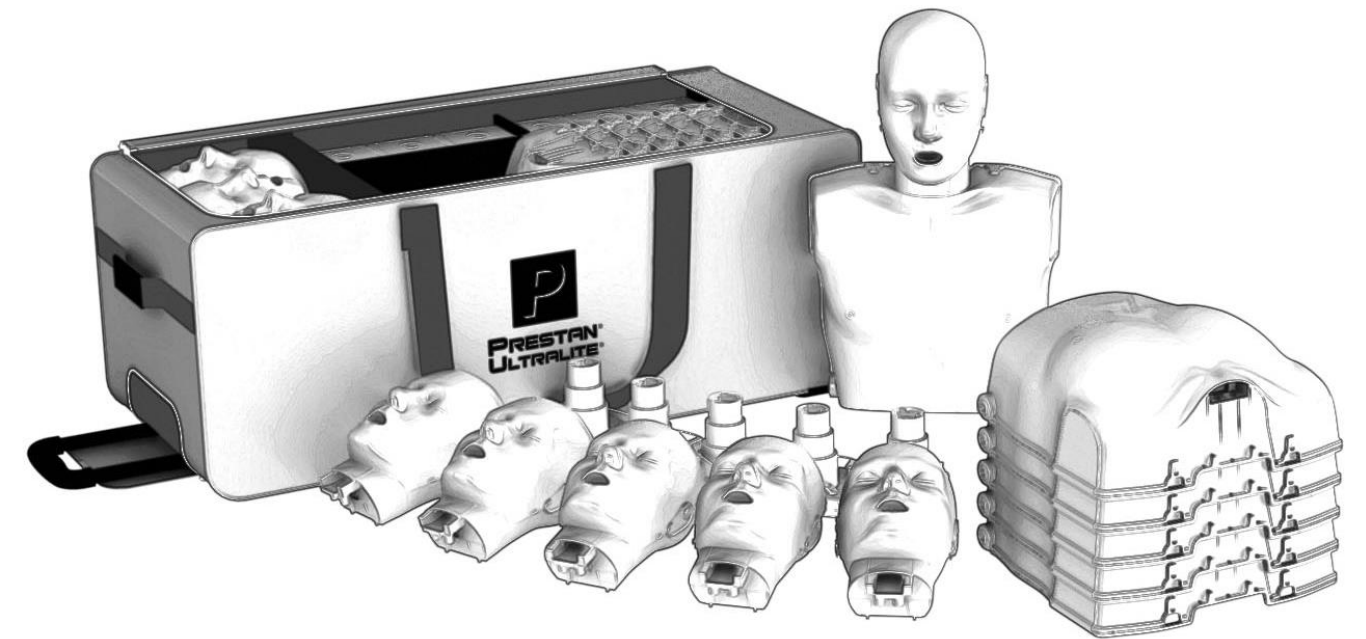

PRESTAN® Ultralite® Manikins with CPR Feedback

THANK YOU for buying the Prestan Ultralite Manikin 12-Pack with CPR Feedback, # PP-ULM-1200M-MS.

CONTENTS:

- Twelve (12) Manikin Head Faces
- Twelve (12) Manikin Head Backs
- Twelve (12) Manikin Torsos
- Twelve (12) CPR Feedback Pistons (2 AA Batteries required, not included)
- One (1) Rolling Carrying Case with Telescopic Handle
- Three (3) sets 50-pack Ultralite Face-Shield/Lung-Bags

CARE AND CLEANING:

- When not in use, the components should be stored in the carrying case and in a cool, dry place.
- Ultralite Manikins may be cleaned in a variety of ways. A non-abrasive cleaner is recommended
 - They can be wiped down with a Clorox® or Lysol® disinfecting wipe.
 - Cleaned with a mixture of soap and water.
 - Sprayed or wiped with common household cleaners such as Formula 409®, Windex® or Vim®.
- The face may also be sanitized with alcohol, Clorox® or Lysol® wipes. Between student uses (and prior to applying the face shield), the instructor (or student) may clean the faces with a sanitizing wipe as an extra measure of protection.
- Students may be provided with an individual face shield (sold separately) which may be placed over the Face-Shield/Lung-Bag. This allows multiple students to use one Face-Shield/Lung-Bag during a session.

Order through your Prestan Authorized Distributor:

Ultralite Manikins Face Shields, 50-count	#PP-UFS-50
Ultralite Manikins Face-Shield/Lung-Bags, 10-count	#PP-ULB-10
Ultralite Manikins Face-Shield/Lung-Bags, 50-count	#PP-ULB-50

TROUBLESHOOTING:


- If you cannot get the chest to rise when you blow into the face shield lung bag:
 - Check to make sure that the head is tilted back to create an open airway.
 - Check the lung bag itself. Ensure that it is properly positioned over the compression plate, not twisted and is lying flat.
 - Check the face shield. Ensure that it is attached to the 'buttons' on the side of the manikin head and that it is flat against the face of the manikin.
 - Check the bag to ensure that it has not been torn.
- If the chest does not compress and/or it seems the piston is binding:
 - Ensure proper hand placement - check horizontal hand placement and adjust to center in between the nipples.



PRESTAN®

Ultralite® Manikins with CPR Feedback








ASSEMBLE

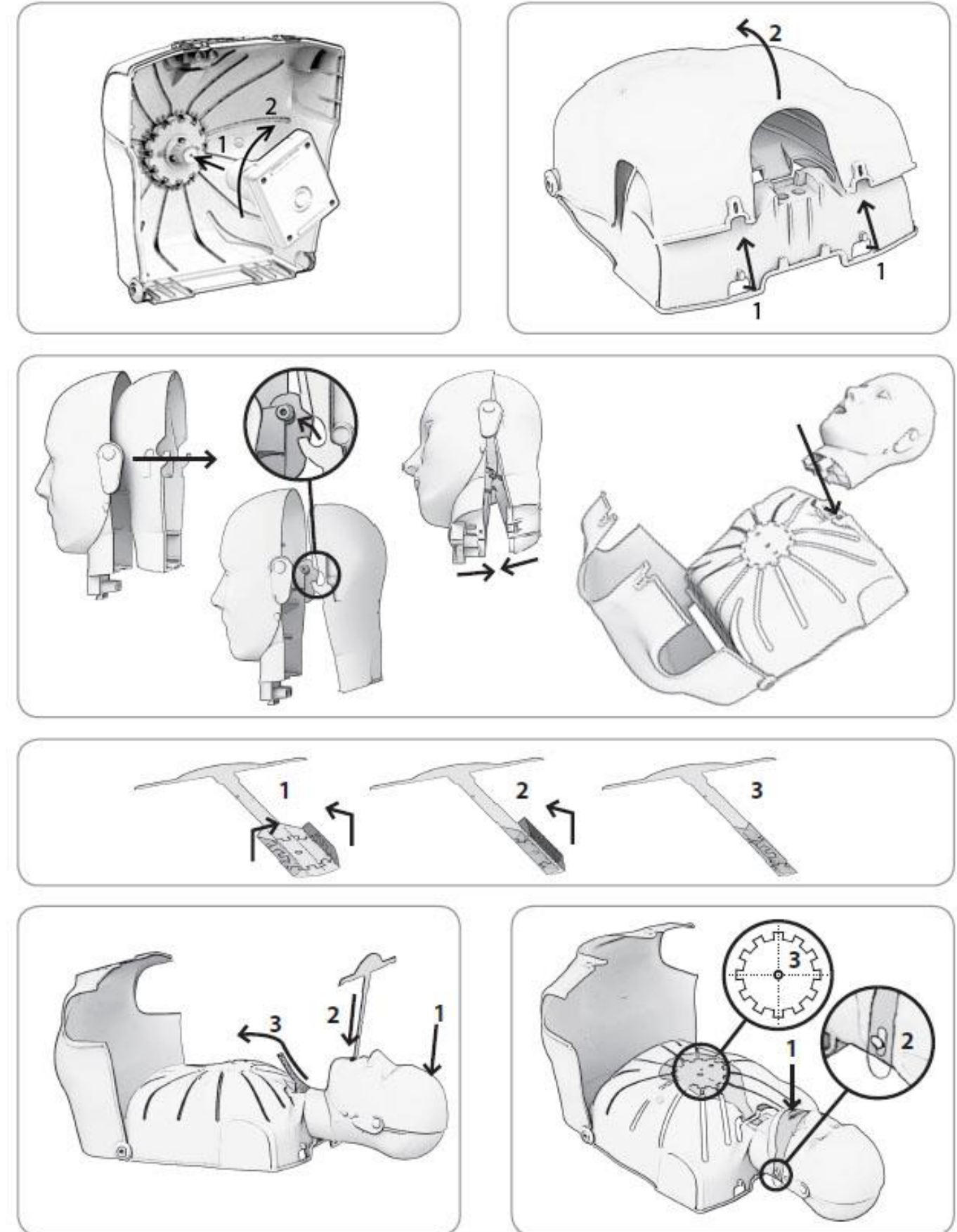

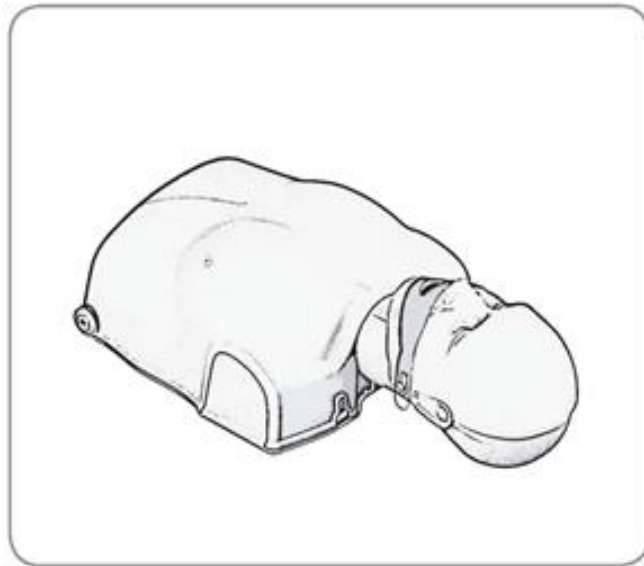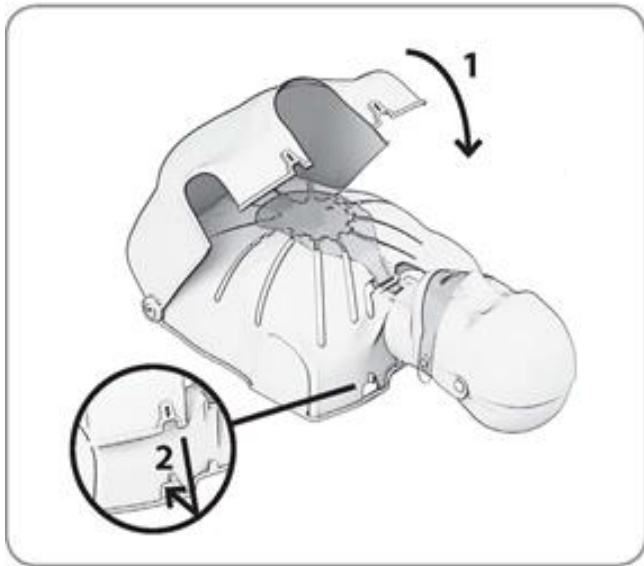

DISASSEMBLE

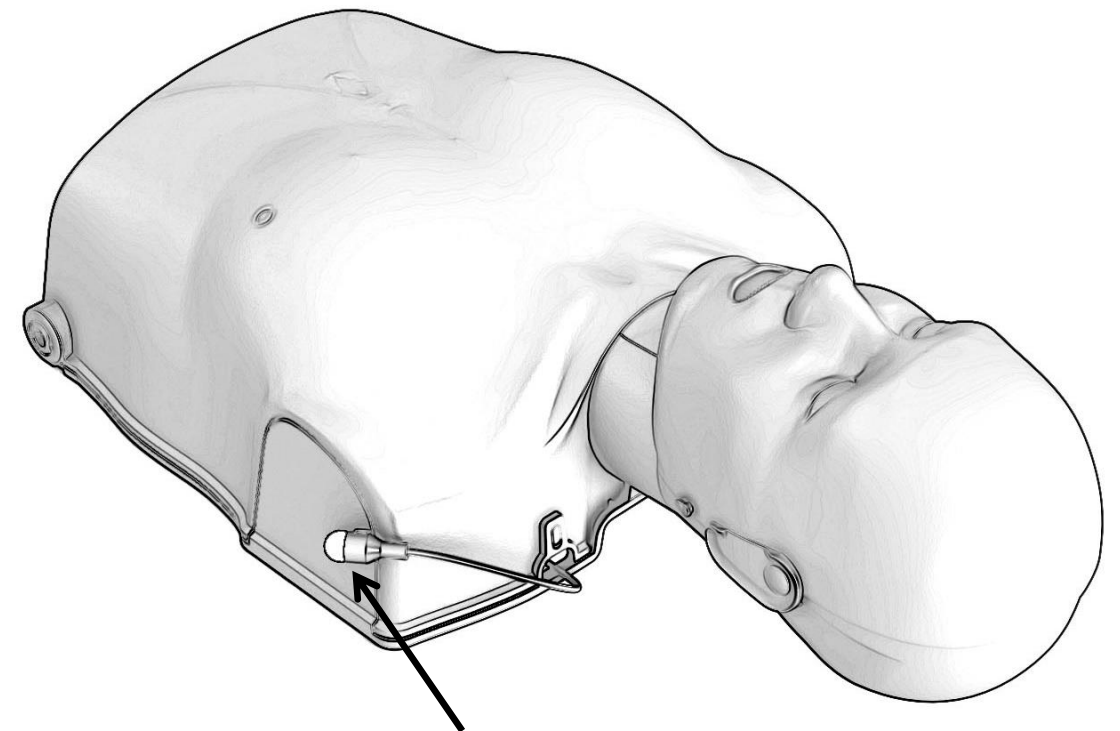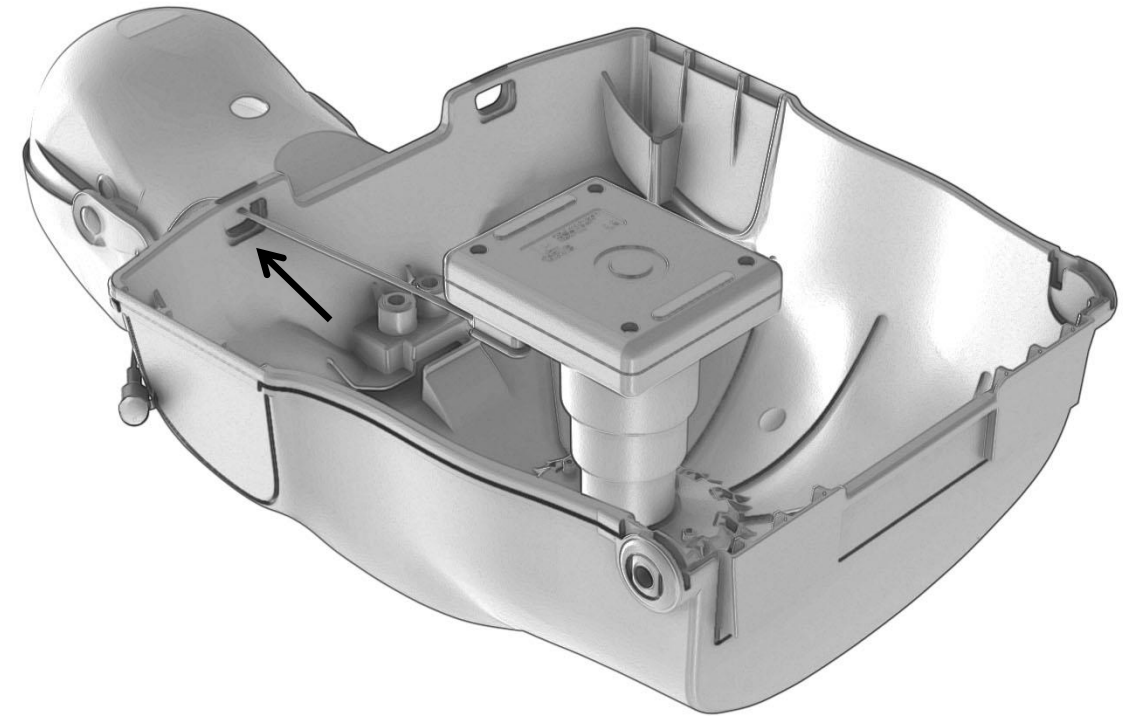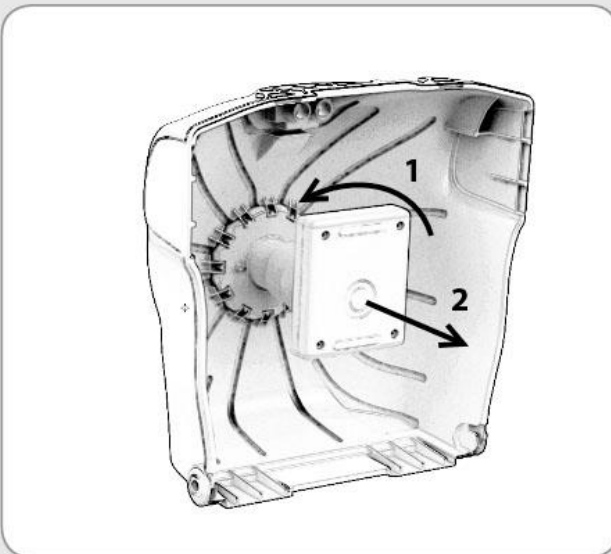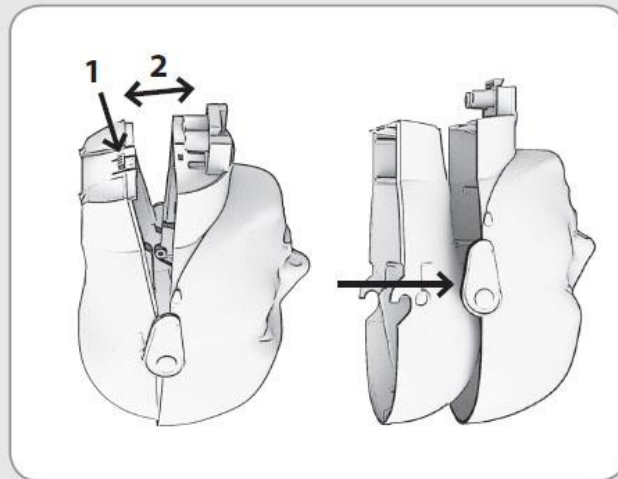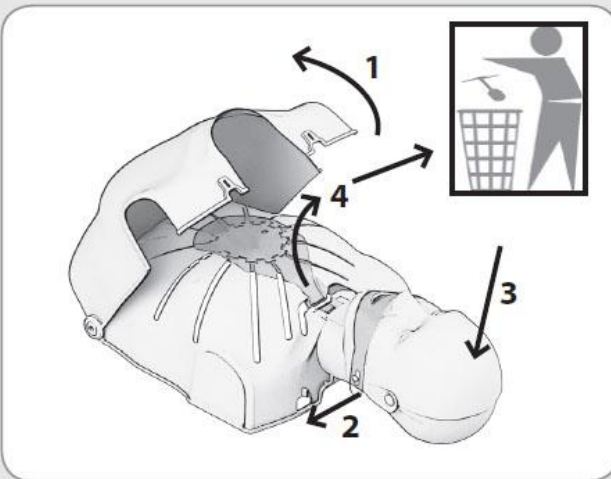

- Solid RED Light indicates under 100 CPM
- Solid GREEN Light indicates 100-120 CPM
(As recommended in 2015 AHA and ARC Guidelines)
- Flashing RED Light indicates more than 120 CPM

(CPM: Compressions per minute)



PRESTAN®

PRESTAN Ultralite® Manikin 12-Pack

Prestan's most portable manikin, the **Ultralite® Manikin** is amazingly easy to set up for convenient and lightweight training-on-the-go! This durable manikin offers an affordable method with CPR Feedback training, with all the quality and realism you expect from Prestan.



Prestan's most portable manikin, the **Ultralite® Manikin** is now available in an **ULTRA-CONVENIENT 12-PACK!**

- > Wheels and telescoping handle make for effortless maneuverability
- > Only 39 pounds for easy lifting in and out of vehicles
- > Stackable packing system for portability

**A CLASSROOM
OF CPR TRAINING
MANIKINS IN ONE
PORTABLE PACK!**



Ultralite® Manikin Features:

- Uniquely fast and easy set-up
- Clamshell opening provides easy access
- Manikin is easy to clean
- Vinyl and latex free
- Compatible with AED Trainer Pads
- Anatomically correct design includes xyphoid process
- Available in medium and dark skin tones
- Realistic head tilt
- Visible chest rise with lung bag use
- Realistic chest compressions to a 2" depth
- Three (3) year warranty

Ultralite® Manikin 12-pack with Feedback	Ultralite® Manikin 12-pack without Feedback
PP-ULM-1200M-MS	PP-ULM-1200-MS
PP-ULM-1200M-DS	PP-ULM-1200-DS

Reorder consumables from your Prestan Authorized Distributor:

- #PP-ULB-50 (50-pack Ultralite Manikin face-shield/lung-bags)
- #PP-UFS-50 (50-pack Ultralite Manikin face-shields)

