



# HOW TO INSTALL

## RADIATION SHIELDING PLEATED TABLE SHIELD

COPYRIGHT ©2025. ALL RIGHTS RESERVED

# ABOUT PHILLIPS SAFETY

Phillips Safety is the industry leader in occupational safety eyewear. In business for over 100 years, you are being served with the dedication and craftsmanship of a family owned company.

Phillips Safety has been producing occupational safety glasses since 1905 and continues to develop and produce the highest quality glasses and lenses today. A leading manufacturer of specialty safety eyewear in the United States, Phillips Safety adheres to strict ANSI Z87-2+ and EN 207 CE standard certifications, holding itself to the highest

standard and prides itself on producing glasses with the finest quality materials available.

All safety glasses produced and sold by Phillips Safety meet strict industry standards for military, medical, research laboratories, and a variety of other industries.

With Phillips Safety's dedicated team, you can rest assured knowing that all your prescription safety needs are met.

**HOW TO INSTALL**  
RADIATION SHIELDING PLEATED TABLE SHIELD

**STEPS**

**ATTENTION**

BEFORE INSTALL THE RADIATION SHIELDING PLEATED TABLE SHIELD, YOUR IMAGING TABLE MUST BE CLEAN AND DRY.

WE RECOMMEND CLEANING YOUR IMAGING TABLE WITH CAVI WIPES.

**1**

**WITH THE NON-SLIP PAD FACING THE IMAGING TABLE. PEEL THE PROTECTIVE FILM**



**HOW TO INSTALL**  
RADIATION SHIELDING PLEATED TABLE SHIELD

**STEPS**

**2**

**POSITION THE LEAD CURTAIN  
ALIGNED WITH THE CENTER OF  
YOUR TABLE.**



**REMEMBER**  
The table surface  
**MUST BE** completely  
clean and dry.

**3**

**ATTACH THE ENTIRE SURFACE  
WITH THE TABLE.**



4

**PRESS FIRMLY TO ENSURE  
ADHERANCE.**

