

# Connector Change

## Coming September 1, 2022

Due to new regulatory guidance from the FDA, we are changing to bayonet connectors on all Diagnostix brand clock aneroids and Adcuff brand inflation systems.


### Beginning September 1, 2022:

- **ADC brand clock aneroids** (models 750W, 750D, and 752M) will be fitted with a female bayonet connector, item #8974F, on the coiled tubing
- **ADC brand inflation systems** (models that start with 865) will be fitted with a male bayonet connector, item #8974M.

### Are the New Connectors Compatible?


if you've ordered a new clock aneroid and new inflation systems, both will arrive with the updated connector style and can be used together immediately. Nothing else is required.

#### New Clock Aneroids and New Inflation Systems

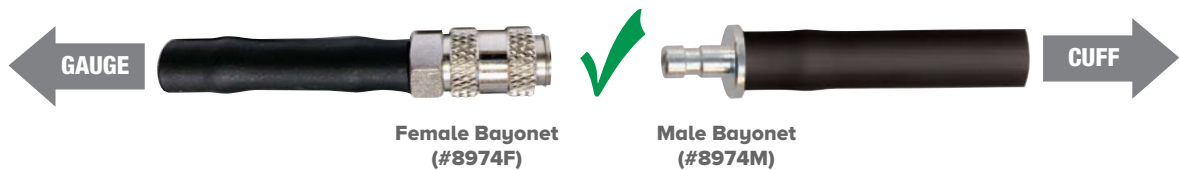

if you've ordered new inflation systems to use with a legacy clock aneroid instrument, you'll need to convert one of the connectors.

#### Old Clock Aneroids and New Inflation Systems

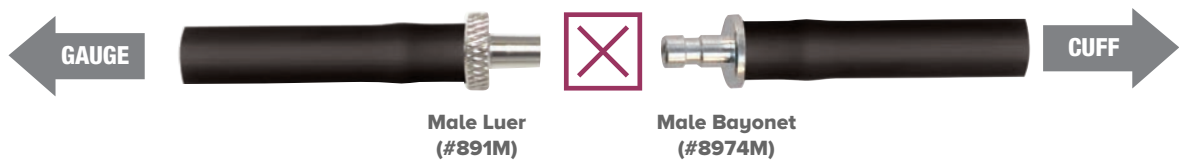

*See the back page for instructions on how to convert a connector*

# How Do I Convert a Connector?

It's easy, and you have two options that will allow you to use a new inflation system with a legacy clock aneroid instrument:

## Option 1:


### Replace the connector on the new inflation system

During the transition period, you can replace the connector on your new inflation system with one that's compatible with a legacy gauge. For several months, ADC will be including a replacement legacy connector and instructions (#891FKIT), so you can quickly retrofit your cuff in the field. Check your inflation system packaging for this part.




**#891F**

Included with new inflation systems through January 1

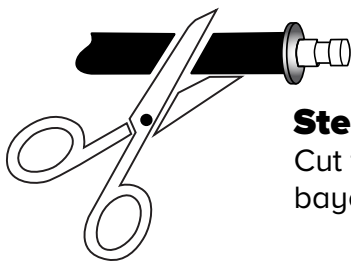

#### Step 1

Cut tubing to remove bayonet connector.

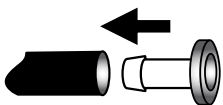


#### Step 2

Insert the new female luer. Slip into tubing until fully seated.

## Option 2:


### Add on adapter to your legacy clock aneroid

If you prefer to use the new inflation system as is, we offer a simple adapter that you can connect to the coiled tubing on the clock aneroid. Contact your ADC sales rep., dealer, or our Customer Service team and ask for part #8974FKIT.

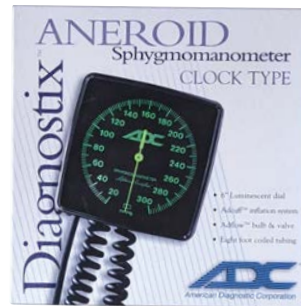

**#874FKIT**

Available on request

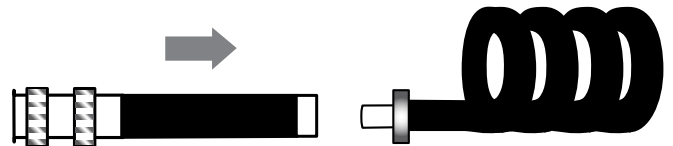

**8974FKIT**

**Slide adapter onto coiled tubing until tight.**