

Cleaning Guide

5WRS-TLMD | Telmedia 5 Wheels Roll Stand

- Clean in room temperature, using warm water & neutral detergent on disposable cloth.
- Cleaning can be performed with most common detergents & disinfectants on the market.
- Avoid usage of concentrated bleach, synthetic detergents, solvents, wax polishes, or aerosol sprays.
- Disinfect – with Chlorine (NaDCC) Disinfectant.
- Do not use Phenol-based disinfectants, e.g. cresol.
- Do not saturate to avoid accelerated corrosion, and generally extend the product lifespan.
- Use damp cloth only. Wring out rag before wiping the surface of the product.
- Continued wet or humidity could result in staining or corrosion.
- Dry with a lint-free cloth after wiping with clean water.
- No rubbing with abrasive cloths or tools.
- Do not wash in automatic washing machine.

