

Quick Start Guide

Nasco Healthcare Auscultation Trainer 2.0

LF01290 & LF01290EX

What's Included:

- Auscultation Trainer Torso 2.0
- Smart Scope 2.0
- iPad (LF01290 Only)
- Soft Carry Case

Also Included:

- Dual-Head Teaching Stethoscope (not pictured)
- USB-C Charging Cable (not pictured)
- Digital Instruction Manual (QR below)



Setting Up:

1. Ensure all devices are fully charged before use, using the provided USB-C Charging Cable.
2. Download the Be READY™ Auscultation App onto the iPad using the QR-Code provided.
3. Press Power Buttons on Torso and Scope to power on both devices.
4. Open the Be READY™ Auscultation app. On the Dashboard, both devices should appear on the Device menu (if not, press the refresh icon). Press the Bluetooth icons to the right of the device names to connect each device.
5. To begin a simulation, press "Launch" under Simulations → Auscultation.
6. Select a scenario from the list of preset scenarios OR create your own by pressing "Add Scenario".
7. Customize scenario settings as desired & press "Start Scenario".



Download the
Be READY™ App

Please refer to this QR-Code
for the Instruction Manual:

