


PIG Surgical Absorbent Mat

Improve O.R. safety and save money by using PIG Surgical Absorbent Mats instead of hospital blankets.

Features / Benefits:

- **Highly absorbent** — made with 50% recycled content, our eight-layer pads absorb and retain up to 3 liters to help keep your O.R. floor clean and safe
- **Disposable** — save money with fast cleanup and O.R. turnover while avoiding the need to launder heavy, liquid-soaked blankets
- **Lies flat on the floor** — no more tripping over bunched-up blankets; also allows for easy movement of mayo stands and other carts
- **Floor-gripping fibers** — put mats down any time during a procedure and reposition as needed; special microfibers grip wet floors so they won't slide around
- **Superior durability** — mats won't rip, tear or fray under heavy foot traffic
- **Perforated** — easily create smaller pads by tearing at perforations
- **Latex-free**
- **Mobilize your mat** — Add a wheeled, stainless steel mat roll dispenser for easy access to your absorbents

NOTE: PIG Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

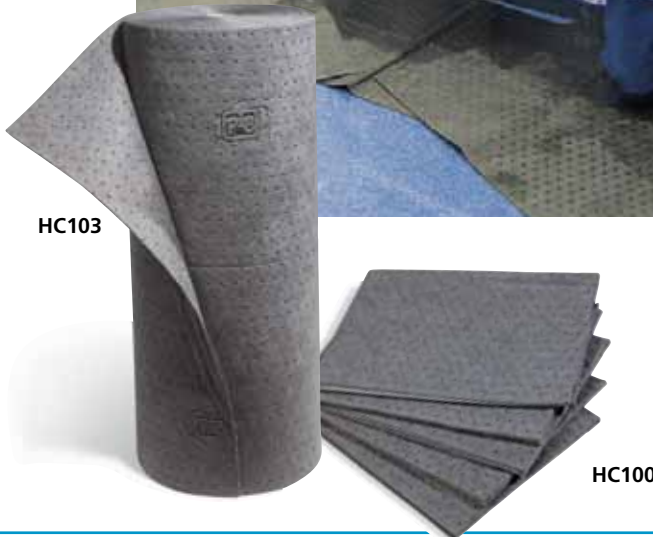

Available in either rolls or mats

PIG® Surgical Absorbent Mat

Item #	Description
HC100	20 Mats • 30" x 40"
HC103	1 Roll • 30" x 150'
HC300	Roll Dispenser

Flip over for instructions >



New Pig

PIG Surgical Absorbent Mat

PIG Surgical Absorbent Mat can be applied to your floor before, during or after procedures. Reposition mats as needed throughout the surgery.

1. Tear mat along perforations to fit your space.
2. Place mat on dry floor to catch liquids as they accumulate; place mat on wet floor to soak up existing liquids.
3. When mat is saturated, pick it up and throw it away.
4. Replace saturated mats with new mats until procedure is complete.

NOTE: PIG Surgical Absorbent Mats are designed for one-time use. Please dispose of properly.

WARNING: Used absorbents can take on the hazardous characteristics of liquids absorbed. Follow local, state and federal regulations when storing or disposing of used absorbents.



PIG Surgical Absorbent Mat won't bunch up like hospital blankets.



New Pig

[Flip over for product details >](#)