

# CLEANING RICE LAKE MECHANICAL SCALES

Knowing how to clean your Rice Lake health scale is essential to its longevity. Proper care and cleaning will help ensure accurate and effective operation.

Instructions are for the following products:

**Rice Lake Mechanical Physician Scale Models:**

RL-MPS    RL-MPS-20    RL-MPS-30  
RL-MPS-40    RL-MPS-50

**Rice Lake Mechanical Chair Scale Models:**

RL-MCS

**Rice Lake Mechanical Baby Scale Models:**

RL-MIS    RL-MIS-20



---

**Follow these tips to ensure proper cleaning of your Rice Lake health scale.**

---

## STEP 1

Wipe all external surfaces with a damp cloth using a mild soap (liquid or antimicrobial) and water solution.

## STEP 2

Dry scale completely with a clean cloth.

## STEP 3

Do not immerse the scale in cleaning solution.

## STEP 4

Do not use an abrasive cleaner as this can damage the scale surface.

## STEP 5

An EPA-registered cleaning product can be used on the scale, but use sparingly.

---

We know how important dependable weights and measurements are in healthcare, fitness and medical settings. Learn more about our healthcare products at [www.ricelake.com/health](http://www.ricelake.com/health).

**RICE LAKE**  
WEIGHING SYSTEMS