



Sterilization Instructions:

Please note, sterilization is ONLY for items that are fully Stainless Steel!!

Sterilization Methods

1. Steam Autoclave (Preferred)

- Cycle Type: Pre-vacuum or gravity displacement.
- Temperature: 132–135 °C (270–275 °F).
- Exposure Time: 5 minutes at temperature (pre-vacuum) or 15–30 minutes (gravity).
- Drying Time: At least 20–30 minutes to prevent moisture staining.
- Notes: Make sure instruments are disassembled and hinged devices are open. Wrap or package according to hospital protocols.

Important Handling Notes

- Avoid prolonged exposure to saline or bleach; both damage stainless steel.
- Inspect for corrosion, cracks, or pitting after each cycle.
- Store in a clean, dry, dust-free environment after sterilization.