



POLTEX

CLEANING AND DISINFECTING INSTRUCTIONS FOR ACRYLIC PRODUCTS

Cleaning:

Follow these instructions to keep your acrylic products looking clean and clear just like the day you bought them!

Wash acrylic sheets with a mild soap or detergent and plenty of lukewarm water. Use a soft cloth, applying only light pressure. Rinse with clear water and dry by blotting with a damp cloth or chamois. Or, "Plastic Clean and Shine" or "Brillianize" can be used to clean and polish acrylic.

CAUTION: Do not use ethanol or any other alcohols (in high concentration), window cleaning sprays, kitchen cleansers, gasoline, acetone, or paint thinner to clean your acrylic products. These agents will eat away at the acrylic and degrade your product decreasing the life span.

Disinfecting:

Use a 10% bleach solution to disinfect acrylic products. Rinse with water and towel or air dry.

Also, Clorox or other disinfecting wipes can be used on acrylic. Just take care to wipe down, and dry the products without letting the cleaner sit on the surface for too long.

Ultra violet light can damage acrylic over a long period of time. However, short (15 minute) treatments with UV light will help to disinfect and decontaminate acrylic without severely compromising its integrity.

Do not use phenolic compounds like LpHse or similar germicides. These will severely damage acrylic products.