


## Cardio P1

### Front View



- ① **LED Indicator:** DC power connection status
    - Blue light: DC power is well connected to the PC
  - ② **Start / Event**  
Start record, event marker key
    - **Short Press:** Perform as a marker when abnormalities such as arrhythmias were found
    - **Long Press** (more than 3 sec): Perform as the AUTO key for saving, transmission and printing
- LED Indicator:** Lead status
- Red light: Lead fault
  - Blue light: All leads are well connected

### Connection Ports

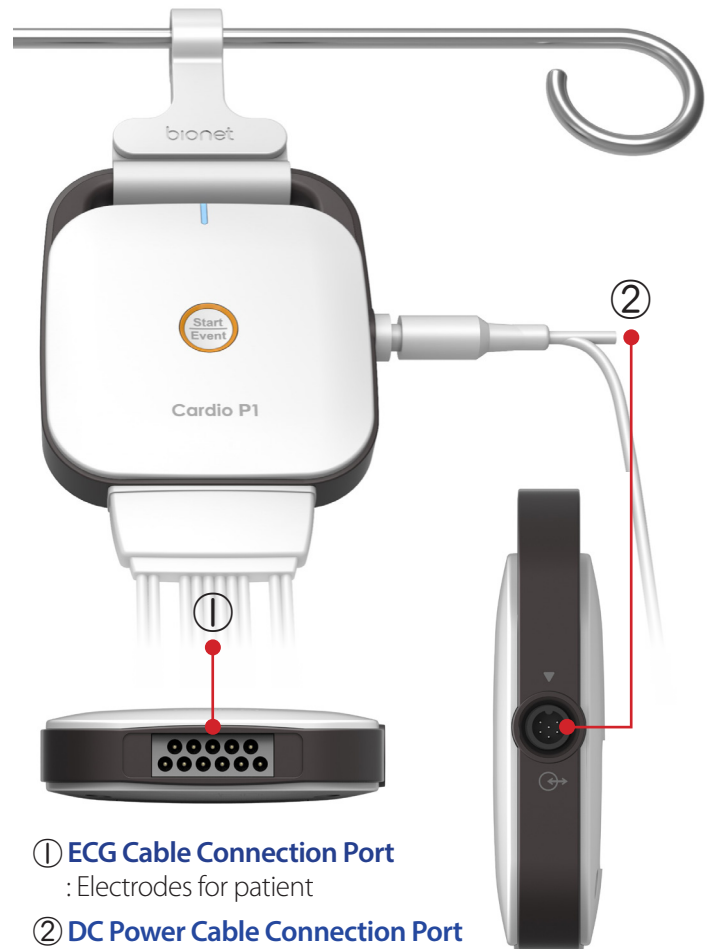

- ① **ECG Cable Connection Port**  
: Electrodes for patient
- ② **DC Power Cable Connection Port**  
: Cable connected to PC

### Back View



- ① **ID Label:** Part for ID Label



# Quick Guide Cardio P1

## 1. Connect

Connect DC power cable to Cardio P1 and PC.


## 2. Launch CardioSync

Double click the icon to launch the CardioSync in the PC.



## 3. Login

Input proper ID and password and press the LOGIN button.



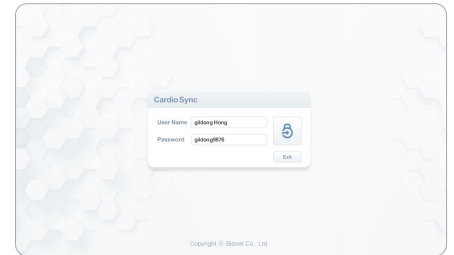


## 4. Worklist / Patient Registration

Opt 1. Press Worklist → Find the patient in the list.



Opt 2. Press ECG → Press [Not occupied] to register the patient info.

Opt 3. Connect a barcode reader and scan the patient barcode.

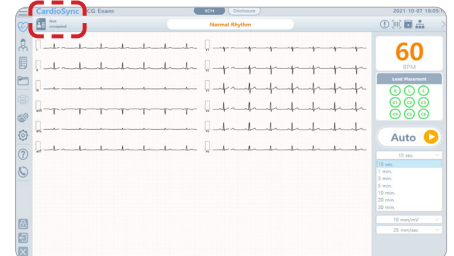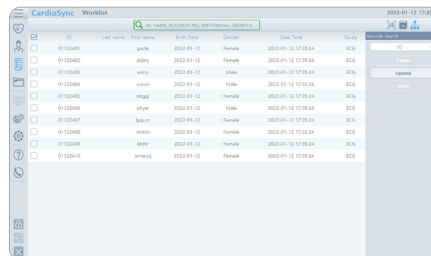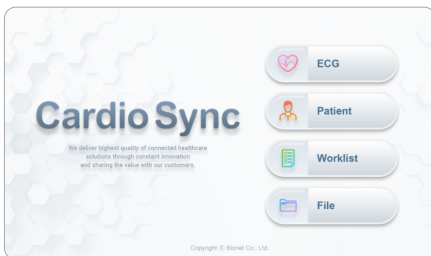

## 5. Locate Electrodes

Touch [Lead Fault Information Indicator] located on the right side of ECG main screen, the image will show to indicate the location for attaching the leads.

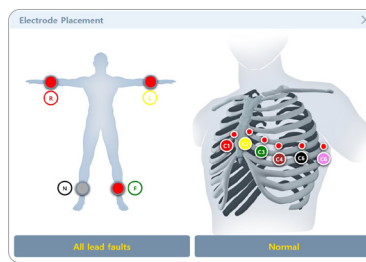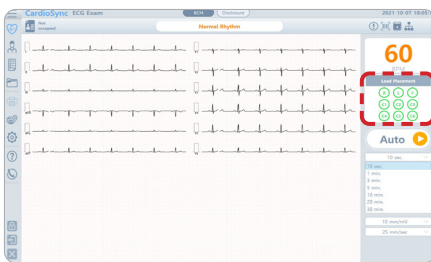

## 6. AUTO

Press the AUTO button (PC) or Start/Event button (P1) to acquire data for 10 seconds.



You can set the AUTO key function at the Setup → ECG → Auto key menu.

## 7. Preview

You can check the automatic analysis result on the preview screen.

## 8. OK

Click OK to finish the AUTO key procedures.

## 9. Check the exam data

You can check the exam data in the Patient menu or the File menu.

