

CardioTouch3000 Operation Manual



Ver. 7.00

2019.05.23



CE
0123

CardioTouch3000 Operation Manual

CAUTIONS

Federal law restricts this device to sale by or on the order of a physician

※ In the event of a malfunction or failure, contact Service Dept. Of Bionet Co., Ltd. along with the model name, serial number, date of purchase and explanation of failure.

Fee-Based Service

If you wish to make a claim when the product is damaged through misuse, we charge repair fees. Please read the manual before you make a claim.

<ul style="list-style-type: none">- For simple inspection without giving instruction or disassembling the product- For reinstalling due to poor installation by the retailer	Fee charged: From the second time Free of charge: One time
<ul style="list-style-type: none">- For poor installation due to moving- For reinstallation after initial installation due to customer's requirements.- For reinstallation due to customer's unskilled installation- For offering service when a foreign substance is introduced or mis-cleaned by the customer	Fee charged

1. The warranty does not cover cleaning, adjusting and instruction for use.

(Separate standard will apply when repair is impossible)

2. Fault of customer

When the product is damaged or broken due to customer's improper use or misuse

- When the power cord is replaced improperly by the user
- When the user drops the product while moving, after the initial installation
- When the user uses unauthorized accessories
- When the product is repaired by unauthorized person

3. Others

- When the product is damaged by a natural disaster (fire, flood or earthquake)
- When lifespan of accessories ends

Definition of WARNING, CAUTIONS and NOTE

- In order to stress the contents of this manual, we define the terms as below. Please follow the warning and cautions instruction.
- The manufacturer or service agents are not responsible for damage resulting from inappropriate use or carelessness.

WARNING

There may be serious injuries, fatal accidents or financial damages if you violate this instruction

CAUTIONS






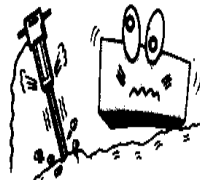
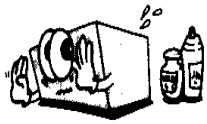
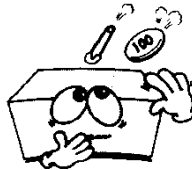
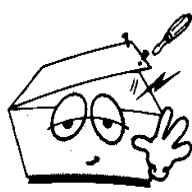
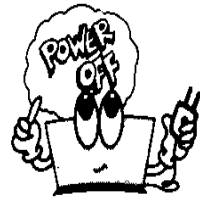
There may be slight injuries or reduced damages if you violate this instruction

NOTE

There may not be any dangerous events, but it is important to note this instruction for installation, use, maintenance or repair

Environment Instructions

- Please do NOT use or place the product in such environments explained below.

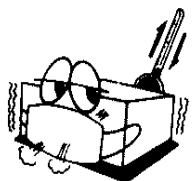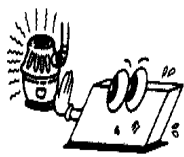	<p>Steamy environment. Do not use the product with wet hands.</p>		<p>Direct sunlight.</p>
	<p>Place where the temperature and humidity condition are extreme (Recommended temperature and humidity range for Storage/Transport are 5~40°C and 10~90%, respectively.)</p>		<p>Near electronic heaters</p>
	<p>High humidity or ill-ventilated place</p>		<p>Place where excessive shock or vibration may occur</p>
	<p>Place exposed to flammable chemicals or explosive gas.</p>		<p>Never introduce dust or metal into the product.</p>
	<p>Do not disassemble the product. The product warranty does not cover problems resulting from disassembling the product.</p>		<p>Do not plug the power supply cord before installing the product completely. It may cause damage to the product.</p>

Safety Instructions for Electricity

Please note the following precautions before using the product.

- Is the power supply cord proper? (100-240Vac)
- Is every cord connected properly to the product?
- Is the grounding connected correctly? (Otherwise, noise can occur.)
- There is a risk of electric shock if the Rest stand of the equipment is damaged or can not be fixed to the product. Do not use the product and immediately ask the manufacturer and the seller for repair

NOTE

Magnetic and electrical fields are capable of interfering with the proper performance of the device. For this reason, make sure that all external devices operated in the vicinity of the device comply with the relevant EMC requirements. X-ray equipment or MRI devices are possible sources of interference as they may emit higher levels of electromagnetic radiation.

NOTE

This device is classified as follows, in accordance with IEC60601-1.
Its classification against electric shocks is Class I, Type CF defibrillation - proof applied part.
Degree of protection against harmful ingress of water: Ordinary
It is not proper to use this product near a flammable anesthetic or solvent.
Continuous operation.
IEC/EN60601-1-2 (Electromagnetic Compatibility Requirements) standard :
Class A

NOTE

Diagnosis provided by CardioTouch3000 must be confirmed by a qualified medical professional.

CardioTouch3000 Operation Manual

NOTE

Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards (e.g. IEC 60950 for data processing equipment). Furthermore all configurations shall comply with the requirements for medical electrical systems (see IEC 60601-1-1 or clause 16 of the 3 Ed. of IEC 60601-1, respectively).

WARNING

When using AC power with multi-tap, be sure to use grounding type.

WARNING

Do not contact the patient's connector pin with the patient at the same time.

WARNING

Do not contacts with the patient while operate the machine It may cause serious danger to the users. Use only the provided cable.
A warning that other cables and accessories may negatively affect EMC performance.

NOTE

This product is a Class A product with EMC4 version and is suitable for professional healthcare facility environment.

NOTE

Exposing this product to EM interference may cause the equipment to malfunction.

WARNING

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

CardioTouch3000 Operation Manual

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.








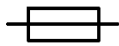



WARNING

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the [ME EQUIPMENT or ME SYSTEM], including cables specified by the manufacturer.









NOTE

The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

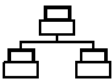Safety Symbols

Symbols	Contents
	ATTENTION : Consult accompanying documents
	Consult instructions for use: This symbol advises the reader to consult the operating instructions for information needed for the proper use of the device.
	Safety Sign : To signify that the instruction manual must be read. Reading the instruction manual before starting work or before operating equipment.
	General prohibition sign
	Defibrillation –proof type CF APPLIED PART
	Type B APPLIED PART
	Alternating Current Power
	Fuse
	Conductor provides a connection between equipment and the potential equalization bus bar of the electrical installation
	ECG Patient Cable Connector
	USB Connector

CardioTouch3000 Operation Manual

Symbols	Contents
	Local Area Network(LAN) Connector
	Power Off
	Power On
	Battery Operation Indicator
	AC Power Connection Indicator
	Manufacturer name and address
	Authorized European representative
	Waste of electrical and electronic equipment must not be disposed as unsorted municipal waste and must be collected separately. Please contact an authorized representative of the manufacturer for information concerning the decommissioning of your equipment.

Chapter 1. General Information

1) Product Overview

2) Indications for Use

3) Recording ECGs during Defibrillation

4) Product Features

5) Product Configuration

Basic Components and Accessories

Optional Components

Body Configuration

Front Panel

Graphic Display LCD

Control Panel

Power

6) System Installation

Precautions For Installation

Power Connection

Patient Cable Connection

Paper Installation

7) System Start

1) Product Overview



CardioTouch3000 is an electrocardiogram (ECG) device capable of measuring and recording 12 channels of cardiac signals of patients. The device enables operators to record cardiac signals, review them using various viewing modes, and perform automatic detection of cardiac conditions.

In addition, the device allows operators to enter patient information for it to be printed alongside cardiac signals, so that operators can sort and manage charts conveniently. It is also possible to manage the digital files by transferring the stored data to a networked PC.

Furthermore, user convenience has been further enhanced by addition of a functionality that can initiate ECG recording & saving, filter enhancing, parameter sorting and automatically detecting cardiac signal all at once by pressing one key.

A battery (optional) can be added to the device so that the device can be operated conveniently during bedside visits or emergency conditions.

2) Indication for Use

The CardioTouch3000 ECG Analysis System is intended to acquire, analyze, display and record electrocardiographic information from adult and pediatric populations.

Basic systems deliver 12 lead ECG's, interpretive analysis, vector loops, and can be upgraded to provide software analysis options such as high resolution signal averaging of QRS and P wave portions of the electrocardiogram. Transmission and reception of ECG data to and from a central ECG cardiovascular information system is optional.

The CardioTouch3000 is intended to be used under the direct supervision of a licensed healthcare practitioner, by trained operators in a hospital or medical professional's facility.

Indications

The ECG has proven to be among the most useful diagnostic tests in clinical medicine. The ECG is now routine in the evaluation of patients with implanted defibrillators and pacemakers, as well as to detect myocardial injury, ischemia, and the presence of prior infarction as well. In addition to its usefulness in ischemic coronary disease, the ECG, in

conjunction with ambulatory ECG monitoring, is of particular use in the diagnosis of disorders of the cardiac rhythm and the evaluation of syncope.

Contraindications

No absolute contraindications to performing an electrocardiogram, other than patient refusal, exist. Some patients may have allergies, or more commonly, sensitivities to the adhesive used to affix the leads; in these cases, hypoallergenic alternatives are available from various manufacturers.

3) Recording ECGs during Defibrillation

This equipment is protected against the effects of cardiac defibrillator discharge to ensure recovery, as required by test standards. The patient signal input of the acquisition module is defibrillation-proof. Therefore, it is not necessary to remove the ECG electrodes prior to defibrillation.

When using stainless steel or silver electrodes a defibrillator discharge current may cause the electrodes to retain a residual charge causing a polarization or dc offset voltage. This electrode polarization will block acquisition of the ECG signal. To avoid this condition, use non-polarizing electrodes (which will not form a dc offset voltage when subjected to a dc current) such as silver/silver-chloride types if there is a situation where there is a likelihood that a defibrillation procedure will be necessary.

If polarizing electrodes are used, we recommend disconnecting the lead-wires from the patient before delivering the shock.

Electrode defibrillation recovery is the ability of the electrode to allow the ECG trace to return after defibrillation. We recommend using non-polarizing disposable electrodes with defibrillation recovery ratings as specified in AAMI EC12 4.2.2.4. AAMI EC12 requires that the polarization potential of an electrode pair does not exceed 100mV, 5 seconds after a defibrillation discharge.

4) Product Features

- Print 12-channel (ch) cardiac signals with reports in: 3ch+1 rhythm, 3ch+3 rhythms, 6ch+1 rhythm, and 12ch rhythm patterns on A4 or Letter size printing paper.
- Record and print 12 channel rhythms continuously and simultaneously.
- Calculate heart rate, P-R-T axis and PR, QRS, QT and QTc widths from recorded signals automatically and print them on a report for use in rhythm analysis.
- Get diagnostic reports using automatic detection functionality.
- Able to modify filter setting, signal sensitivity, printing speed, channel view settings and rhythm settings, and print on previously recorded EKG signals to aid data analysis.
- Able to attach a battery so that the device can become portable.
- More than 200 patient's data can be saved and transferred to a PC through LAN as well as through USB Memory.
- It offers various protocols to make it possible to link with hospitals' computing networks. Also File Databases are strengthened.

5) Product Configuration

The CardioTouch3000 system consists of the items below. Unpack the package and check the items below are included. Also, be sure to check for any damage to the body and accessories.

Basic Components and Accessories



① CardioTouch3000 Body (1 EA) – Dimension: 290(W)×300(D) ×97.5(H)mm

② Patient Cable (1 EA) – Length: 3,700mm (Max)

③ Limb Electrodes (1 SET)

④ Chest Electrodes (1 SET)

⑤ ECG Paper (1 EA)

⑥ Power Cable (1 EA) – Length: 3,000mm (Max)

CardioTouch3000 Operation Manual

Optional Components



① Battery (1 EA) – Replaceable and Rechargeable, Lithium ion, 11.1V, 2600mA

② Hanger (1 EA)

③ Cart (1 EA)

CAUTIONS

You may have distortion or signal noise when you use nonstandard or other brand accessories. We strongly recommend you use only the authorized accessories which we supply.

WARNING

How to replace battery: Please make sure you use the right battery we show here.

Otherwise we are not liable for any damages and/or explosion/fire caused by using the wrong battery.




Lithium-ion battery
(11.1V / 2600mAh)



Ni-MH Battery
(12V / 2600mAh)

Body Configuration


▣ Top View

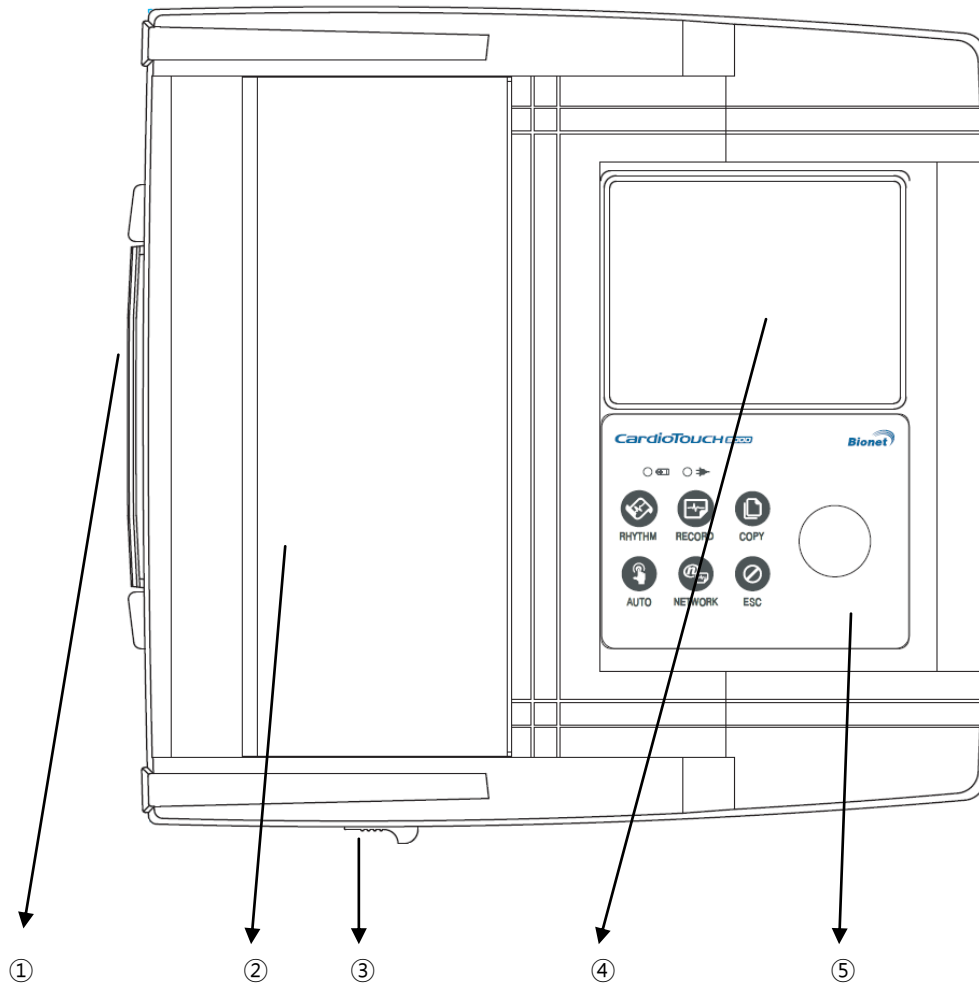

- ① Handle
- ② Printer Cover
- ③ Printer Cover Switch
- ④ LCD
- ⑤ Control Panel


CardioTouch3000 Operation Manual

■ Front View

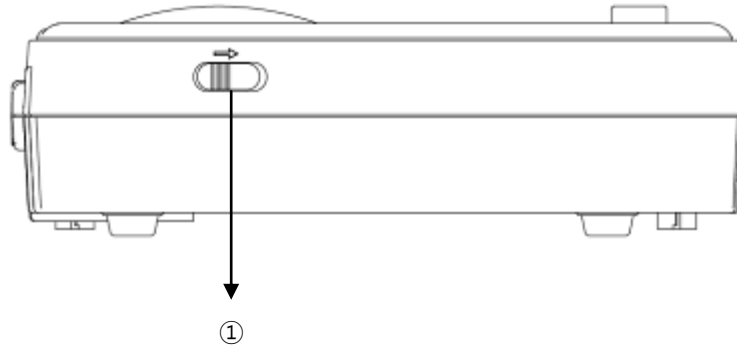

- ① Printer Cover Switch


■ Rear View

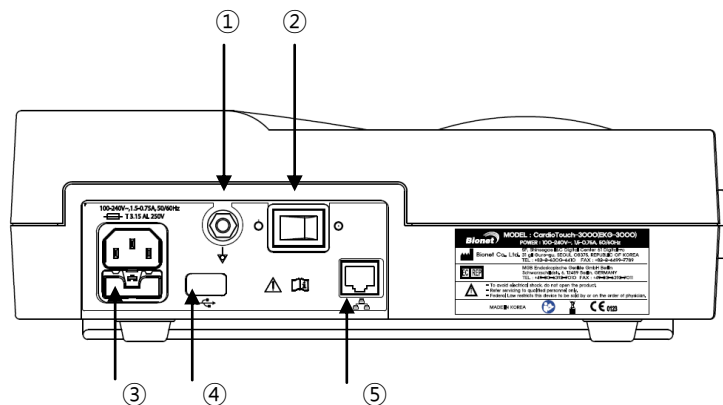

- ① Protective Ground Terminal
- ② Power Switch
- ③ AC Power Connection Port (Fuse : 250V , 5A (High breaking type))
- ④ USB Port
- ⑤ RJ45 LAN Port


CardioTouch3000 Operation Manual

■ Left Side View

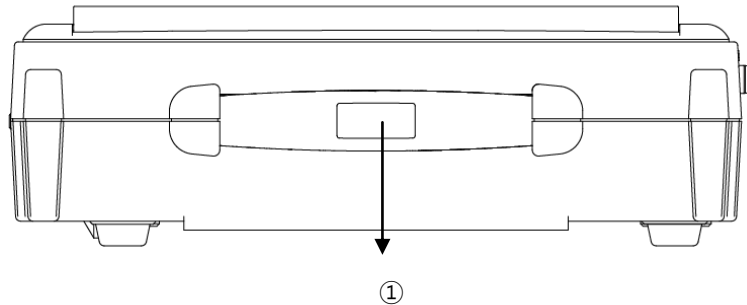

① Handle

■ Right Side View

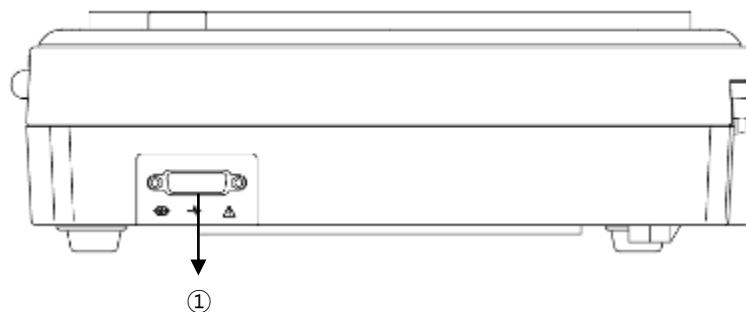

① Patient Cable Connection Port

WARNING

There is a risk of electric shock if the Rest stand of the equipment is damaged or can not be fixed to the product. Do not use the product and immediately ask the manufacturer and the seller for repair

NOTE


To avoid an expected electric shock, do not open the equipment cover or disassemble the equipment. Refer servicing to Bionet, Inc.

CardioTouch3000 Operation Manual

Front Panel



Graphic display window

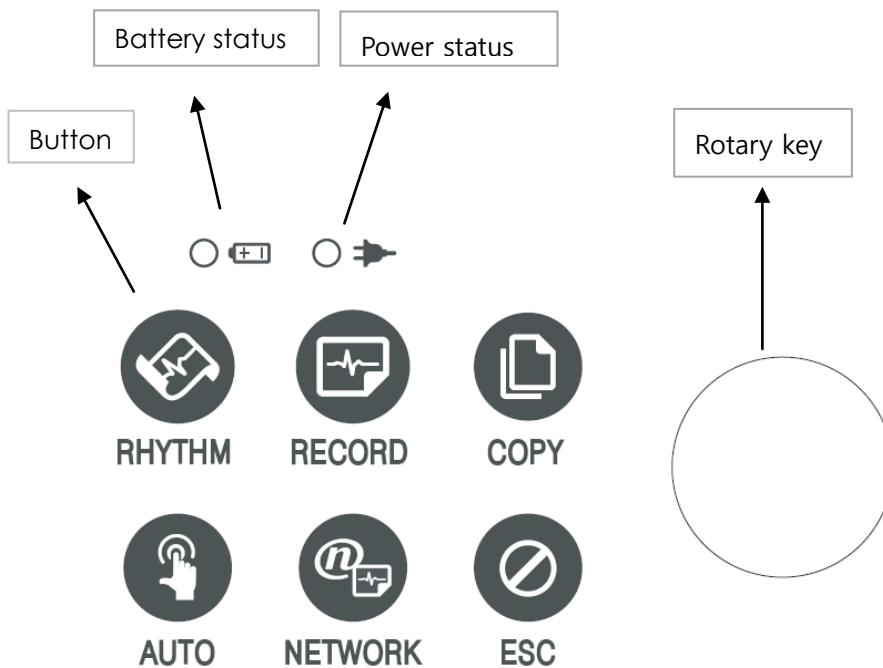

During device boot up, you can see the system software version and the Bionet name.

CardioTouch3000 Operation Manual

Graphic display LCD


The following descriptions explain data on the graphic LCD.







- ① Displays The Heart Rate.
- ② Displays Patient ID.
- ③ Displays USB Connection Status (USB Memory, Barcode)
- ④ Displays Network Connection Status.
- ⑤ Displays Battery Status or AC Power Connection Status.
- ⑥ Displays ECG Sensitivity with One of 2.5, 5, 10, 20, and Auto (mm/mV).
- ⑦ Displays Printing Speed With One of 5, 12.5, 25, 50 And 100 (mm/sec).
- ⑧ Basic Setup, Network Setup, Hospital Setup, ECG Setup, Record Setup and Service Setup.
Modify the preferences of 5 items, i.e.,
- ⑨ Choose the menu where you want to move (ECG, File)

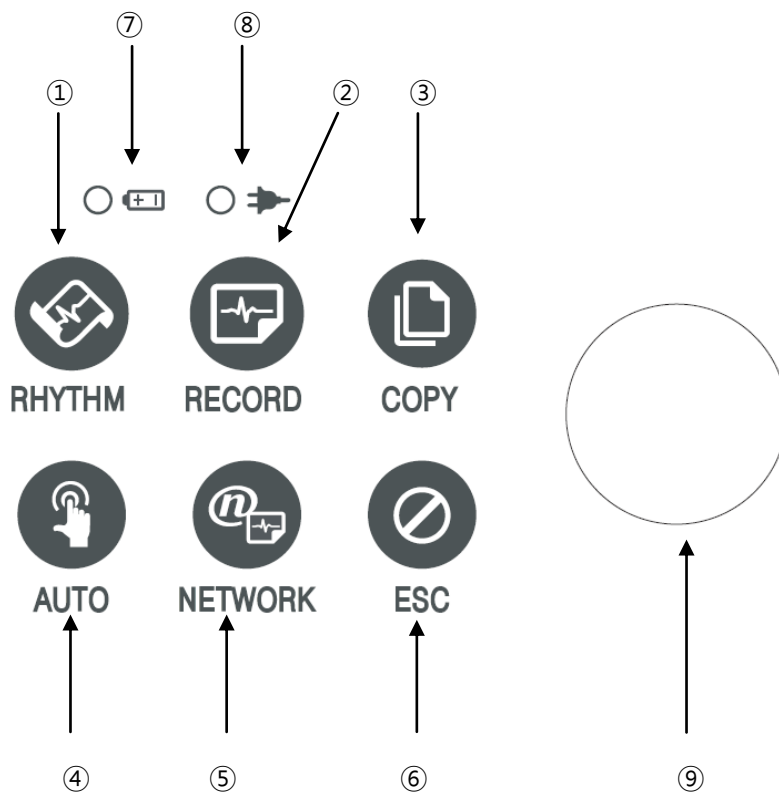CardioTouch3000 Operation Manual



Control Panel





■ Button

①	 RHYTHM	Print out the rhythm data on A4 size or continuous paper.
②	 RECORD	10sec. are saved and are printed as an output form after processing.
③	 COPY	Process the recorded data with or without revision and print them out.
④	 AUTO	It operates the product in the most frequently used operation from the ECG diagnosis test to data saving.


CardioTouch3000 Operation Manual

⑤	 NETWORK	It transfers the test data, or saved data to the server PC.
⑥	 ESC	Cancel commands or change into the former mode for main menu printing.

▣ LED

⑦		A light indicator lets you know that the battery is on and shows your current battery charging status. A red light indicator lets you know that the battery is charging and a green light indicator lets you know the batter is fully charged.
⑧		Green light indicates connection with an AC adapter.


▣ Rotary Switch

⑨		Use when navigating or selecting menu items (Same functionalities can be achieved by touching menu items on the screen).
---	---	---

Power

▣ AC Power

When AC Power is connected to the device, power LED light will be green; if the battery is installed, charging will commence.

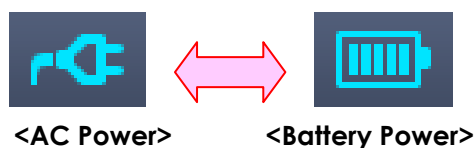





CardioTouch3000 Operation Manual

■ Battery Power

The device will get its power from the battery and the unit will display the battery power icon (shown above), when the system is powered on, the battery is connected to the device and the AC power is disconnected. When the battery power is low, an alarm sound will ring from the device speaker and LCD display will show a "Battery Low" message. Connect the AC power immediately or the device will automatically shut down in 1 minute.

- Time to complete full recharge after full discharge: Max. 3 hours
- Duration of continuous usage after full recharge: if running ECG record in 12 channel format by choosing to set successive function for about 360 min. at 25mm/s and 10mm/mV or in the absence of ECG record, it is possible to record a max. of 200 ECG.

■ Display Battery Power Status

-  : Battery Fully Charged
-  : Battery Charge Half-Full
-  : Battery Charge Low
-  : Battery Almost Fully Drained

■ Replacing Battery

When replacing battery for this device, the same type of battery should be used.

- Type: Lithium-ion battery 3ICR19/65 (11.1V-2600mAh)
- When to replace: Battery will automatically be charged when the device is connected to AC power, and cannot be charged when separated from the device. Battery is designed to have a charging cycle of 300 times or more. If the device only lasts 20 minutes or less on battery power, the battery needs to be replaced. Additionally, when a battery pack is damaged or leaking chemicals, replace it immediately. Do not use damaged battery packs with the device.

WARNING

EXPLOSION HAZARD —

DO NOT incinerate the battery or store at high temperatures. Serious injury or death could result.

6) System Installation

Precautions for Installation

While installing CardioTouch3000, please pay attention to the following items:

- Use the equipment between the ambient temperature 5 to 40°C and humidity 10 - 90%.
- Check the power cord is properly connected, and the probe carefully handled.
- Do not plug multiple cords in a power outlet.
- Install and operate unit on a flat surface.
- If you experience noise, ground the device.
- Do not use a power cord that may make a connection noise.
- Device settings will be recorded in the internal memory even when it is off.
- Prevent any shock or excessive force that may cause damage to the device.
- Place the device away from any dust or flammable materials.

Power Connection

The equipment needs electrical power to operate. Plug in one end of the power cable to wall socket and the other to CardioTouch3000.

Patient Cable Connection

- Connect the patient cable to the patient cable connection port on the right side of the body.
- Connect limb electrodes to RL (N), LL (F), RA(R), and LA (L) leads of the patient cable and chest electrodes to V1 (C1), V2 (C2), V3 (C3), V4 (C4), V5 (C5), and V6 (C6), respectively.

Paper Installation

- Push the printer cover release switch to the right to open the printer door of the CardioTouch3000. Install EKG paper with the side to be recorded appearing on top. Close the cover to finish the paper installation process.

WARNING

No modification of this equipment is allowed.

Do not modify this equipment without authorization of the manufacturer.

If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of equipment.

7) System Start

When every preference is set, the company name will be displayed when the switch is turned on, and then after approx. 10 seconds, the initial menu selection will appear on the screen, as shown in the figure below.



You can move to the main page of the chosen menu by touching the menu on the screen or rotating the key on the control panel.

CardioTouch3000 Operation Manual

The following is the main page of ECG menu.



If you remember the information shown below, it will be easier for you to use the main page of ECG.

Menu	Description
	Heart rate mark
	To Input or check the patient's information
	Barcode reader or USB Keyboard connection mark
	USB memory connection mark
	Network status mark (wired : , wireless :). Move to the Network Setup screen by touching the icon. : The error icon will appear in the event of an IP conflict
	Display battery status
	To set up the size of the ECG waveform
	To set up the speed for printing out the ECG waveform
	To set up the various contents related to the equipment
	To skip to main page of ECG or File management

CardioTouch3000 Operation Manual

The following is the main page of ECG file management, shown when selecting the 'File' menu.

No	ID	Name	Age	Date	Study
1	gby665	JohnnyBurr	35	20180905	ECG
2	vvgfr13	ChuckCorker	73	20180905	ECG
3	vredn79	JoeSenders	39	20180905	ECG
4	tgfreed	ElizabethBalwin	40	20180905	ECG
5	tedx45	CharlesDurbin	51	20180905	ECG
6	dder	PattyDurbin	68	20180905	ECG
7	ghc12	JohnSchumer	27	20180905	ECG

If you remember the information shown below, it will be easier for you to use the main page of ECG file management.

Menu	Description
File 1/2	Going to Patient List
	To skip to previous page
	To skip to next page
	To check the patient's information
	To search the data by inputting what you want to find
Print	To print out the chosen data
Delete	To delete the chosen or all data
Setup	To set up the various contents related to the equipment
ECG	To skip to main page of ECG, file management

Chapter 2. Preparation For Electrocardiography

1) Location of Electrodes

2) Electrode Connection

Patient Cable Connection

How to Attach Electrodes

Countermeasures for Bad Lead Connection

3) Starting ECG recording

4) Basic Setup

General

Patient Information Input

Sensitivity Setup

Printing speed Setup

5) Screen Printing

6) Rhythm Mode Printing

7) Record Mode Printing

10 seconds ECG recording

8) Printing Form

9) Copy Mode Printing

10) System Setup

Basic Setup

Network Setup


Hospital Setup

ECG Setup

Service Setup

1) Location of Electrodes

Attach electrodes to the patient's body to record an electrocardiogram of twelve standard leads [I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6], as shown below.

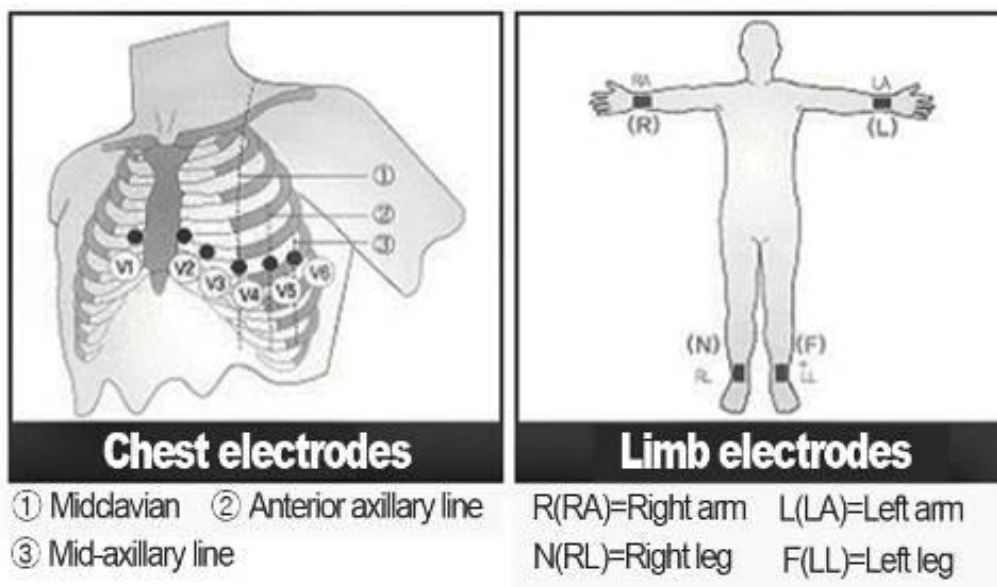

Limb electrodes are located as follows.

- RL (N): Right Leg
- LL (F): Left Leg
- RA (R): Right Arm
- LA (L): Left Arm

Chest electrodes are located as follows.

- V1 (C1): Fourth Intercostal Space At The Right Border Of The Sternum
- V2 (C2): Fourth Intercostal Space At The Left Border Of Sternum
- V3 (C3): Midway Between Location V2 and V4
- V4 (C4): At The Mid-Clavicular Line In The Fifth Intercostals Space
- V5 (C5): At The Another Axillary Line On The Same Horizontal Level As V4
- V6 (C6): At The Anterior Axillary Line On The Same Horizontal Level As V4 and V5

2) Electrode Connection

* Check Points

- Check the status of equipment and attachment to patient's body before measuring.
- Check for any mechanical danger.
- Check cables and accessories connected to the outside.
- Check all the devices to measure the patient.

Patient Cable Connection

Connect the device cable to the patient cable connection port on the right side of the main body, and then connect electrodes for extremities to the terminal of patient cable RL (N), LL (F), RA(R) and LA (L), while you can connect electrodes for the chest to the terminal of 'V1 (C1), V2 (C2), V3 (C3), V4 (C4), V5 (C5) and V6 (C6).

How to Attach Electrodes

Lay down the patient on the bed and let the patient relax. Attach the electrodes cleaned with water or antiseptic alcohol to the patient's body. If it is hard to attach the electrodes due to the patient's hairs or curves of the body, apply the ECG gel. Please make sure the electrodes are attached correctly as explained above. When using the gel, please remove the gel after use as it may cause noise of the ECG signals.

WARNING

Use only electrodes and patient cables provided from Bionet, Inc. Bionet America, Inc. will take no responsibility for any accidents involving 3rd party accessories.

CardioTouch3000 Operation Manual

Countermeasures for Bad Lead Connection

After turning on the power switch, enter the ECG test to check waveforms of all leads. On this menu, operators can check waveforms of all leads and their noise, and may see "Lead Fault" message displayed on the LCD screen.

NOTE

Message will be displayed when Lead fault is set to ON in System Setup, and will NOT be displayed when set to OFF..

Lead Fault! All Leads.

The following two cases could cause a lead fault error.







- First case, leads are detached from the patient's body. In this case, re-attach leads normally.
- Second case, signal conductivity between leads and the patient's body is low. In this case, replace the electrodes.

If the two above cases have been performed and the operator is still experiencing noise or lead fault conditions, the patient cable may be faulty. Please contact the Bionet service center.

3) Starting ECG recording

- Connect leads to the patient, and then turn on the device in accordance with the preparation steps for measuring electrocardiogram.
- Check the setting for filter, size of signals, printing speed, channels and rhythm. Set up the new values if you want to modify them.
- Input the patient's information
- If the waveform displayed on the LCD is abnormal or has excess noise, follow the instruction of "Countermeasures by Bad Lead Connection".
- If the waveform displayed on the LCD is normal, press the 'RECORD' key to record the electrocardiogram of the patient.
- Press the 'COPY' button to print out the previously saved 10-second data or modified data.
- Press the 'RHYTHM' key to monitor the ECG waveform through the printer not through the LCD.
- Press the 'ESC' key to stop the operation while printing or recording.

The following buttons will be most commonly used when operating CardioTouch3000.

 RHYTHM	Print out the rhythm data on A4 size or continuous paper.
 RECORD	10sec. are saved and are printed as an output form after processing.
 COPY	Process the recorded data with or without revision and print them out.
 AUTO	It operates the product in the most frequently used operation from the ECG diagnosis test to data saving.
 NETWORK	It transfers the test data, or saved data to the server PC.
 ESC	Cancel commands or change into the former mode for main menu printing.

4) Basic Setup

General

When turning on the system, LCD will display component values set by the current system in order of heart rate, patient information, connection status of external device, power state, size of ECG signal and output speed.

They can be modified in two ways.

First, these configuration values can be changed using either a rotary key or touch screen. Second, with the rotary key, the desired menu and changed configuration values can be set easily.

Same functionality can also be achieved with the touch screen by pressing and changing the desired information on the LCD.

Changing The Configuration Value With The Rotary Key

In this case, please follow these steps: enter Menu Mode, navigate and select Menu and then change values.

- Entering Menu Mode

The rotary key is supposed to rotate clockwise. When the user selects the ECG Main, the initial focus will point to the 'patient' item. The following is the page you will see when you rotate the key clockwise and click it to enter the speed menu.



CardioTouch3000 Operation Manual

- Menu Navigation

Operate by rotating the rotary key. Menu movement will occur in the same direction as the direction of the key's rotation.

- Menu Selection

Initiate by pressing the rotary key. Menu box for setting of the selected menu will be displayed.

- Modifying setting values

Move to desired setting, then modify & select values in the same manner as menu movement shown above. When you select either 5mm/sec, 12.5mm/sec, 25mm/sec, 50mm/sec or 50mm/sec at the 'Speed Setting' POP-UP, the POP-UP window will disappear.

Using the touch screen to modify the information

When using the touch screen, only two procedures, menu selecting and information modification, are required.

- Select Menu

Press the menu on the screen. Unlike the rotary key, the menu box you want to set up will show at the same time.

- Change the value

Press the chosen item on the screen after moving to the page.

CardioTouch3000 Operation Manual

Patient Information Input

In this menu, operators can input the patient's ID, name, age, sex, height, weight.

The screenshot shows a 'Patient Information' dialog box with the following fields and values:

Field	Value
ID	bionet1
Name	bionet
Age	25 years
Gender	Female
Height	182 cm
Weight	87 kg

Buttons: OK, Cancel, New

When the user selects the line of ID, NAME, Department, a keyboard will show up on the screen. The default of every line is a blank and if there is a certain figure on the line, it should be shown exactly when loading the keyboard window.

When the user wants to set the data after inputting, the user can press OK button on the keyboard and if the user wants to cancel the setting, pressing ESC button will do it and the keyboard window will disappear.

The screenshot shows a virtual keyboard for the ID field. The text 'bionet' is entered in the ID field. The keyboard has the following layout:

Row	1	2	3	4	5	6	7	8	9	0
Row 1	1	2	3	4	5	6	7	8	9	0
Row 2	q	w	e	r	t	y	u	i	o	p
Row 3	a	s	d	f	g	h	j	k	l	'
Row 4	z	x	c	v	b	n	m	,	.	-
Row 5	↑	CAP	@	<	SPACE	OK				

NOTE

When entering an ID, it is impossible to enter special characters such as " , < , > , ? , / , * , | , : , \ , etc. (these invalid characters are set to be inactive)

When entering an ID, an English keyboard will be displayed.

When entering an ID by connecting to an external keyboard, proceed by using general alphabet and numbers only. If you enter an ID using Latin extended characters and Russian, an error may occur when transferring files via PC and USB.

CardioTouch3000 Operation Manual

When the user selects the line of Age, Height or Weight, a keypad will show up on the screen. The default of every line is a blank and if there is a certain figure on the line, it should be shown exactly when loading the keypad window.

Enter an age and select the year of birth for >1 year old; enter age in the form of week or day for 1< year old. Automatic diagnosis is available from postnatal day 1 for pediatrics.

When the user wants to set the data after inputting, the user can press OK button on the keypad and if the user wants to cancel the setting, pressing ESC button will do it and the keypad window will disappear.



CardioTouch3000 Operation Manual

When clicking the Gender Line, the letter Male or Female will be shown respectively. After selecting the data which you want, input all the information of the patient. When pressing OK button, the information will be saved, while pressing cancel button will cancel the setting. The default of the Gender line will be set as a Female.

The screenshot shows a 'Patient Information' dialog box with the following fields and values:

Field	Value
ID	bionet1
Name	bionet
Age	25 years
Gender	Female
Height	182 cm
Weight	87 kg

Buttons: OK, Cancel, New

NOTE

Patient information will be initialized if the language preference is changed during the system setup.

NOTE

Patients' ID can be registered using a barcode scanner. Patients' ID will be entered automatically when the user scans the barcode on ECG main screen.

CardioTouch3000 Operation Manual

Normally barcode scanners are compatible with all products. However, because of inconsistencies in barcode scanner manufacturers implementation of input methods, you need to verify the scanner is supported by Bionet.

- Input methods supported by Bionet : International standards, USB
- Products below are tested and confirmed by Bionet.

No	Manufacturer	Product name	Product image
1	Symbol	LS-2208	
2	ZEBEX	Z-3110	
3	Honeywell	MS5145	

CAUTIONS

You must read the user manual of the barcode scanner to get complete information about it. The initialization codes for various products would be included. You must run the initialization after verifying the input type.

CardioTouch3000 Operation Manual

Sensitivity Setup

This is a functionality that can modify sensitivity of the output signal, when the outputting signal's amplitude is too large and causes overlap with neighboring signals, or too small so that signal diagnosis is difficult.

Users can set all 12 channels to 2.5mm/mV, 5mm/mV, 10mm/mV or 20mm/mV. Users can also set it to Auto which will automatically set the sensitivity level for each channel; for example limb electrodes(I, II, III, aVR, aVL, aVF) to 10 mm/mV and chest electrodes(V1, V2, V3, V4, V5, V6) to 5mm/mV. Note that 10mm/mV means printing 1mV signal as 10mm in amplitude on the printing paper.

The left side of the graph display will show the channel name and a bar graph, so users can notice recently changed values easily. However, for 20mm/mV, the size of signal is set to be shown for 6 signals only to avoid overlapping signal size displays.



CardioTouch3000 Operation Manual

Printing speed Setup

This is a function to adjust the width of the printing signal. The values are available in 12.5mm/sec, 25mm/sec and 50mm/sec, 100mm/sec. If you set 25mm/sec up, the ECG signal will be recorded in 25mm for one second. Therefore, the 12.5 mm/sec width is equal to half of the 25 mm/sec width; the 5 mm/sec width is equal to 1/5 of the 25 mm/sec width.

The 50mm/sec width is equal to twice that of the 25mm/sec width; the 100mm/sec width is equal to twice that of the 50mm/sec width. Therefore, if you want to have a wide signal, set up the bigger value.



NOTE

When monitoring is set to 'Mon Size' -> 'A4', the printing speed is 25mm/s.

5) LCD Monitor display

Monitor mode display shows the currently measured electrocardiogram signal in real time. The monitor mode display function is used for the following two purposes: One is to check whether the signals from all channels are coming out properly; the other one is to see the status of the electrocardiogram of the patient for an extended time.

The printing speed can be selected to 5mm/s, 12.5 mm/s, 25 mm/s, 50mm/s and 100 mm/s, and the sensitivity can be selected to 2.5 mm/mV, 5 mm/mV, 10 mm/mV, 20 mm/mV, and Auto.

Displayed heart rate is the average of 4 heartbeats

NOTE


The heart rate detection range is 30 to 300 bpm, and the error range is ± 2 .

6) Rhythm Mode Printing

In rhythm mode printing, the measured ECG signals would be printed in real-time. Rhythm mode printing allows operators to perform the following two functions: one is to verify that signals in all channels are active before recording ECGs, and the other is to print the ECGs rhythm status of the patient for a longer duration.


Printing speed can be set to 5mm/s, 12.5 mm/s, 25mm/s, 50mm/s, 100mm/s and signal level can be set to 2.5mm/mV, 5mm/mV, 10mm/mV, 20mm/mV, and Auto.

When pressing the 'RHYTHM' key on the control panel, you will see the message as shown below and printing will start in accordance with the speed you set up.

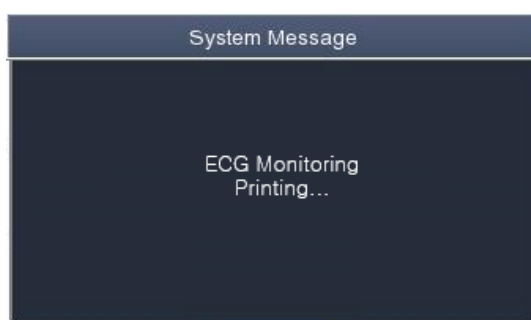

CardioTouch3000 Operation Manual

Press 'ESC' key to stop printing the ECG rhythm. After pressing the key, the system will stop printing and the display system setting after indicating the following message.

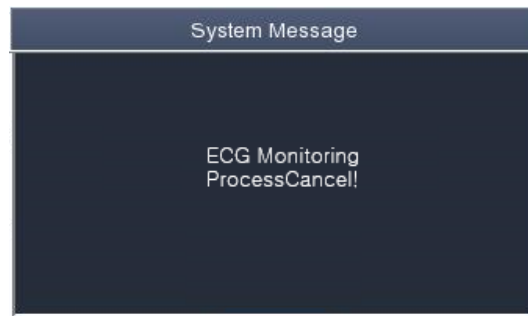

NOTE

Even if ECG output speed is set as 100 mm/sec, it is not supported when outputting monitoring.

NOTE


You should avoid connecting or disconnecting a USB device while printing out, as it could cause the printer module to rattle.

7) Record Mode Printing


For the record mode display, save the electrocardiogram in the memory first, and apply the filter which is set up by the user to the saved electrocardiogram, and then extract the parameters for the heart rate, PR interval, QRS duration, QT/QTc, and P-R-T axes. Then, print out the heart rate according to the sensitivity, printing speed, and channel form that are set up by the user.

10 seconds ECG recording


When one of 3ch+1rhy, 6ch+1rhy, 12ch, 3ch+3rhy and beat is selected for the channel form, press the 'RECORD' key to display the saving time as below and save the 10-second data.

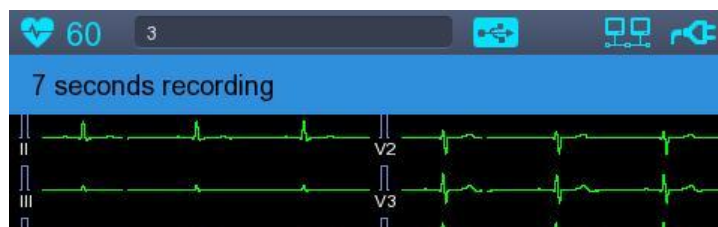

After saving the 10-second data and the parameters for diagnosis will be extracted, it will show the message as shown below and printing will begin.

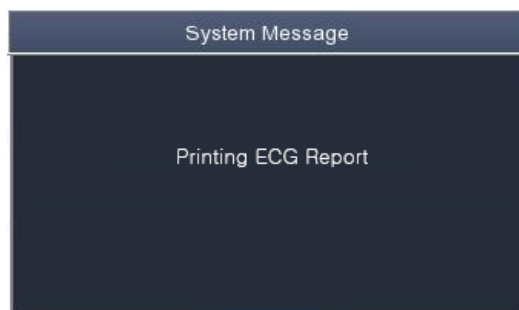

Enter 'ESC' key on the operation panel in order to stop while obtaining or printing out data. In this case, display the following message on the screen; cancel ECG recording if saving, otherwise end output while printing out.

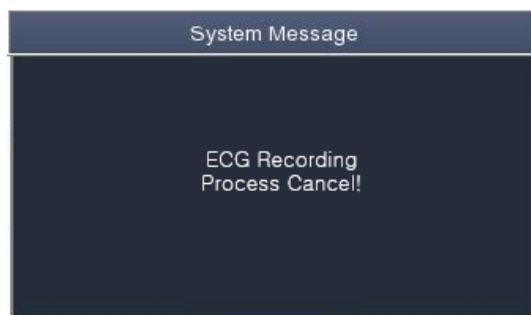

8) Printing Form

In this section, descriptions and samples of various printing forms are provided.

■ Rhythm Printing Form

- 3CH printing: To print 3 channels at the same time (I ~ III / aVR ~ aVF / V1 ~ V3 / V4 ~ V6)
- 6CH printing: To print 6 channels at the same time (I ~ aVF / V1 ~ V6)
- 12CH printing: To print 12 channels at the same time (I ~ V6)

■ Record Printing Form

- '3CH+1': I, II and III will be recorded for the first 2.5 seconds, V1~V3 for the next 2.5 seconds, V4~V6 for the next 2.5 seconds and 1 channel of rhythm will be recorded for 10 seconds at the bottom.
- '6CH+1': I~ aVF will be recorded for the first 5 seconds and V1~ V6 for the next 5 seconds. And 1 channel of rhythm will be recorded for 10 seconds at the bottom.
- '12CH': 12-channel rhythm will be recorded as I~ V6 at the same time for 10 seconds.
- '3CH+3': I~ III will be recorded for the first 2.5 seconds, aVR~ aVF for the next 2.5 seconds, V1~V3 for the next 2.5 seconds and V4~V6 for the next 2.5 seconds. And 3 channels of rhythm will be recorded for 10 seconds at the bottom.

■ Form of Beat Report Output


- Choose an additional output form after printing out diagnosis; either Text, Guide or Vector.
- Text: to output representative beat and each diagnostic parameter. Output will come at 50mm/sec if the output speed is set at '5, 12.5 and 25 mm/sec', whereas it will come at 100 m/sec if set at '50 and 100mm/sec'.
- Guide: to output representative beat and each diagnostic parameter, with diagnostic guide for arrhythmia. Output will come at 50 mm/sec regardless of the output speed.
- Vector: to output representative beat and each diagnostic parameter and Vector of QRS, as indicated by the arrow. Output will come at 50 mm/sec regardless of the output speed.

CardioTouch3000 Operation Manual

Rhythm Report Form (3CH)




Rhythm Report Form (6CH)

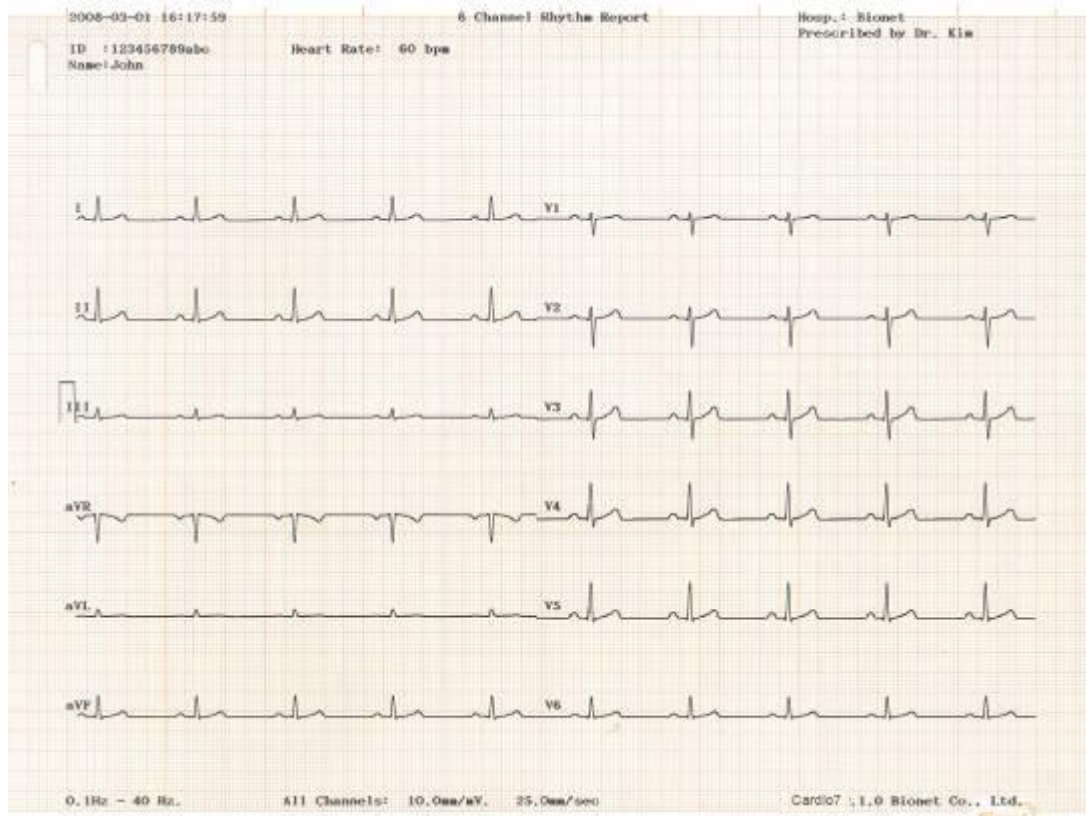


CardioTouch3000 Operation Manual

Rhythm Report Form (12CH)

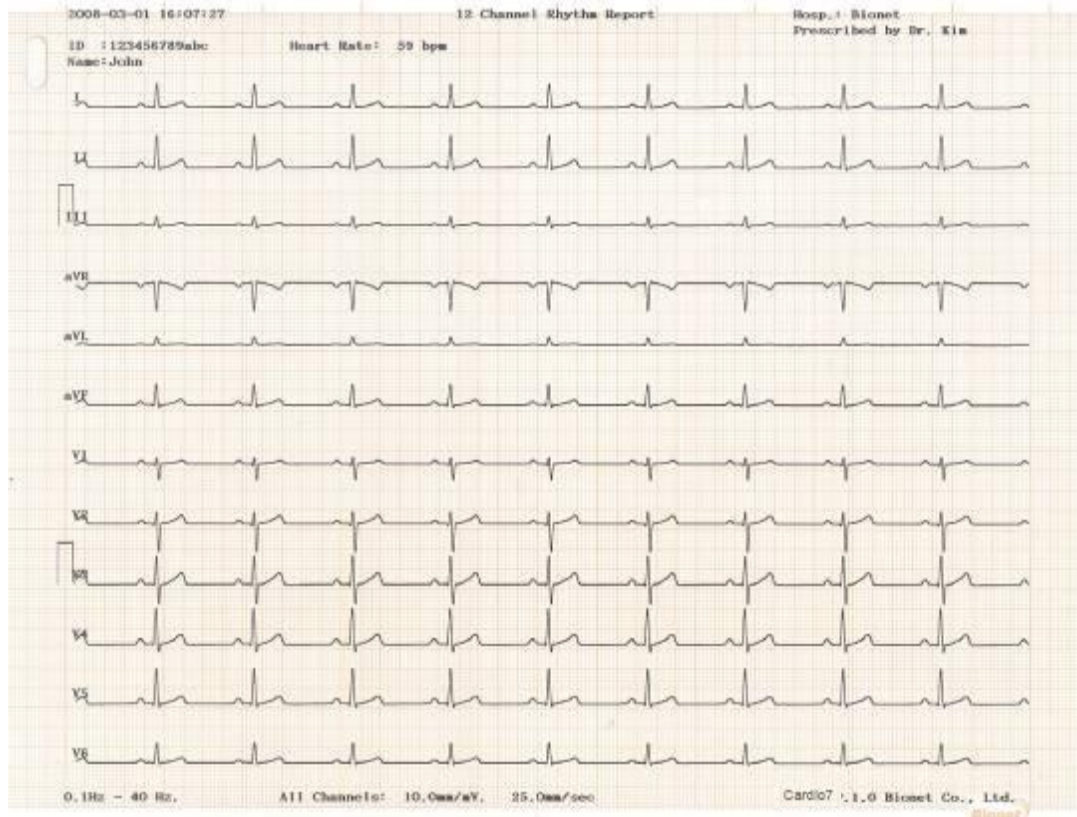


Record Report Form (3CH + 1 RHY)

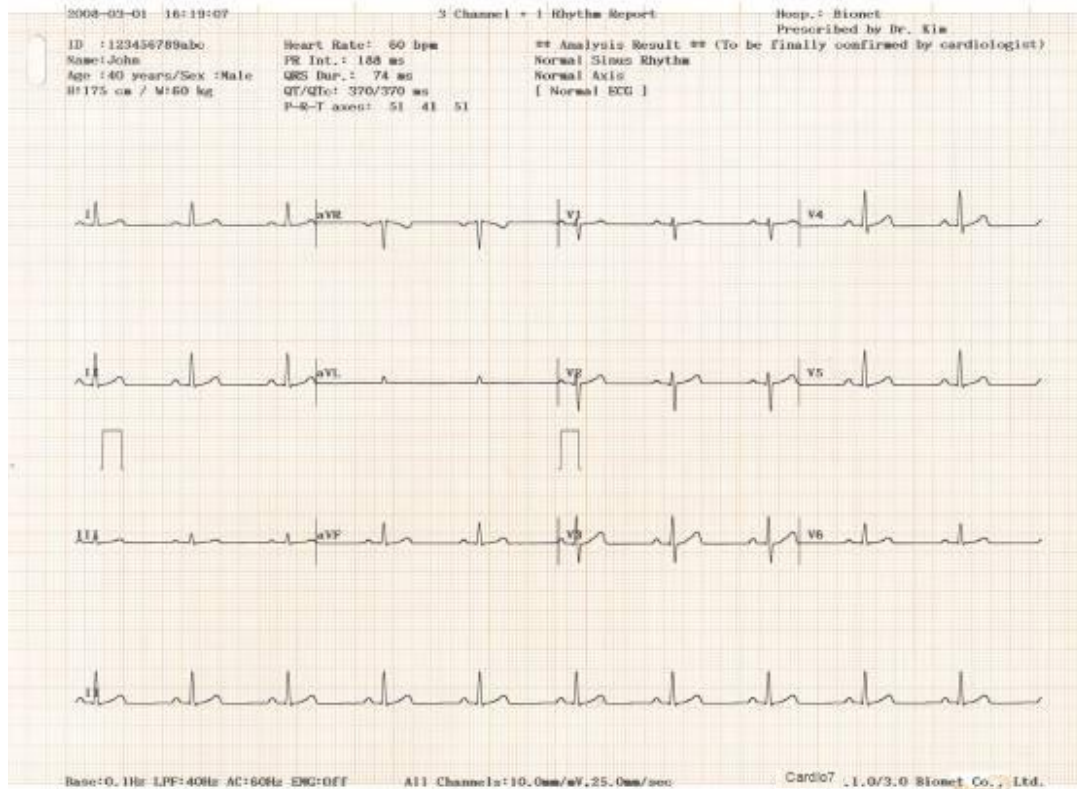


CardioTouch3000 Operation Manual

Record Report Form (3 CH + 3 RHY)

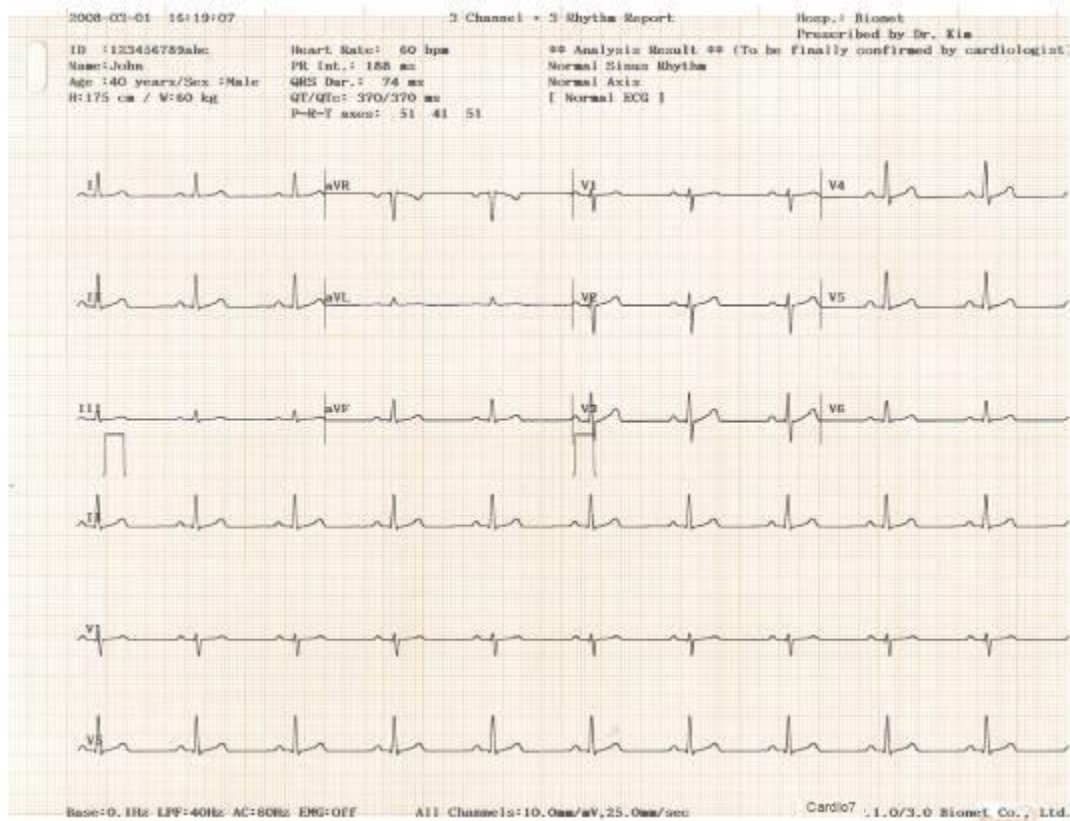


Record Report Form (6 CH + 1 RHY)

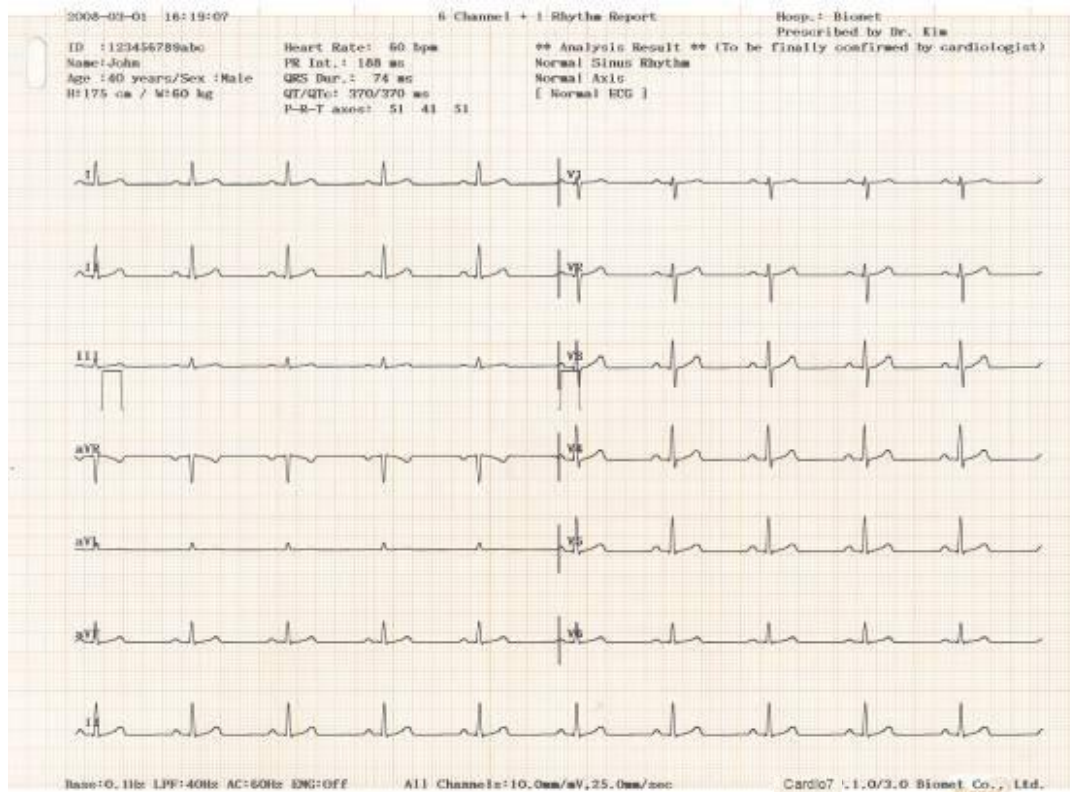


CardioTouch3000 Operation Manual

Record Report Form (12 CH)

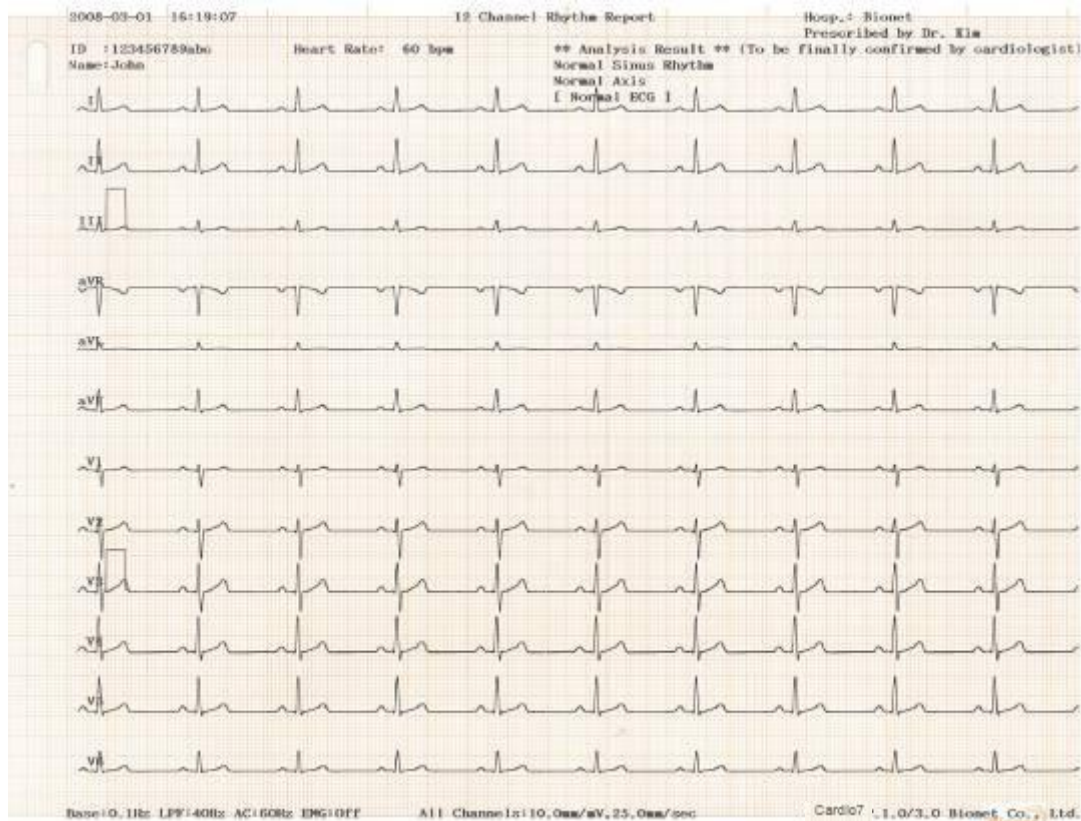


Record Report Form (BEAT REPORT - TEXT)

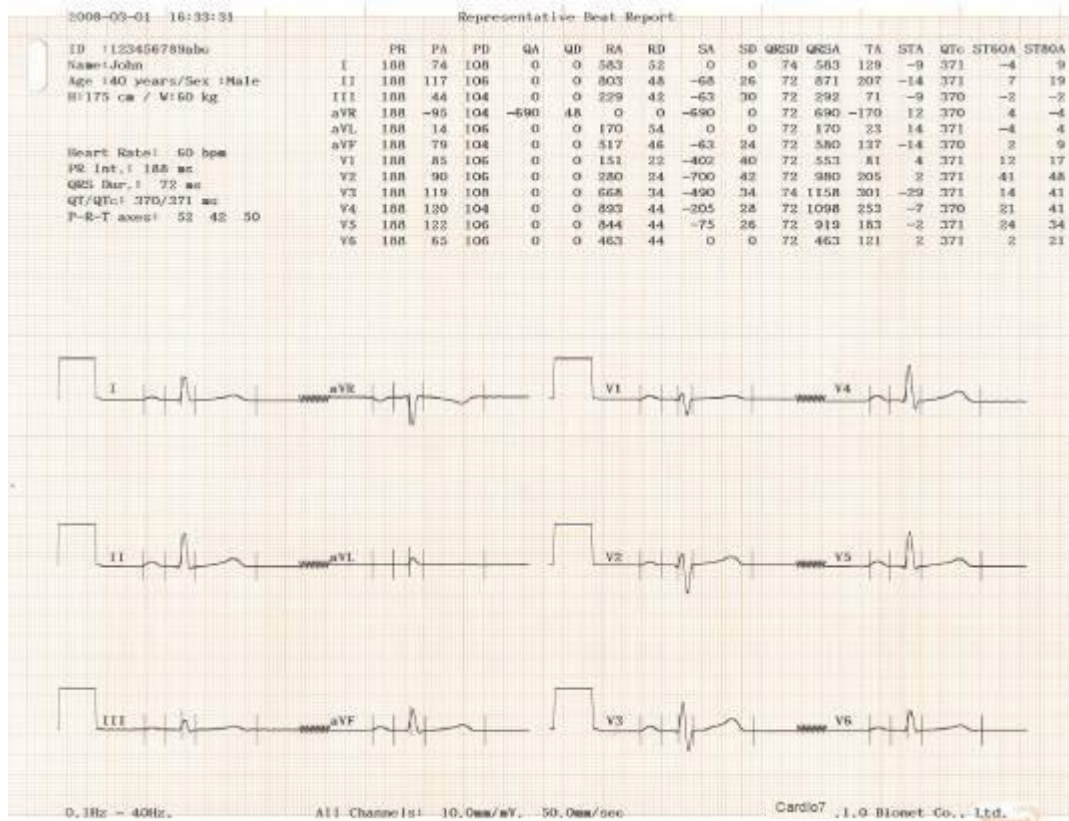


CardioTouch3000 Operation Manual

Record Report Form (BEAT REPORT - VECTOR)

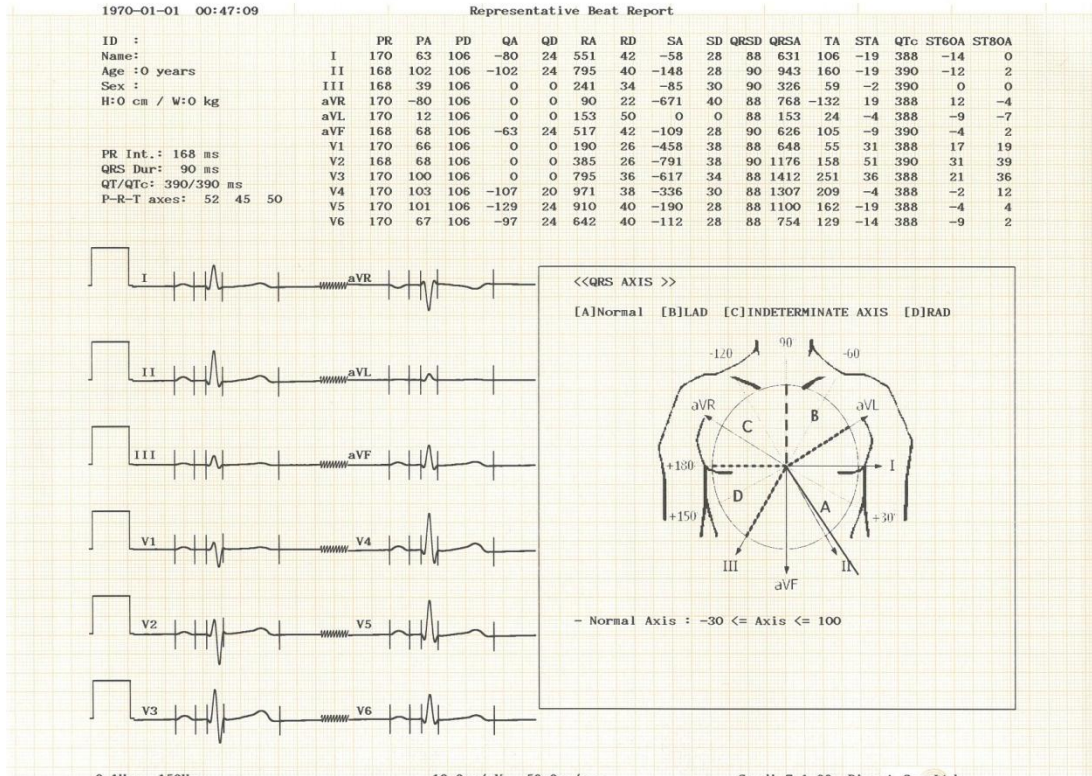


Record Report Form (BEAT REPORT - GUIDE)

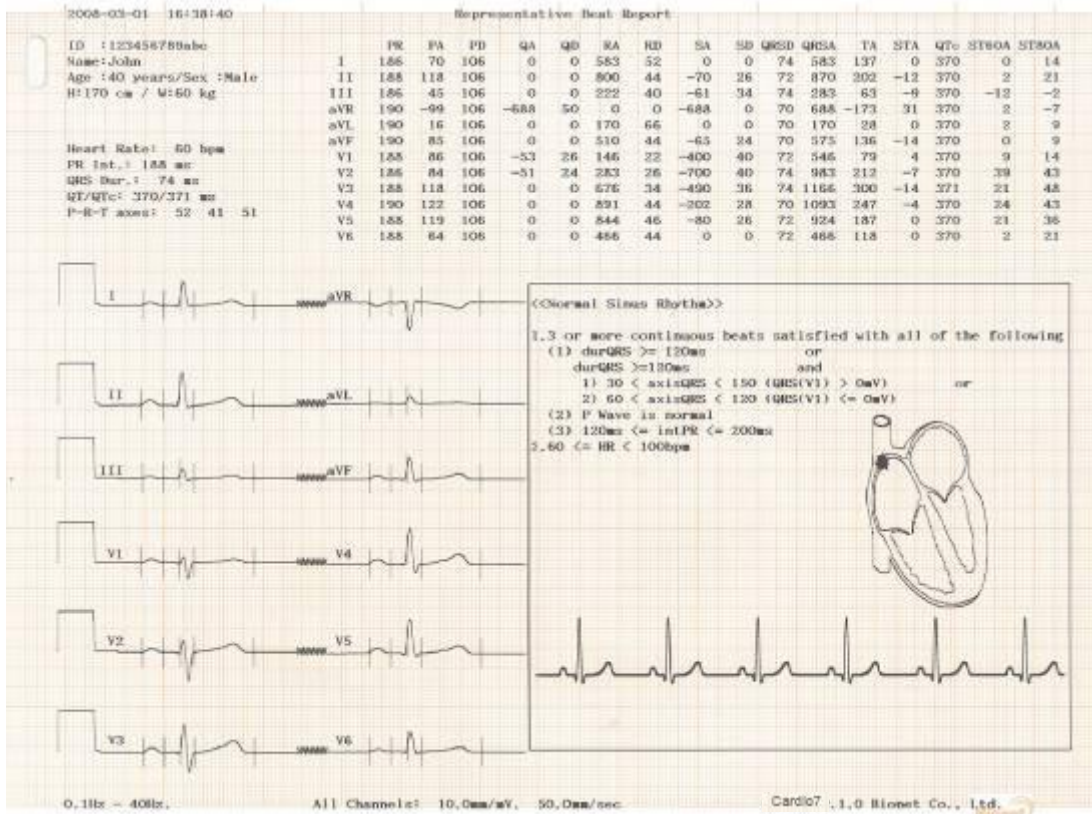

CardioTouch3000 Operation Manual

Beat Report Parameters

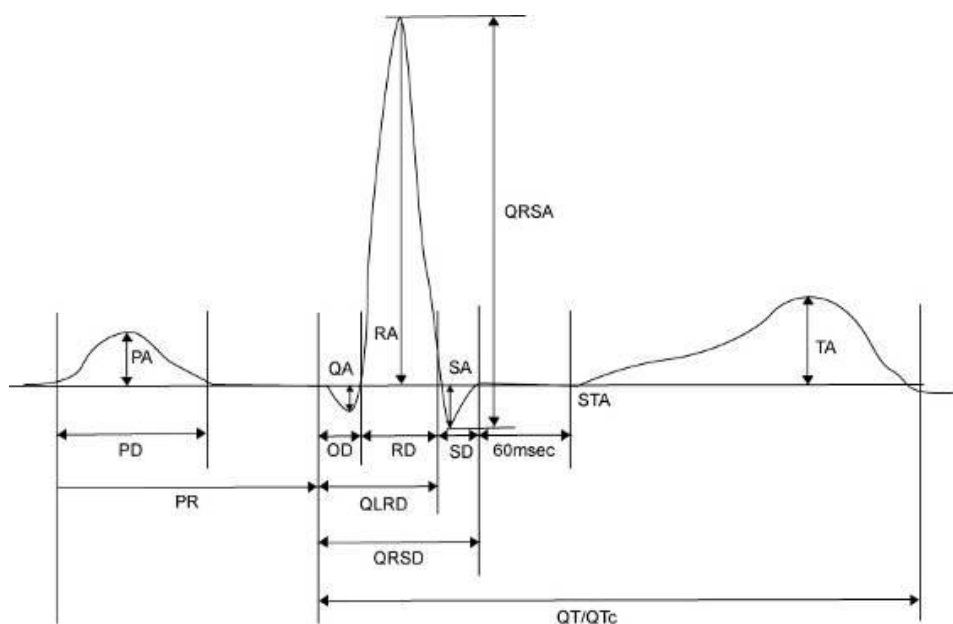

- PR : PR Interval
- PA : P Amplitude
- PD : P Duration
- QA : Q Amplitude
- QD : Q Duration
- RA : R Amplitude
- RD : R Duration
- SA : S Amplitude
- SD : S Duration
- QRSD : QRS Duration
- QRSA : QRS Amplitude
- TA : T Amplitude
- STA : ST Amplitude
- QTc : collect Q-T Interval
- ST60A : ST 60ms Amplitude
- ST80A : ST 80ms Amplitude

NOTE

The unit of the interval of the parameter output (duration, Interval) in the Beat Report is 'ms', and the unit of amplitude is 'uV'.

NOTE

Dextrocardia


The human heart is normally located on the left side of the chest. However, in rare cases it is located on the right side chest, which is a condition called Dextrocardia.

There is potential for dextrocardia if:

- P, QRS and T are all reversed on Lead I.
- aVR and aVL as well as lead II and III are switched with each other.
- As you proceed from V1 to V6, R wave is reducing on Chest lead.

For patients with dextrocardia, the result of a normal automated diagnosis can be obtained if measurement is conducted by switching electrodes over, as follows:


- Switch between Right hand (R) and Left hand (L) electrodes.
- Attach chest leads starting from the left side, rather than from the right.

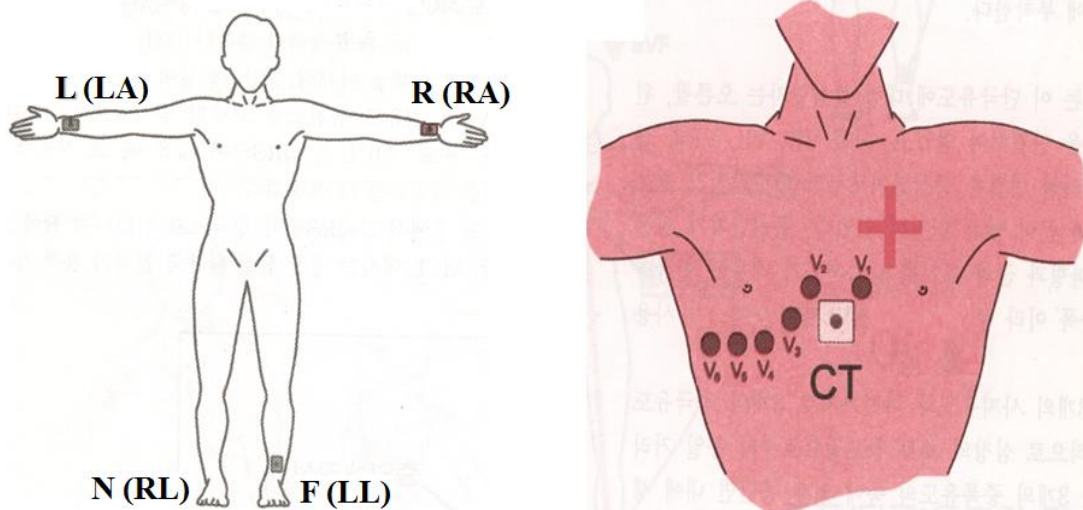


9) Copy Mode Printing

Copy Mode Printing is a function to print out the same report as the previously printed report. In Copy Mode, users can also print out the same previous data but with changed settings such as filter type, sensitivity, printing speed, channel form, and rhythm number if more than one rhythm is recorded.

Copy Mode can be entered by pressing the 'COPY' key following an EKG recording. After pressing the COPY Key, the following message will be displayed on the LCD screen, and the printing process will start:

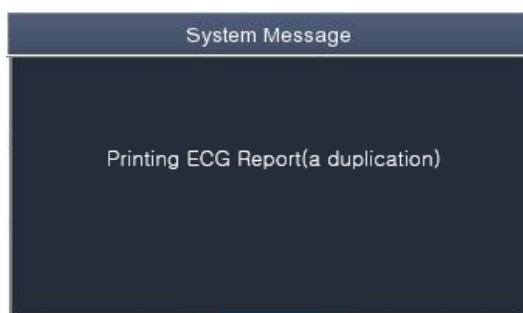


If Copy Mode Printing is called without performing an EKG recording, there would be no EKG data to copy. In this case, the following message box will be displayed:

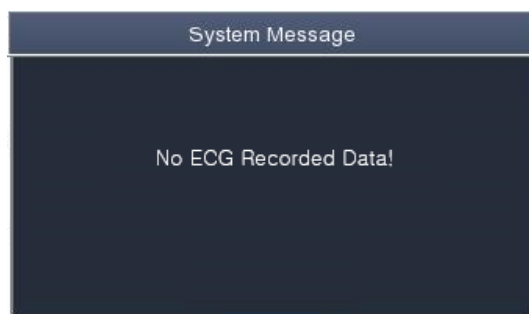


CardioTouch3000 Operation Manual


If you change the filters on a previously saved ECG, it will adapt to the new filter. When the new filter is adapted completely, it will start printing after showing the message as below.

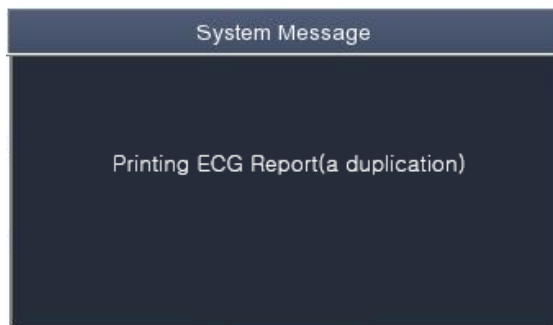

When printing is abnormally completed, the state of the system setting will be displayed after showing the message as shown below for a second.

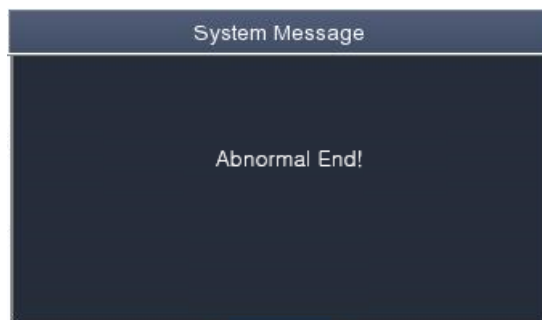

If you want to stop the process of printing, press the 'ESC' key. Then the process will stop and you will see the ECG Main page after the message shown for a second as below.

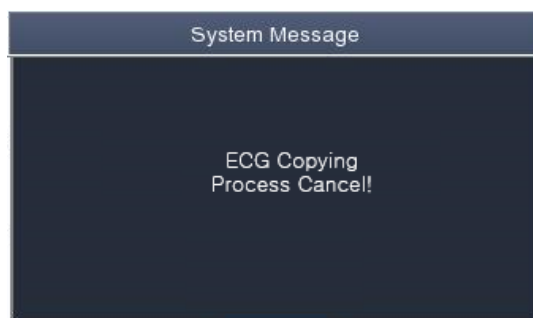

CardioTouch3000 Operation Manual

10) System Setup



Choose the 'Setup' button on the initial ECG main screen to change preferences.

Setup menu consists of Basic Setup, Network Setup, Hospital Setup, ECG Setup, Service Setup and you can change the setting at each window. The main page shows Basic Setup.

After finishing the settings, press 'OK' to save the new information or press 'Cancel' to cancel it. You can choose 'Exit' or 'ESC' button to exit from the setup menu.

'Default' button is used to default every setting.

Basic Setup

When choosing Basic menu at the left side of the Setup page, you will see the page for Basic Setup.

Basic Setup consists of Date, Time, Touch screen, Start Option, Unit, Date Type and Device Name menus.



CardioTouch3000 Operation Manual

Setup date and time

After choosing the Setup menu, press the Basic button then the focus will go to the 'Date' item.

By using the touch screen or rotating the rotary switch, choose the Date (yyyy/mm/dd) or Time (hr/min/sec), then the keypad will show up. You can set up the information by using this keypad.


The Focus will go 'year → month → day → hour → minute → second' consecutively.



Touch Setup

This is a menu to set up the coordinates of the touchpad. When selecting the 'Touch' menu at the Basic Setup page, setup page will disappear and the coordinates for setting calibration will appear as shown below. Follow the instructions shown on the screen, and then the coordinates will be set up.

Please input the information in the right place, otherwise the touchpad may not work properly.

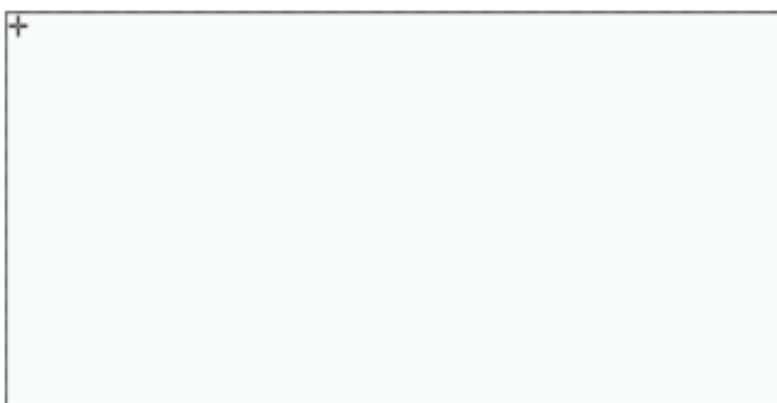

NOTE

When the user selects the Touch Setup menu, all windows will disappear and the touch screen setup will start. No keys will be operational until the Touch Setup is complete.

CardioTouch3000 Operation Manual

Language

As shown below, when selecting 'Language menu at the Basic Setup page, you can choose the language among

English, French, German, Italian, Korean, Polish, Portuguese, Rumanian, Russian, Spanish and Turkish.

Select the language you want to use and click the OK button and then you can use the service in the chosen language.



NOTE

Patient information previously entered will be initialized if the language preference is changed.

NOTE

If language preference is set as "KOR," only the menu on the LCD screen will be shown as Korean, whereas the output will be in English when printing out.

CardioTouch3000 Operation Manual

Start Option

You can set up the index page shown at the time of initial boot up, according to the common dialog.

When clicking the 'Start Option' button, a small menu window will appear and you can select the initial page you want to have among MAIN, FILE, ECG.



Units

This menu is to select the units for height and weight of the patient. When you click 'Units' button, cm/kg or Inch/Lbs will be shown.

If you select Inch/Lbs unit, the patient's height information will be expressed in *** Ft (Feet) *** Inch.



CardioTouch3000 Operation Manual

Date Type

As seen in the figure below, the date format on the top of the paper can be changed while printing out monitoring and recording results if you choose 'Date Type' from the Basic Setup menu. Choose the date format to be used, and then click the 'OK' button; the date format is converted to the one selected and can be seen on the output.




CardioTouch3000 Operation Manual

Color Setup

The device offer two type of LCD screen for easy reading of wave form. In case of choosing "Black" color, it reflect wallpaper as black color and wave color turn to green color. In case of choosing "White" color, the wallpaper reflect white, wave color turn to black.



[Black& Green Color]

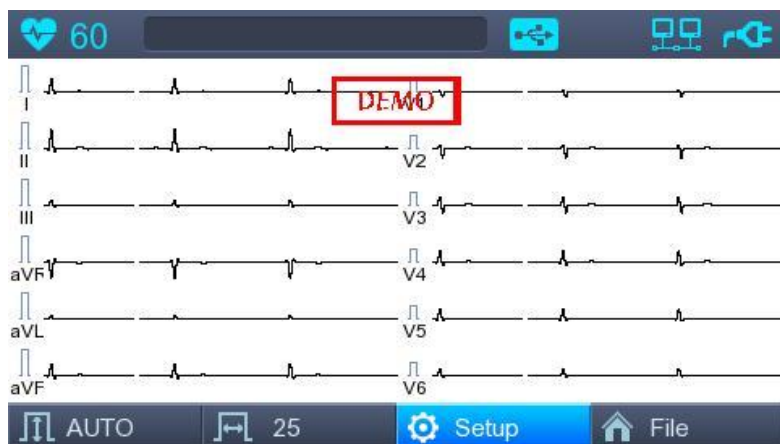

[White& Black Color]

CardioTouch3000 Operation Manual

Network Setup

When connecting the equipment with LAN to interface with the outside PC, network setup is needed. Network IP is required and DNS is not used.

When selecting the Network menu at the left side of the System Setup page, Network Setup page will show up at the right side.

Menu Item	Value
- Net Device	Wired
- Configure	Manually
- Device IP	129.210.48.4
- Subnet Mask	255.255.240.0
- Gate way	129.210.53.1
- BMS	

The mode of network connection can be selected as Wired or Wireless in 'Net Device' setup.

The automatic IP(DHCP) or fixed(Manually) can be selected through the 'Configure'.

If the Configure is setup as automatic IP(DHCP), the Device IP, Subnet Mask, Gateway value is received automatically from the DHCP server on your network. If the Configure is set to manually, the user has to enter the Device IP, Subnet Mask, Gateway directly.

When selecting one of the 'Device IP', 'Subnet Mask', 'Gate Way', the keypad will show up. Then, input the information required and press 'OK' button to save or 'ESC' to cancel the information.


The server interlocking device can be selected as BMS Server of Bionet.

After selecting the type of interlocking server, select the 'Setting' button; an appropriate setup window to the choice of server will be displayed.

CardioTouch3000 Operation Manual

BMS Server Setting

IP setup on BMS Server-installed PC can be performed by choosing either Automatic IP (DHCP) or Fixed IP (Manually) in 'Configure' setup.

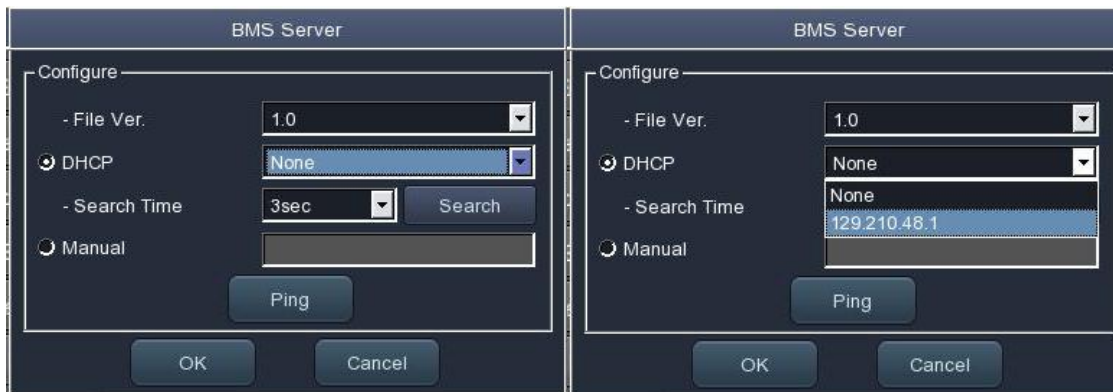


If 'DHCP' is chosen, the IP of the BMS Server-installed PC will be shown on the input box.

Choose the IP of the PC you intend to connect as the IP of a BMS Server-installed PC; At that time, IP will be shown on the input box only if BMS Server is running on the PC.

If no IP of the intended BMS Server to connect is shown on the input box, choose 'Search' button to conduct an additional search.

If no IP of the intended BMS Server to connect is shown on the input box, choose 'Manual' mode to perform manual input.

Upon completion of BMS Server IP setup, the Ping Test can be conducted to check successful network connection. If choosing 'Ping' button, "Ping : 100% Succeeded" message will be shown for a successful connection, whereas "Ping : Failed" message will be shown for connection failure.

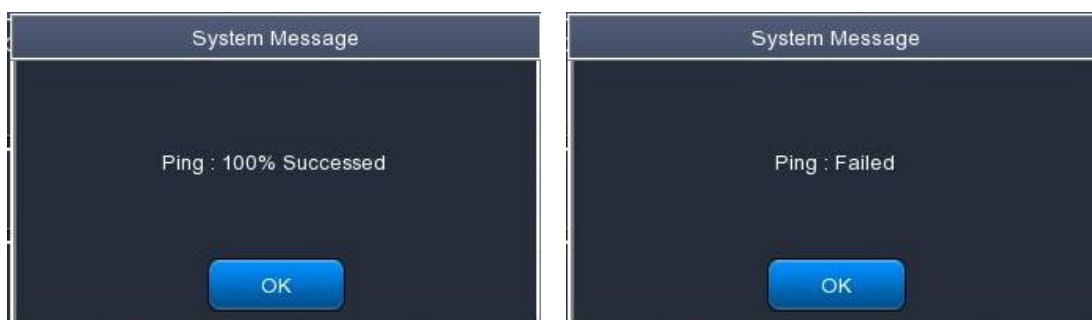


NOTE

Wired usage is recommended for a BMS Server-installed PC network.
When setting BMS Server, directly enter the IP of the BMS Server-installed PC by selecting "Manually" if searching with DHCP has failed.

CardioTouch3000 Operation Manual

Wireless Network Setting




Connect the USB wireless LAN card to the USB port of CardioTouch3000, and check whether or not the wireless icon () appears in the Status bar on the top of the screen.

- 1) Navigate to System Setup -> Network, and Set the Device as "Wireless".
When selection is done properly, the "Survey" button on the bottom will be activated.
- 2) Select the "Survey" button to search for the AP.
The AP is to be searched automatically when the "Search" button is pressed; the searched AP will appear in the list upon the completion of the search.

Specifications of Wireless Network USB Adapter Support

Company Name	Product Name
ipTIME	N100mini
	N150UA
	N150U
TP-LINK	TL-WN725N

CardioTouch3000 Operation Manual

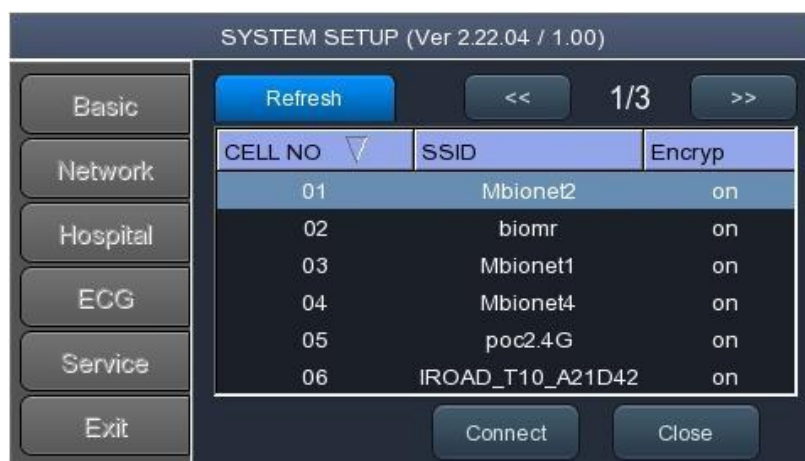

- **Refresh** : AP re-searching
- **<<** : Page up
- **>>** : Page down
- **Connect** : Attempting to connect
- **Close** : Closing the window

- 3) Select the AP to be connected by using the rotary key or touching. Click the "Connect" button. At that time, the security key input window will appear if the "Encrypt" field is "On".




NOTE

The user should set the AP Search and Security Key in order to use the wireless network. If there is no security set on the AP to be connected, it will be connected automatically without Security Key setting. In the event that there is an automatic connection to the AP without Security Key setting, the safety of network communication is not guaranteed.


CardioTouch3000 Operation Manual

- 4) After clicking the Key Value input line by using the rotary key or touching, enter the security key.
- 5) Upon the completion of input, click the "OK" button; Connection will be attempted and the message describing the connection status will appear.

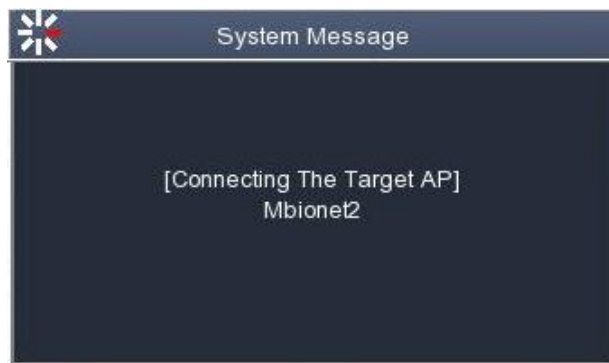

- 6) When the message reporting a proper connection appears, click the "Close" button.



When connection is successful, the wireless icon on the status bar at the bottom of the screen will be changed into .

- 7) If connection fails, connection can be re-attempted by pressing the "Retry" button. If connection keeps failing, check the AP and the USB-TO-WLAN status.



CardioTouch3000 Operation Manual

- 8) When connection with the AP is successful, set the Device IP, Subnet Mask, Gateway, etc. by selecting manually or Configuring with DHCP.
Refer to the Network Setup part of the User Manual for more details.

NOTE

It is recommended to use English for the name of the AP wireless network (SSID). If it is entered in Korean or other languages, Mojibake or illegible characters can appear.

Hospital Setup

When selecting the 'Hospital' menu at the left side of the Setup page, Hospital Setup page will show up at the right side. You can input the names of the hospital and doctor.

Moreover, for classifying with the other devices in hospital, it is offer to make " Device Hospital Setup menu, user can set up hospital name and doctors name. But "Device ID" is innate ID and user can not adjust. Automatically, " Device ID" is generated.

When clicking the area to input the information, the keypad will show up. Then you can input the information required and press OK button to save or ESC to cancel the information.

SYSTEM SETUP (Ver 1.00.02 / 1.01)	
Basic	- Hospital: Bionet
Network	- Doctor: Dr.kim
Hospital	- DeviceName: CardioTouch
ECG	- Device ID: NP0032a2
Service	
Exit	
	New OK Cancel

CardioTouch3000 Operation Manual

ECG Setup

When selecting the 'ECG' menu at the left side of the Setup page, ECG Setup page will show up at the right side. In ECG Setup menu, there is 'AUTO' button to set up and change set up related automatic diagnosis.



CardioTouch3000 Operation Manual

Printing channel setting [Print Setup]

[Monitoring Form]

Set up real time-printout form. Click 'Mon Form,' and then a POP-UP window for channel list will be displayed. Default is set as '12CH'.



- 3CH printing: To print 3 channels at the same time (I ~ III / aVR ~ aVF / V1 ~ V3 / V4 ~ V6) If you want to change the channel, press 'RHYTHM' on the control panel.
- 6CH printing: To print 6 channels at the same time (I ~ aVF / V1 ~ V6) If you want to change the channel, press 'RHYTHM' on the control panel.
- 12CH printing: To print 12 channels at the same time (I ~ V6)

NOTE

At 3CH and 6CH setup, 'RHYTHM' key will become active only if 'Mon Size' is set as 'Continue.'

CardioTouch3000 Operation Manual

[Monitoring Size]



You can set up the real time printing size. When you click the 'Mon Size' menu, 'Report' will be selected and when clicking it again, 'Continue' will be selected. Default is set as 'Continue'.

- 'Report' : To print out 10-second data in the form of Report.
- 'Continue': The leads will be printed out continuously until you press the 'ESC' key on the control panel.

NOTE

Output speed will be 25.0mm/sec if 'Mon Size' is set as 'Report.'

CardioTouch3000 Operation Manual

[Grid]

If general fax paper is used for printout, it needs to be marked with a grid. A 5mm X 5mm grid must be in a solid line with the point mark on the 1mm X 1mm location. If using standard ECG recording paper instead of general fax paper(thermal paper), undo the grid marking setup as the recording paper will already have it.

The grid marking setup can be done using output format selection mode. Press rotary key or touch the intended field to select among the Grid selection items, and then the setup will be modified as ON or OFF. If ON, the grid marks will be printed out; if OFF, it will not be printed out.



[Recording Form]

You can set up the recording form.



If you choose the 'Rec Form' menu, a small POP-UP menu will appear below; once you choose either 3CH+1, 6CH+1, 12CH or 3CH+3, the POP-UP menu will disappear. The default is '3CH+3'.

CardioTouch3000 Operation Manual

- '3CH+1': 10-second ECG will be recorded as I, II and III for the first 2.5 seconds, aVR, aVL and aVF for the next 2.5 seconds, V1, V2, V3 for the next 2.5 seconds and V4, V5, V6 for the next 2.5 seconds. 1 channel of rhythm will be recorded for 10 seconds at the bottom.
- '6CH+1': 10-second ECG will be recorded as I, II, III, aVR, aVL and aVF for the first 5 seconds and V1, V2, V3, V4, V5 and V6 for the next 5 seconds. 1 channel of rhythm will be recorded for 10 seconds at the bottom.
- '12CH': 12-channel rhythm will be recorded as I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5 and V6 at the same time for 10 seconds.
- '3CH+3': 10-second ECG will be recorded as I, II and III for the first 2.5 seconds, aVR, aVL and aVF for the next 2.5 seconds, V1, V2, V3 for the next 2.5 seconds and V4, V5, V6 for the next 2.5 seconds. 3 channels of rhythm will be recorded for 10 seconds at the bottom.

The set value will be displayed on the screen, and when you choose '3CH+1', 3 Channels + 1 Rhythm Report will be shown at the top of the printing form, '6CH+1' for 6 Channels + 1 Rhythm Report, '12CH' for 12 Channels Rhythm Report, '3CH+3' for 3 Channels + 3 Rhythm Report will be shown.

[Paper Size]

You can set up the paper size. When you click the 'Paper Size' menu, 'A4' will be selected and when clicking it again, 'Letter' will be selected. Default is set as 'A4'.


- 'A4' : to output at 297mm for A4 Size printout.
- 'Letter' : to output at 279 mm for Letter Size printout.



CardioTouch3000 Operation Manual

[Print Line]

You can set the line thickness of the printouts. Three setting options are available; "Thin", "Normal" and "Thick".

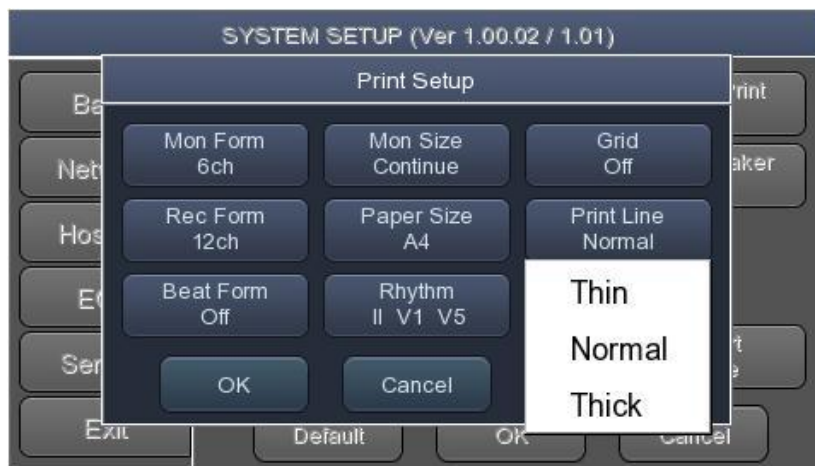

[Beat Form]

You can set up the additional beat report printing.



When you select the 'Beat Form' menu, a small POP-UP menu will appear at the bottom and it will disappear when choosing OFF, Text, Vector or Guide.

Default is set as 'OFF'.

Beat report will be attached to the printing form of channels of '3CH +1', '6CH +1', '12CH' and '3CH +3'. It will record every important result by marking the representative waveform of each form.

CardioTouch3000 Operation Manual

[Rhythm]

You can set up the rhythm leads for printing out the diagnosis of 3CH+1, 6CH+1, 3CH+3 and Long Time(1,3,5,10 Minute). You can choose 3 rhythm leads and when '3CH+3' is chosen, you can use all of the 3 leads, while when '3CH+1', '6CH+1' or 'Long Time' is chosen, you can use the first value only.

Default is set as 'II, V1, V5', so when 3CH+1, 6CH+1 is chosen, you can use the lead 'II' and 3CH+3 for lead 'II, V1, V5'.

If you choose 'Rhythm' menu, 'Rhythm Channel' window will be displayed as shown below.



At that time, select either the 'CH1,' 'CH2' or 'CH3' menu; and then choose one from Lead I ~ V6 in the select menu.

CardioTouch3000 Operation Manual

[Quick Print]

Set up the Quick Print Options. Output time can be saved by using the buffering function during selection of 0 - 10 seconds.



NOTE

If setting up as ON, diagnosis button should be used only after signalling is stabilized. Upon ON setup, diagnosis and output can be affected if the saved data is not stabilized; this is because the data is retained before use for as long as the time set up.

CardioTouch3000 Operation Manual

[Filter Setup]

The signal output generated by the device may contain extra unwanted signals in addition to the EKG wave such as AC noise, base line drift caused by respiration, noise generated by an electrical field nearby, etc. Applying filters could help to reduce or remove those unwanted signals from the output of the device.

To change filter information, enter the Filter Setup Menu either by moving the selection block over and select it using the rotary key or by touching the menu on the touch screen.



Upon selection, the Filter Setup menu, shown below, will appear. Use either the rotary key or touch screen to navigate and select menu items to edit. After editing, OK must be selected in order to apply the changed value.

CardioTouch3000 Operation Manual

Base line drift is caused by the patient's breathing. It can be seen as ECG signals being overlapped on a large parabola. The BASE LINE filter provides 0.05Hz, 0.1Hz and 0.2Hz, and its set values will appear as Base:0.05Hz, Base:0.1Hz and Base:0.2Hz at the bottom of the Output Form.



There are three types of an AC power noise filter; 60Hz, 50Hz and OFF. When set to OFF, the filter will not be applied. If the filter is set to 50Hz or 60Hz, then the AC filter will remove 50Hz or 60Hz power noise correspondingly. In the U.S., the AC filter should be set to 60Hz. Check the properties of your power source before setting AC filter properties.

The status of the AC filter can be checked on the bottom part of the printout - indicating AC 60Hz, AC 50Hz or AC OFF



CardioTouch3000 Operation Manual

If the recorded EKG signal contains a lot of noise, Low Pass Filter (LPF) may help to clear it. For Low Pass Filter, 40Hz, 100Hz, 150Hz and OFF option is offered. Here, 40Hz option means the filter will remove any unwanted signal with frequencies above 40Hz. Low Pass Filter values will appear on the bottom of the printout form; if LPF setting is OFF then as LPF: Off; if the LPF setting is 40Hz then as LPF: 40Hz; if LPF setting is 100Hz then as LPF: 100Hz; if LPF setting is 150Hz, then as LPF: 150Hz.



Select 'ON' when applying 'EMG' filter; otherwise, select 'OFF.' Setup values will be shown on the bottom of the printout form as 'EMG:On' or 'EMG:Off' when applying the filter.




CardioTouch3000 Operation Manual

Lead fault Signal Display Setting

When the connection between the patient and Leads is improper, lead fault can occur. In that case, you can choose whether to have the lead fault message on the ECG Main page or not. Selecting 'Lead Fault' menu will switch between ON and OFF in the menu, with ON turning on the message display and OFF switching off the message display.



The following message for Lead Fault will be shown with affected channels:

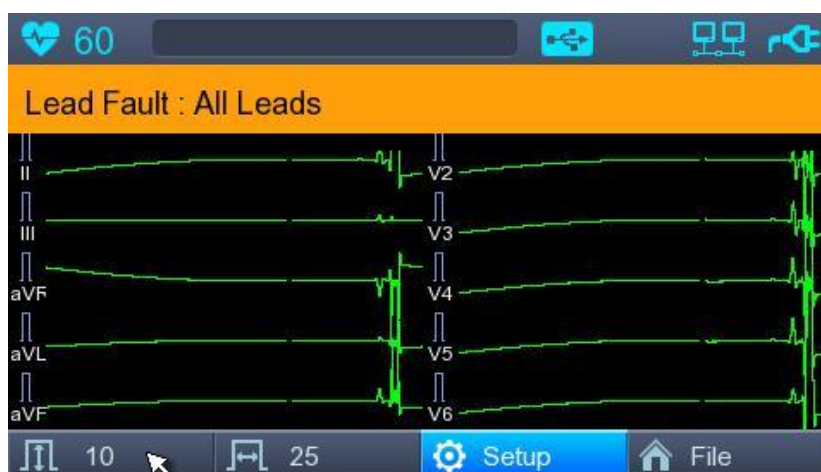

NOTE

If 'RA' Lead becomes Fault, the wave types of all Leads will not be indicated.

If 'LA' Lead becomes Fault, the wave types of I and V1-V6 Leads will not be indicated.

If 'RA' Lead becomes Fault, Lead Fault message might not be shown and the wave types of all Leads might be indicated.

If 'LL' Lead becomes Fault, the wave type of II and V1-V6 Leads will not be indicated.

If a Lead Fault occurs during monitoring or recording, a buzzer alarm will sound and the message box will pop up.

No pacemaker signal may be detected. Additionally, if a Lead Fault occurs, please re-measure the EKG as it could have affected the measurement by the device.

Pacemaker signal printing

In case of using the pacemaker signals, you can choose whether to print out the location of the pacemaker when monitoring or recording.

You can select ON or OFF by clicking the 'Pacemaker' item. When it is on, the location of the pacemaker will be printed out, while when it is off, it will not be printed out.



NOTE

It is recommended to set the pacemaker off for normal use and only when the patient uses a pacemaker, set it on.

When having a Lead Fault, the Pacemaker signals may not be detected.

CardioTouch3000 Operation Manual

QRS Sound Setup

In this menu, users can set the device to ring an alarm sound when a QRS beat has generated while it is in waiting mode. Users can either set the alarm ON or OFF by selecting the menu with ON turning on and OFF turning off the alarm sound.




NOTE

No sound will ring while printing even when the QRS sound is set to ON.

CardioTouch3000 Operation Manual

Demo Mode Setting

This menu puts CardioTouch3000 in demo mode. When it is set to ON, 60bpm Sinus Normal Rhythm signal will be displayed on the device and DEMO sign will appear on the upper part of the LCD screen. Users can test rhythm, diagnostic, copy printing functions as well as PC communication and other miscellaneous functions.

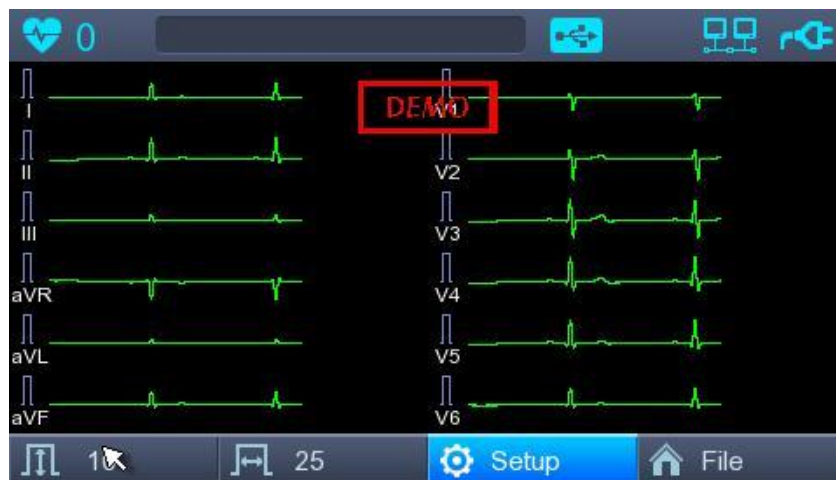

CardioTouch3000 Operation Manual

'AUTO' key setting

When the 'Store' menu has the focus, click it to set up 'Yes' or 'No'.

When 'Yes' is selected, you can operate the ECG test by pressing the 'AUTO' key on the control panel and the ECG data will be automatically saved, but when 'No' is selected, the ECG data will not be saved. Default is set as 'Yes'.



When the 'Print Out' menu has the focus, click it to set up 'Yes' or 'No'.

When 'Yes' is selected, you can operate the ECG test by pressing the 'AUTO' key on the control panel and the ECG data will be automatically print out, but when 'No' is selected, the ECG data will not be printed out. Default is set as 'Yes'.




CardioTouch3000 Operation Manual

When selecting the 'Export' menu, a small menu window as below will appear and it will disappear when selecting one of None, Server, USB.

If it is 'None,' ECG data will not be externally transferred; if is 'USB,' ECG data will be automatically saved to the external USB memory. In addition, if it is set up as 'Server,' data will be automatically transferred to PC Server IP among Network options. (Default : None)



NOTE

When you remove the external device after sending to USB Memory, you should click icon  that is correct to each device on top of the main screen and remove the device. Otherwise data can be damaged.

CardioTouch3000 Operation Manual

Service Setup

If 'Service' menu on the left of the Setup screen is selected, the other Setup screen related to 'User Security Set' and 'Manufacture Set' will appear.


At 'User Security Set' setup, it is possible to change Factory Setting and User Password.

At 'Manufacture Set' setup, it is possible to change Upgrade and Device Options; please contact the Bionet Service Center.



Factory (Reset Settings)

In order to reset to factory conditions, press the 'Factory' button and enter the Password. Reset will proceed, with the message below appearing on the LCD screen for 1 second.

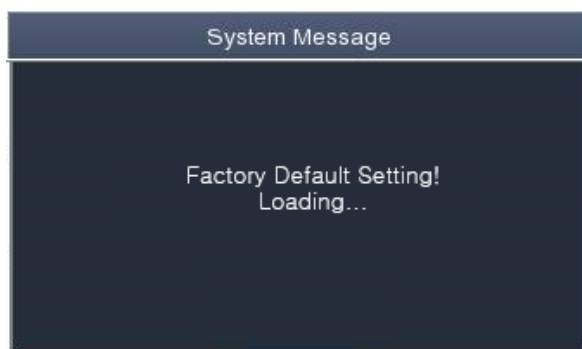

CardioTouch3000 Operation Manual

The initialized values are as follows.

Contents	Setting Value
Gain	10mm/mV
Speed	25mm/s
Start Option	Main
Unit	Height : cm, Weight : kg
Date Type	YYYY-MM-DD
Patient Info (File)	Delete
Device Name	CardioTouch3000
Net Device	Wired
Configure	Manually
Device IP	192.168.30.224
Subnet Mask	255.255.255.0
Gate way	192.168.30.1
BMS Server	0.0.0.0
Hospital	blank
Doctor	blank
Quick Print	Off
Lead Fault	On
Pacemaker	Off
QRS sound	Off
Demo	Off
Mon Form	12ch
Mon Size	Continue
Grid	Off
Rec Form	3ch + 3
Paper Size	A4
Print Line	Normal
Beat Form	Off
Rhythm	II, V1, V5
Base Filter	On
AC Filter	For ENG and KOR language, '60Hz' For other languages, '50Hz'

CardioTouch3000 Operation Manual

Low Pass Filter	150Hz
EMG Filter	Off
ECG Store	Yes
Print Out	Yes
Export	None
Target	PC
Method	Manual
Format	EKG
Delete	Yes
JPEG Size	1024 x 768
Patient information	initialized

CardioTouch3000 Operation Manual


Data Erase

All data in Files will be deleted.



Change PW

User Password can be configured. User Password is the number entered for Factory and Data Erase. The password should be a 4-digit number.

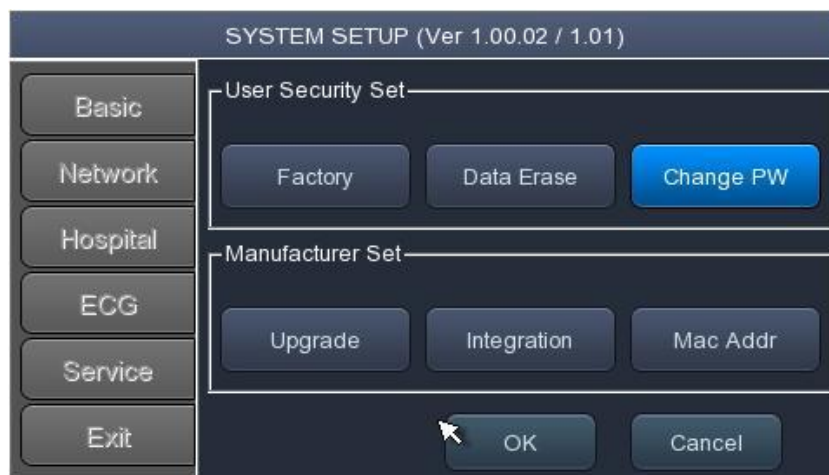

NOTE

The default User Password is '1234'. If you forget the password, enter '1234' and reset.

CardioTouch3000 Operation Manual

CAUTIONS

Do not touch patient cables or equipment when using the ventricular defibrillator.

CAUTIONS

When connecting electrodes, patient cables, etc., ensure that the connector does not touch conductive areas or ground. In particular, be careful not to touch the conductive parts or ground when you attach each electrode to the patient's body.

CAUTIONS

The supplied ECG patient cable should be used only for ECG measurements and not use for breathing.

CAUTIONS

Use electrodes with products such as those provided or with a biocompatibility certificate certified by international standards.

CAUTIONS


Use CardioTouch3000 in the presence of a specialist(health care professional) for use by patients undergoing cardiac assist device surgery.

Chapter 3. ECG File Management

- 1) Display and Function
- 2) Data Printing
- 3) Data Deleting
- 4) System Setup
- 5) Switching Menu
- 6) Patient Information
- 7) Data Search
- 8) Data Transfer
- 9) Data Import

CardioTouch3000 Operation Manual

1) Display and Function

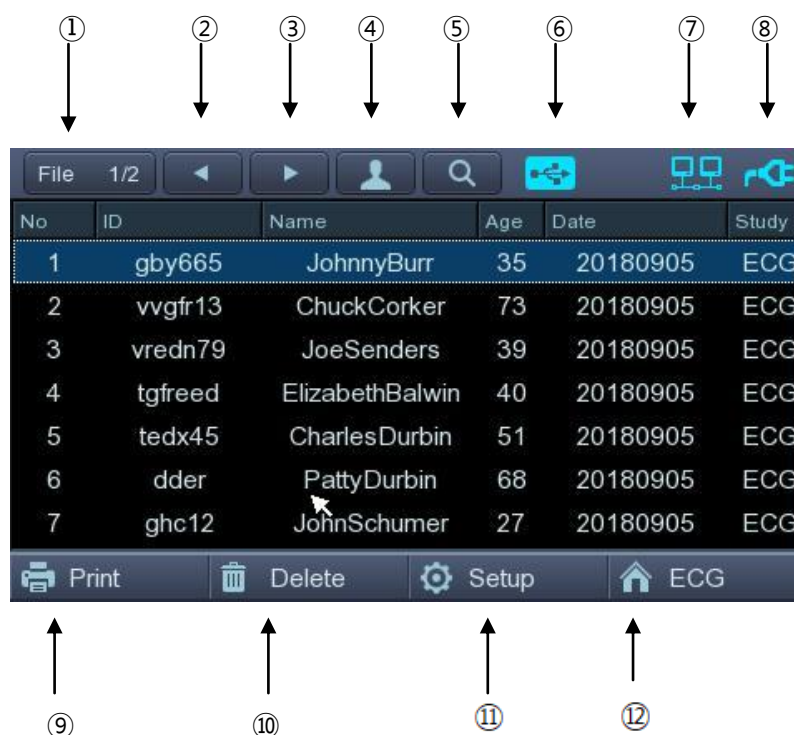

- ① Page information
 - For example, [1/20] means that you see the first page among total 20 pages. 1 Page contains 24 lines of Data.
 - If Rotary is selected, the focus will be moved to Data list.
- ② Button to skip to the previous page
- ③ Button to skip to the next page
- ④ Button to check the patient's information of the chosen data saved
- ⑤ Button to search the data you want
- ⑥ State of USB connection
- ⑦ State of Network connection
- ⑧ State of battery or AC power connection
- ⑨ Button to print out the chosen saved data
- ⑩ Button to delete the data
- ⑪ Button to set up the environment of 'File Management'
- ⑫ Button to move to other main screen from 'File Management'

At the initial file management page, the first data of the list has the focus. The focus will automatically move to the menu bar at the bottom of the screen, when you click the data by touching the screen or rotating the rotary key on the control panel.

If there is no data in the page, the focus can be found on [Print] button.

When there is no patient information you want to select, please press [<] or [>] button or the 'Search' menu to move to other pages.

CardioTouch3000 Operation Manual

- Menu direction (Rotary Key's right spin)

Print → Delete → Setup → Main → Page information → [<] → [>] → Info → Search → Print


- Menu direction (Rotary Key's left spin)

Print → Search → Info → [>] → [<] → Page information → Main → Setup → Delete → Print

2) Data Printing

When selecting a saved data at the ECG file management page by using rotary key or touch screen, the focus will appear at the data list.

Press the 'Print' button to print out the data. Then you will see a message as shown below.

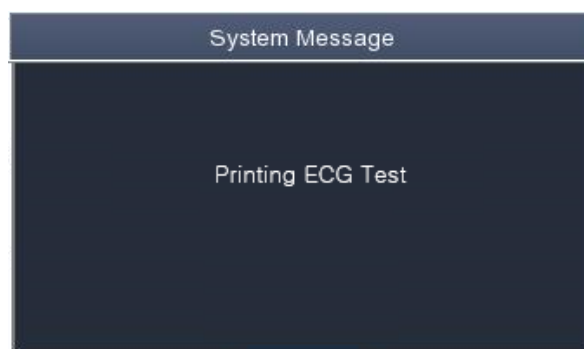

NOTE


You should avoid attaching and detaching a USB device while printing out, as it could cause the printer module to rattle.

3) Data Deleting

When pressing the 'Delete' button, a small pop-up menu will show up. You can choose to press the 'select' or 'all' button.

When you press the 'Select' button, one data will be deleted, but if you press the 'All' button, all data in the list will be deleted.

In order to help the user not to make a mistake when deleting the data, a message shown as below will appear to reaffirm whether you really want to delete the data.

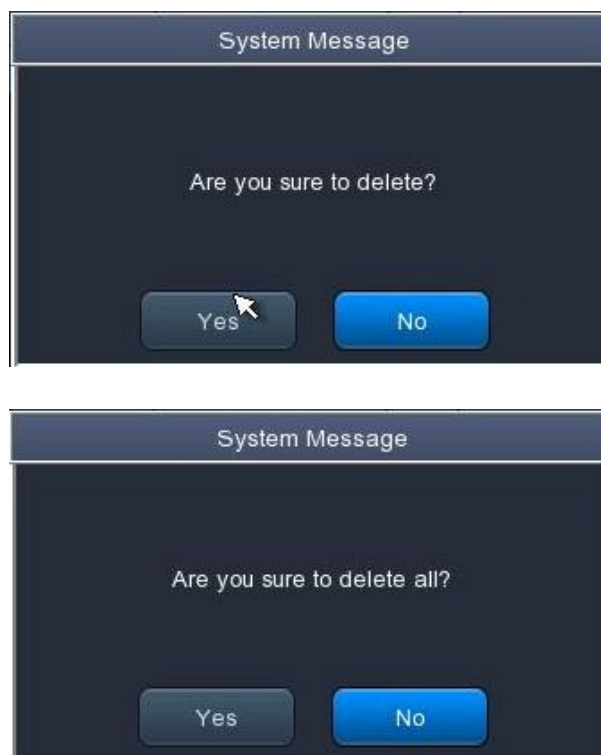

CAUTIONS

Please think carefully before selecting the 'All' button, as it will permanently delete all of the ECG data.

4) System Setup

When clicking the 'Setup' button at the ECG data file management page, you will see a 'System Setup' window as shown below.



'PAT Info.' menu is to set up whether the patient information is retained or not, when you move the main screen of the ECG. When you click the menu, 'Retain' will be set up and when clicking the menu again, 'Delete' will be set up. Default is set as 'Delete'.


- 'Delete': To set up the patient information is deleted, when you move to main screen of the ECG.
- 'Retain': To set up the patient information is retained, when you move to main screen of the ECG. This may result in more than one record for the same patient.

NOTE

The steps for Network, Hospital and Service Setting are identical with the steps for ECG Main setting.

CardioTouch3000 Operation Manual

When clicking the 'Export' button, you will see 'Export' window as shown below.

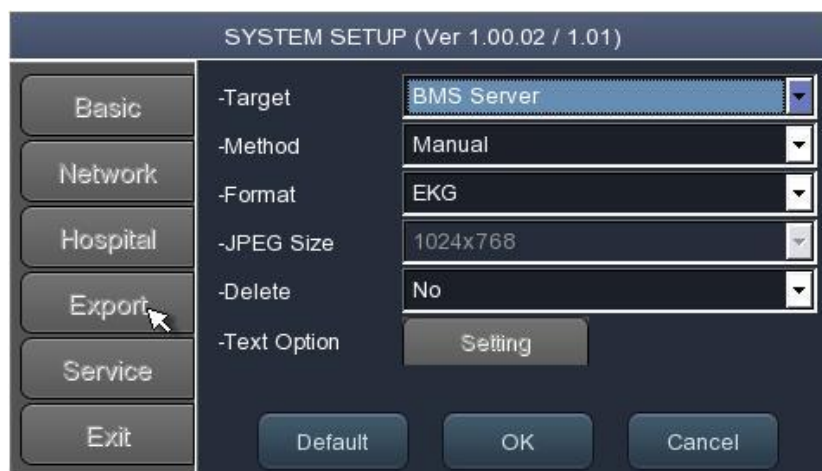

When clicking the 'Target' button, a POP-UP window will show up and it will disappear when selecting one of BMS Server or USB.

- BMS Server : To save the data in the BMS Server or USB memory.

When clicking the 'Method' button, a small line window will show up and it will disappear when selecting Manual, Select, All or All New.

- Manual : menu window allows user to select either 'Selected, All, All New, Cancel' or 'Selected, Cancel' whenever transferring data
- Select : To transfer the selected data
- All : To transfer all data
- All New : To transfer the newly saved data

When clicking the 'Format' button, a line window will show up and it will disappear when selecting EKG, MFER(ECG), XML(ECG), JPEG, PDF. 'Format' menu is used to select the user's options when transferring the data.

- EKG: To save the data by type of ECG Raw Data(*EKG)
- MFER: To save the data by type of '***.mwf'
- XML: To save the data by type of '***.xml'
- JPEG: To save the data by type of '***.jpg'
- PDF: To save the data by type of '***.pdf'

When clicking the 'Delete' button, a line window will show up and it will disappear when selecting Yes or No. 'Delete' menu is used to select the user's options after the data has transferred.

- Yes: To delete the data after it has transferred
- No: To not delete the data after it has transferred

NOTE

If 'Format' menu has only 'EKG' options, it is possible to select 'All' or 'All New' in 'Method' menu; for other formats, 'Selected' is the only option.

5) Switching Menu

When selecting the 'Main' button at the main page of 'File management', a line window will show up. You can choose ECG to go to the chosen page.


But it depends on the setting value of 'PAT Info.' menu when skipping to 'ECG' main screen.

- ECG : Skip to ECG main screen

6) Patient Information

When selecting a saved data at the ECG file management page by using a rotary key or a touch screen, the focus will appear at the data list.

Press the 'Info' button. Then you will see a 'Patient Information' window as shown below. Information other than patient name cannot be changed.

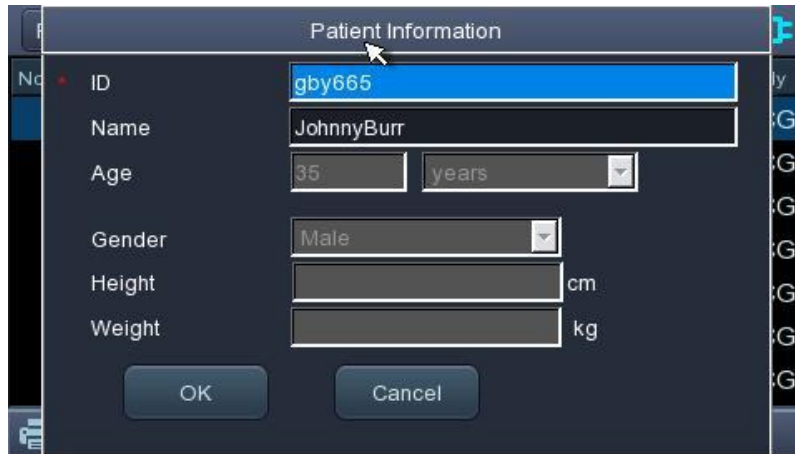

The screenshot shows a 'Patient Information' dialog box. The ID field is highlighted in blue and contains the text 'gby665'. The Name field contains 'JohnnyBurr'. The Age field has '35' and a dropdown menu set to 'years'. The Gender field has a dropdown menu set to 'Male'. The Height field is empty with 'cm' as a unit. The Weight field is empty with 'kg' as a unit. At the bottom are 'OK' and 'Cancel' buttons.

7) Data Search

When clicking the 'Search' button at the ECG data file management page, you will see a 'Patient Search' window as shown below. If you rotate the rotary key, the 'search condition' button will have focus. Press the 'Find' button after inputting the information about ID, Name, Date or Age.

When the search takes a long time, a message will show up to let you know the progress. After searching, you will see a data search result page.



The screenshot shows a 'Patient Search' dialog box. It has a 'Search Condition' section with several input fields. A dropdown menu is open over the ID field, showing options: ID, Name, Day, and Age. The ID field is currently selected. At the bottom are 'Find', 'Close', and 'All' buttons.


8) Data Transfer

You can transfer the saved data to the external device at the ECG data file management page by using the 'NETWORK' key on the control panel.


When pressing the 'NETWORK' key, a message will show up, asking which menu you want to select.



If you transfer a selected data, a message with progress speed or other information will be shown as below.

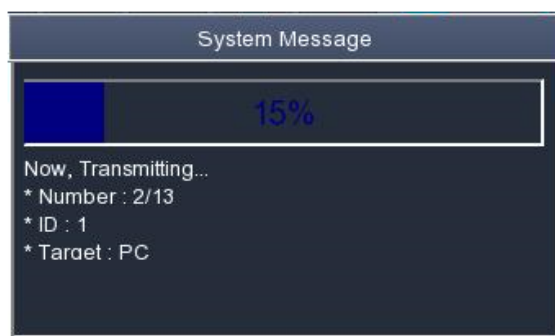

If you transfer all of the data, a message with progress speed or other information will be shown as below. In this case, the total number of files and the file being transferred currently will be displayed.

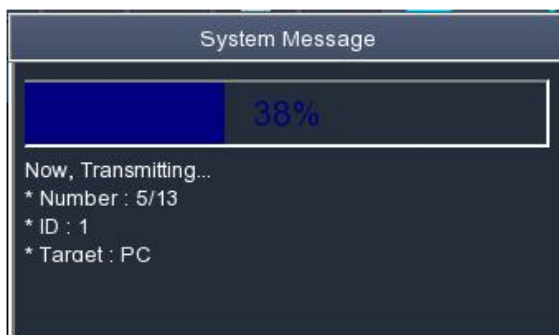

CardioTouch3000 Operation Manual

NOTE

When transmitting multiple sets of data to PC after saving, a network error could occur which may cause transmission halt and subsequent loss of data. Therefore, we recommend users to practice following procedures to minimize such loss.

1. In EKG Main, press [NETWORK] button to transmit data to PC immediately after measuring..
2. When transmitting multiple sets of measurements, rather than sending all data at once, we recommend users to transmit individual records after each measurement.

NOTE

If the system boots up without having LAN cable connected, the network function may not work properly. In this case, enter [System] -> [Network] -> [OK] then press 'NETWORK' button to attempt running the network function again. Please refer Network Setup part of the manual to get more information regarding IP configuration setup.

9) Data Import


You can import the saved data from the external device at the ECG data file management page by using the 'RECORD' key on the control panel.

When pressing the 'RECORD' key, 'Import File List' screen will be shown as below.




No	Name	Type
1	10min_20180502140104.FVC	FVC
2	1_20180502140835.FVC	FVC
3	1_20180514165000.ekg	ekg
4	2_20180502142300.FVC	FVC
5	3ksj_20180502142819.FVC	FVC
6	4_20180514163540.ekg	ekg

If the external device is not connected to CardioTouch3000, the error message will be shown as below.

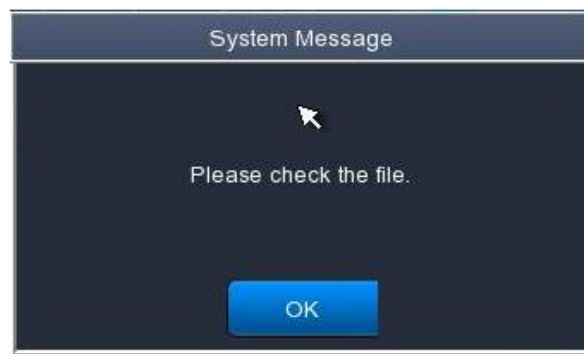


If you select the 'Selected' menu in the 'Import File List' screen, after receiving the selected data from the external device a system message will be shown as below.

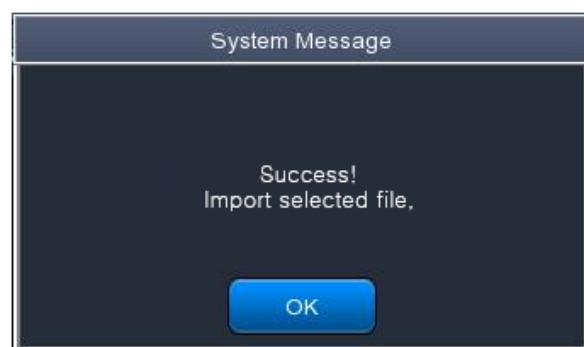

CardioTouch3000 Operation Manual

If you select the 'All' menu in the 'Import File List' screen, a system message with progress speed will be shown as below.

After receiving all data from the external device, a system message will be shown as below.

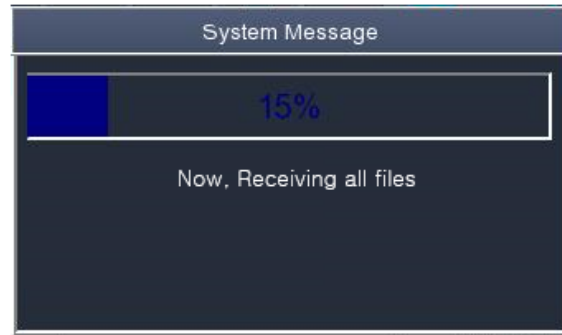

Chapter 4. System Management

- 1) Maintenance and Cleaning
- 2) Regular Check-up
- 3) Trouble Shooting
- 4) Manufacturer Declaration

1) Maintenance and Cleaning

There are many ways to clean CardioTouch3000, but it is best to use our recommendation to avoid damage or sanitary issues.

The warranty does not cover problems resulting from the use of harmful substances (unauthorized substances).

CAUTIONS


After cleaning the product, check the hardware and the electrodes.
Do not use the product if it is damaged or worn.

Rub the product with gauze damp with alcohol about once a month to clean the product as well as the electrodes. Do not use lacquer, thinner, ethylene or oxide.

Make sure there is no dust or dirt on the cables and limb and chest electrodes and clean them with wet gauze (40°C/104°F) after use and clean it with clinical alcohol once a week.

Do not submerge the product or the ECG cable into any liquid or detergent. Also make sure there is no liquid in the product or the cable.

Disposal of your old appliance

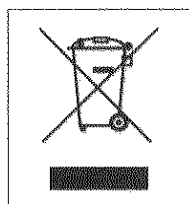

1. When this crossed out wheeled bin symbol is attached to a product it means the product is covered by the European Directive 2002/96/EC.
2. All electrical and electronic products should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.
3. The correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health.
4. For more detailed information about disposal of your old appliance, please contact your city office, waste disposal service or the shop where you purchased the product.

2) Regular Check-up

Like every other medical product, CardioTouch3000 requires a regular check-up once a year. Please refer to the Service manual for the information on the check-up.

3) Trouble Shooting

- **Printing was not successful:**

The cover may not be closed properly. Please close the cover and print again.

- **When using the battery power, there was a beep for more than 3 times and a message came as shown below:**



The battery is low. Please connect to the AC power to use.

- **The signal is printed out with noise:**

First, check that you set up the AC power filter. If it still makes noise with the AC power supply, please connect grounding line to the grounding electrode of the system.

The grounding should not be used with the AC power's grounding and please connect it to the metal around the patient's bed or the building.

- **When The external USB keyboard is not working properly:**

Connect the external USB keyboard to the USB port before you turn on the power of CardioTouch3000. If you connect the external USB keyboard to the USB port after you turn on the power of CardioTouch3000, you should turn off the power of CardioTouch3000 and turn on again.

If it is still not working, please contact our customer service center.

- **System messages**

The following system messages may occur while you are operating this system. You may be required to perform some action. If you perform the recommended actions and the condition still remains, contact authorized service personnel.

CardioTouch3000 Operation Manual

No.	System message	Cause	Solution
1	Please check date and time	Internal coin battery is discharged.	Please contact CS (Customer Support) team of Bionet to replace old coin type battery with new one
2	Address is not valid	When wrong network address(IP, SM, GW) is entered	Check the right network address before input
3	Invalid value. [Use: 0~9, A~F]	When wrong character is entered at time of inputting Mac address	Please input valid character such as 0~9, A to F in number and alphabet respectively
4	There is no patient ID. Please enter patient ID.	When you implement AUTO or NET button without patient ID	Please enter patient ID in patient information window to implement
5	Wrong password. Please re-enter your password.	When wrong password is entered	Check and input right password
6	Enter password length error. Please re-enter your password	When wrong number of characters is entered as password	Please check and input valid and correct password
7	The internal memory is full. Please erase other files or transfer to external storage.	When over 200 files are stored at internal memory	Erase files in File menu
8	Not enough free disk space	USB memory device has insufficient space to receive files.	Remove unnecessary files from USB memory (Minimum 500KB space is needed to export 1 file)
9	Please check the USB memory.	When USB memory stick is not inserted, or is out of order	Please check and insert USB memory stick in right position. Note that USB memory file format supports FAT, FAT32 only (this can be checked on PC)
10	There is no Access Point. Please check the AP	There is no wireless access point detected.	Check AP and retry
11	Fail, connecting. Do you want to retry?	When wireless AP connection is failed	- Press Retry button - Retry after checking AP

CardioTouch3000 Operation Manual

12	Ping : Failed	When there is connecting failure between network and PC with BMS Server	- Check System setup and network - Check PC network setup -Check LAN cable or wireless environment
13	IP address conflict.	The devices IP address is already in use on the network	Please contact 'Network Team' to check IP address, and set right IP in turn of System setup and Network
14	Connection Error. Check lan cable, network setting and relevant program.	When there is failure of network connection	Please check LAN cable connection, network setting, server PC's networking and related S/W programs
15	File Trans Fail!!	When file transmission to server PC or USB memory fails	Check server PC's networking conditions, and USB memory, and retry
16	Recharge Battery!	When the battery level is Low	Please connect power cable or charge battery
17	Printer: Time out.	Printer stops working due to technical problem of device	React the printer
18	Printer: Busy continue	When the wrong printer data were input	React the printer
19	Printer: Temperature high	When the Printer gets heated due to long and continuous use of printer	Please stop the use of printer for at least one minute and try again
20	Printer: Paper empty	When the printing paper is completely out	Please put new roll of printing paper for use
21	Printer: Printer not ready	When there is no 'ready' answer from printer module	React the printer
22	Printer: Unknown data	When the wrong protocol was put into the printer	React the printer
23	Abnormal End!	When the printer was abnormally forced to stop during printing	Re-start printing

CardioTouch3000 Operation Manual

24	Lead Fault!	When the lead is off from the patient body	Please reattach the electrode on patient body
		When the conductivity between patient body and electrode is weak	Apply some water to reusable electrodes, and reattach them
25	No ECG Recorded Data!	When the user tries to execute copy printing without any recording data	Try make copy after recording data by "RECORD", "AUTO", and "NET" button

CardioTouch3000 Operation Manual

4) Manufacturer Declaration

Electromagnetic Compatibility Information

Phenomenon	Basic EMC standard or test method	Test level/requirement
Mains terminal disturbance voltage	CISPR 11	Group1, Class A
Radiated disturbance	CISPR 11	Group1, Class A
Harmonic Current Emission	IEC 61000-3-2	Class A
Voltage change, Voltage fluctuations and Flicker Emission	IEC 61000-3-3	Pst: 1 Plt: 0.65 Tmax:0.5 dmax: 4% dc: 3.3%
Electrostatic Discharge Immunity	IEC 61000-4-2	± 8 kV/Contact $\pm 2, \pm 4, \pm 8, \pm 15$ kV/Air
Radiated RF Electromagnetic Field Immunity	IEC 61000-4-3	3 V/m 80 MHz - 2.7 GHz 80% AM at 1 kHz
Immunity to Proximity Fields from RF wireless Communications Equipment	IEC 61000-4-3	Table 9 in IEC 60601-1-2: 2014
Electrical Fast Transient/Burst Immunity	IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency ± 1 kV, 100 kHz repetition frequency
Surge Immunity	IEC 61000-4-5	Line to Line ± 0.5 kV, ± 1 kV Line to Ground ± 0.5 kV, ± 1 kV, ± 2 kV
Immunity to Conducted Disturbances Induced by RF fields	IEC 61000-4-6	3 V 0.15 MHz - 80 MHz 6 V in ISM bands Between 0.15 MHz and 80 MHz 80% AM at 1 kHz
Power Frequency Magnetic Field Immunity	IEC 61000-4-8	30 A/m 50 Hz and 60 Hz
Voltage dips	IEC 61000-4-11	0 % U_T : 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
		0 % U_T ; 1 cycle and 70 % U_T ; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0 % U_T ; 250/300 cycle

CardioTouch3000 Operation Manual

Electromagnetic compatibility - Guidance and manufacturer's declaration

Guidance and manufacturer's declaration – electromagnetic emissions		
The CardioTouch3000 is intended for use in the electromagnetic environment specified below. The customer or the user of the CardioTouch3000 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The CardioTouch3000 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The CardioTouch3000 is suitable for use in all establishments other than domestic, and may be used in domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded:
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/ system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the CardioTouch3000 or shielding the location.

CardioTouch3000 Operation Manual


Guidance and manufacturer's declaration – electromagnetic immunity

The CardioTouch3000 is intended for use in the electromagnetic environment specified below. The customer or the user of the CardioTouch3000 should assure that it is used in such an environment.

IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5%UT (>95%dip in UT) for 0,5 cycle 40 %UT (60 %dip in UT) for 5, 6 cycles 70 %UT (30 %dip in UT) for 25,30 cycles <5%UT (>95%dip in UT) for 5 s	<5%UT (>95%dip in UT) for 0,5 cycle 40 %UT (60 %dip in UT) for 5, 6 cycles 70 %UT (30 %dip in UT) for 25,30 cycles <5%UT (>95%dip in UT) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the CardioTouch3000 requires continued operation during power mains interruptions, it is recommended that the CardioTouch3000 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE U_T is the a.c. mains voltage prior to application of the test level.

CardioTouch3000 Operation Manual

Guidance and manufacturer's declaration – electromagnetic immunity			
The CardioTouch3000 is intended for use in the electromagnetic environment specified below. The customer or the user of the CardioTouch3000 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V rms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the CardioTouch3000, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3,5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected			

CardioTouch3000 Operation Manual

by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the CardioTouch3000 is used exceeds the applicable RF compliance level above, the CardioTouch3000 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the CardioTouch3000.

^b Over the frequency range 150 kHz to 80 MHz, field strength should be less than 3 V/m

Recommended separation distances between portable and mobile RF communications equipment and the CardioTouch3000

The CardioTouch3000 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the CardioTouch3000 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the CardioTouch3000 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter [W]	Separation distance according to frequency of transmitter [m]		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2,5 GHz
	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
	V ₁ = 3 Vrms	E ₁ = 3 V/m	E ₁ = 3 V/m
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.70	3.70	7.37
100	11.70	11.70	23.30

CardioTouch3000 Operation Manual

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Chapter 5. Specification

ECG Leads	Simultaneous 12 Leads Resting ECG
Recording Channel	3CH+1RHV, 3CH+3RHV, 6CH+1RHV, 12CH, 1CH Long Time (1min) and Beat Report
Gain	2.5, 5, 10, 20, Auto (I~aVF: 10, V1~V6: 5)mm/mV
Printing Speed	5, 12.5, 25, 50, 100mm/sec
Sampling Rate	Analysis Sampling Rate – 500sample/sec Digital Sampling Rate - 8,000sample/sec
Filters	AC (50/60Hz, -20dB or better), Muscle (25~35Hz, -3dB or better), Base Line Drift (0.05, 0.1, 0.2Hz, -3dB or better), Low Pass Filter (off, 40Hz, 100Hz, 150Hz)
Display	272 x 480 Pixel Resolution, 4.3" Color TFT Wide Display 12 Channels Preview, ID, HR, Gain, Speed, Power Status
User Interface	Touch Screen, Rotary Push-knob
Printer Resolution	Thermal Print Head, Roll Paper Report Paper - Width: A4/Letter – 215mm (8.5") - Length: A4 – 297mm (11.7") Letter – 279mm (11") Resolution: 8dot/mm (0.125mm pitch)
Patient data	ID, Name, Age, Gender, Height, Weight
Basic Measurement	Heart Rate: 30~300bpm (± 2 bpm) PR, QRS, QT/QTc, P-R-T axis, SV1/RV5/R+S amp.
ECG Data Storage	Internal Storage for 200 ECGs: Built- in Memory
Electrical	Internal Noise: 20 μ V(p-p)max Input Impedance: More than 50M Ω Input Voltage range: ± 5 mV CMRR: Greater than 105dB DC offset Voltage: $\geq \pm 400$ mV Patient Leakage Current: < 10 μ A Frequency Response: 0.05~150 with in -3dB Isolated, Defibrillation and ESU Protected
Signal Quality Control	Pacemaker Pulse Detection Lead Fault Detection, Signal Saturation Detection
Line Power	Input: 100-240Vac, 1.5-0.75A, 50-60Hz, Output: 15Vdc, 4.2A
Battery type	Replaceable and Rechargeable, Lithium ion, 11.1V, 2600mA
Battery Capacity	360 minutes of continuous operation without recording or

CardioTouch3000 Operation Manual

	200 ECGs in 12 channel format at 25mm/s and 10mm/mV with a battery charged during approximately 3 hours from total discharge (with display off)
Communication	LAN, WIFI (option)
Safety Conformity	Class I, Type CF Defibrillation-proof applied Part.
Environmental	Operating Temperature: 5~40℃ Operating Humidity: 10~90%RH Atmospheric Pressure: 70~106KPa
Dimensions	290(W)×300(D) ×97.5(H)mm, Approx. 3.5kg (Main Body)
Standard Accessory	Patient Cable (1 EA), Limb Electrodes (1 SET), Chest Electrodes (1 SET), ECG Paper (1 EA), Power Cord (1 EA)
Options	Rechargeable Battery (1 EA), Cart(1 EA), Hanger (1 EA)