



Distributed by:

Universal Medical Inc.

www.UniversalMedicalinc.com

Care and Cleaning Guide for UMF Products

Care of Upholstery

The upholstery material used on the top, leg rest, and headrest is resistant to most medical stains. Care should be used where solvents or dyes are used. Mild soap and water should be used for daily cleaning of upholstery. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid. If dirt or stains still remain, use rubbing alcohol.

Note: Immediately remove any fluid spilled on upholstery surface.

CAUTION: WHEN THE USE OF RUBBING ALCOHOL IS NECESSARY, TEST AN INCONSPICUOUS AREA TO ASSURE THAT DAMAGE TO UPHOLSTERY WILL NOT OCCUR.

Care of Painted surface:

A chemical acid resistant paint is used, but extreme care must be taken not to use ammonia based cleaners or discoloration of paint may occur. A damp cloth or mild soap solution should be sufficient. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid.

Care of Bright Metal Surfaces:

All non-painted surfaces, chrome plated or stainless steel should be wiped weekly with a clean damp cloth then buffed to a shine with a soft dry cloth.

Care of Base:

The protective base is easily washable with mild soap and water. A soft bristle brush may be used on scuffed or stained areas. UMF Medical recommends the use of Murphy's oil soap or Ivory liquid.