



How to replace Roller Board Covers

1. **REMOVE:** Remove existing cover and clean rollers and ends completely to remove any bacteria and dirt that may accumulate.
2. **TEST:** Ensure Rollers rotate freely. To ensure no rollers are bent, put the board on a smooth flat surface and roll from side to side. The board should roll evenly. If it does not, we recommend replacing the entire roller board. Ensure that both screws on either end are snug and board is not flexible, it should be rigid. If it is not rigid, remove loose screws, and apply thread locker before tightening them again.
3. **START:** Before starting to put the new cover on, unfold it and ensure to ensure you have the correct length cover. Next, stand your roller up vertically on a smooth hard surface at about waist height or table height. Take the new cover and start it onto the board. To start with, just put on the roller an inch or two. It will be very snug, but this is normal.
4. **COMPLETE:** Once you have the cover evenly started and there are no wrinkles, continue pulling the cover onto the board. You can roll from side to side between your hands with downward pressure to continue sliding the cover on. Use pressure either up or down to get the cover in place. Once the cover is evenly covering both ends, your roller is ready to use!