



The CT Hugger consists of 6 parts:
Two foam inserts (infant, child), curved cradle,
and three restraint straps (head, legs, and torso).



into cut-out opening of foam insert. The sewn
end of strap should extend through opening.

Take leg strap and insert strap



opening, separate strap as shown above.

When strap is inserted through



Line up foam insert such that velcro from the foam insert is lined up with velcro in cradle.



The outer leg strap is used as a restraint for the legs as demonstrated above. Adjustments are made dependent on size of infant/child.

The outer leg strap is used as a



The torso/ hand restraint is positioned in the same way as the leg restraint over the torso.

The torso/ hand restraint is



to the torso strap and hands/wrists are positioned as demonstrated above.

The hand restraints are connected



Head restraint is connected as demonstrated above.