



Metal Stand Mount Respiratory Hygiene Station Assembly

FAMILIARIZE YOURSELF WITH THE PARTS OF THE STATION AND THE HARDWARE LISTED BELOW



1. Lay stand on a table and attach base to bottom 4 holes on stand using 4ea 10-24x1in screws. (Image 1 and 2)
2. Stand the base/stand upright.
3. Attach station to top of stand using 2ea 10-24x3/8 screws. (Image 3 and 4)
4. Insert two 10-32 x 1/2in screws into top holes of sign sleeve (Image 6) to help position into corresponding holes on back of station (Image 5).
5. Secure sign sleeve using 10-32 screws and square nuts (Image 7).