


Casco First Responder RescueAll™ Transport, Disposable

The Casco First Responder RescueAll™ Transport is made with two types of polypropylene – white for the main surface of the Transport, and black for the webbing which provides the support network for the Transport, making it safe and easy to use. The product design also enables it to be used as a slide, serving as an alternative means of patient transport.

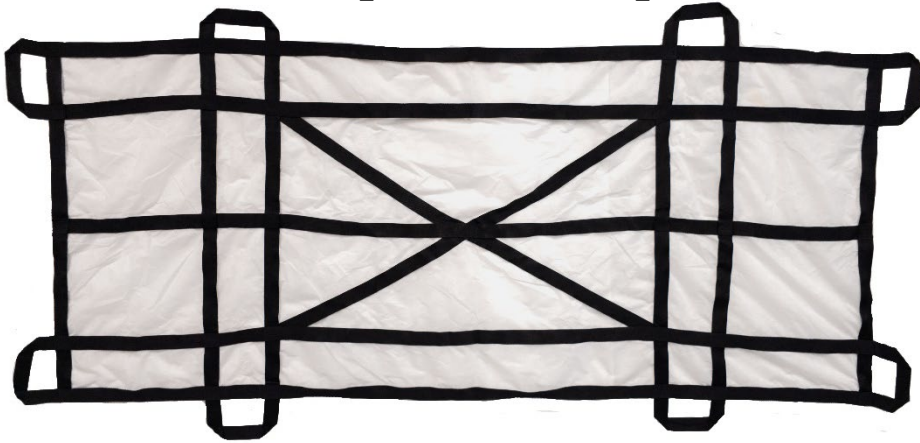

- The Casco First Responder RescueAll™ Transport is rated to transport up to 1000 pounds (454 Kg).
- The Casco First Responder RescueAll™ Transport is recommended for use by professionals for lifting and transporting patients.
- Multiple handles for ease of team/crew handling.
- Unique handle design for ease of movement.
- Note: Rescue/All is not designed for use of spinal/traction stabilization.

Casco First Responder RescueAll™ Transport, Disposable

<u>Product Codes</u>	Part # GTIN UPC
Casco First Responder™ RescueAll Transport, Disposable	H12721 00810125100390 810125100390

Usage Instructions:

- Do Not Machine Wash or Dry
- Avoid Contact with Sharp Objects
- Do Not Use if Punctured, torn, or Excessively worn
- Only Use the Handles for lifting; grip securely
- Minimum 4 persons to lift and use
- Grab a minimum of four (4) handles on each side to ensure safe weight distribution, and prevent sagging.

