



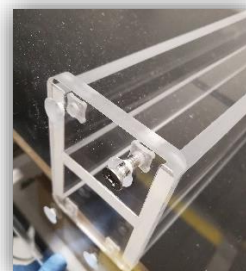


## Respiratory Hygiene Station Assembly

FAMILIARIZE YOURSELF WITH THE PARTS OF THE STATION AND THE INDIVIDUAL BAGS OF HARDWARE AS REFERRED TO IN INSTRUCTIONS BELOW.

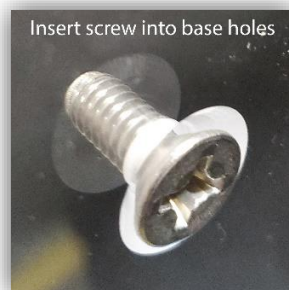

1. Use the "Base to Stand" hardware for this step. Align the 4 square pockets on the STAND with the 4 holes on the BASE. Attach BASE to STAND using the 4 screws and square nuts. Using Phillips screwdriver: turn until snug, then 1/8 turn more to tighten.
2. Stand the BASE and STAND assembly up. The rubber pads should be on the floor.
3. Use the "Station to Stand" hardware for this step. Attach STATION to top of STAND. Align two square nut pockets with screw holes. Using Phillips screwdriver: turn until snug, then 1/8 turn more to tighten.

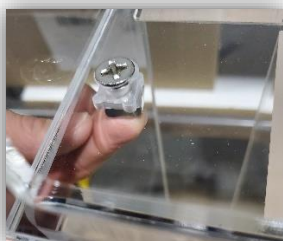

4. Use the "Sign to Station" hardware for this step. Attach SIGN SLEEVE to back of STATION. Align 4 square nut pockets on STATION with 4 holes on SIGN SLEEVE. Using Phillips screwdriver: turn until snug, then 1/8 turn more to tighten.

