




Wall Mount Respiratory Hygiene Station Assembly Instructions

1. Use the 1 ¼ in long screws and white anchors to mount the sign holder to the wall in the 4 wall mount holes. If you can find a stud to mount to, you will not need the anchors in those positions. The anchors are for drywall installations. It is helpful to mark the location of your holes, then pre-drill a 1/8" hole to get the best alignment of the wall anchors. Also, this will tell you if you have hit a stud or not. Use a level during this pre-drilling process to make sure your sign/station will be mounted level.

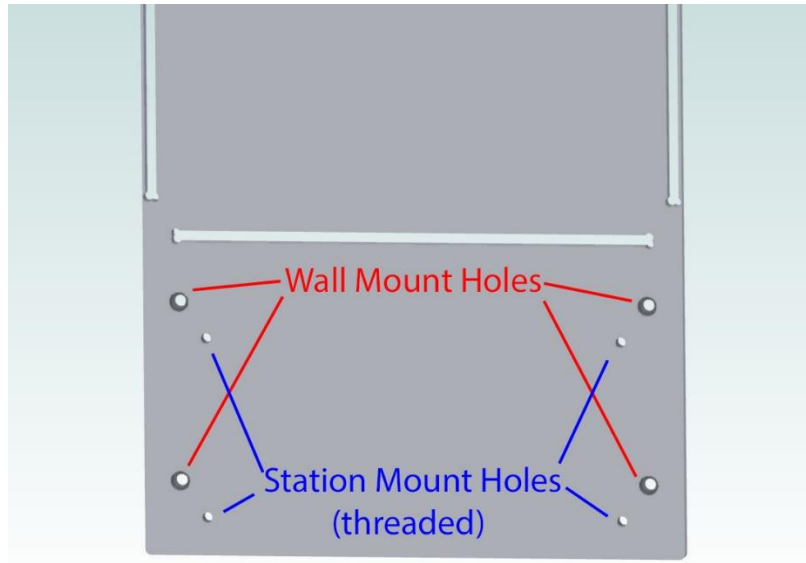

2. Once the sign bracket is mounted securely to your wall, position hygiene station to bracket and align 4 square cut-outs in the back of the station with the 4 threaded Station Mount holes on the sign bracket. Use the 3/8" long button head screws to mount station to wall bracket/sign in the 4 threaded holes.

