NOTIONS DE BASE

OBJECTIFS

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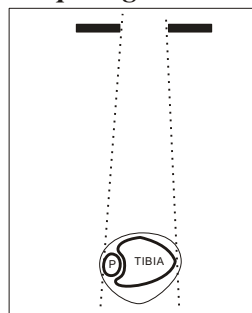
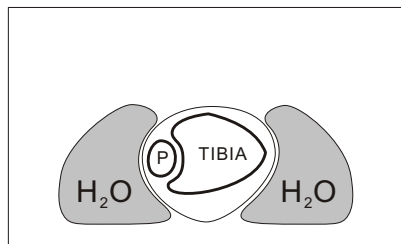
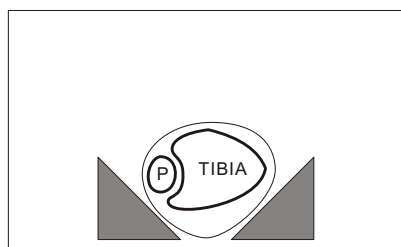
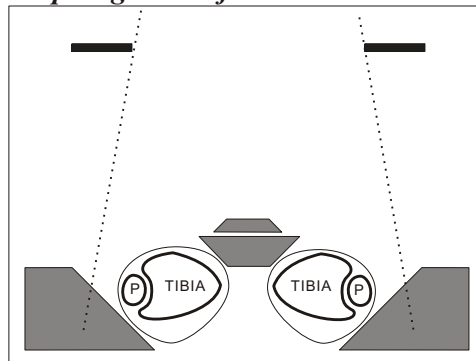
BASIC NOTIONS

OBJECTIVES

To even out the density of an image which would otherwise be too bright or too dark in some areas. This saves films, time and radiation. Also to immobilize at the same time.

MEANS

1. Closure of the **diaphragms** inside the part examined.
2. Use of a **bolus** near the part to be radiographed. This cuts the peripheral "black" which spreads over the anatomy, but also casts artifacts.
3. Use of a **compensating filter**:
 - a) a "black" area will be attenuated, such as the periphery of the shoulder with the "BOOMERANG";
 - b) a "white" area may be better penetrated while protecting the rest of the film, v.g. C-7 with the "INGOT".
4. Local **digital** adjustments on areas too bright or too dark, but not simultaneously with normal areas.
5. An **electronic threshold** cutting off overexposed areas of the image.
6. A **combination** of many factors.

MATERIAL AND ABSORPTION

1. **Rice, flour, etc.** do not absorb sufficiently.
2. **Water** and **serum bags** would be good mainly as boluses, but not practical. Near the trunk they would be large and heavy, although adding an opaque substance improves them.

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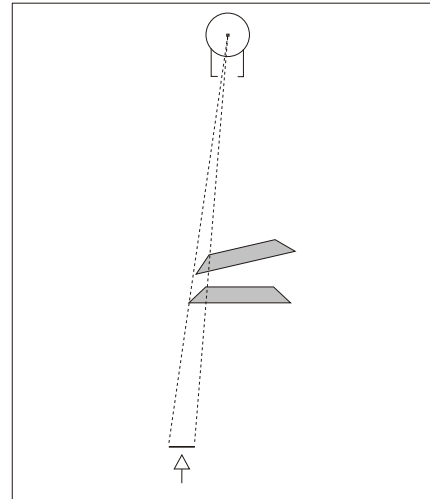
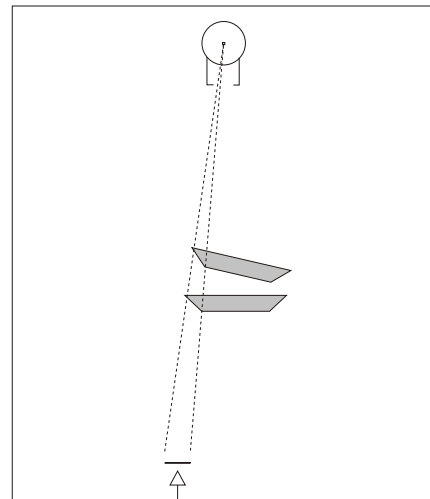
Gradient estompé.

Soft gradient.

3. Rigid **plastic** is good, but the tapered edge may break, thereby injuring a patient or a technician; it does not adhere vertically.

4. Metals, such as **lead** and leaded plastics, are generally too dense. **Copper** and **aluminum** are useful.

5. **OCTOSTOP® compensating filters** are made of **silicone rubber**. Their absorption is ideal at twice that of muscle, which permits placing them on only one side of the patient.

They are light to carry, easy to handle, readily identifiable and not cumbersome. They are positioned with precision close to the patient. If dropped, they do not break or harm anyone. In the vertical position, they adhere with very little pressure (wash with water if they slip). If too sticky, interpose tissue paper.

They resist alcohol and heat. They are waterproof.

6. PHARAOH boluses in polyurethane surround the head and the neck completely: they absorb like water.

GRADIENT OF ABSORPTION

It is proportional to the slope of the wedge: the steeper the gradient, the sharper and more obvious is the absorption. The more gradual it is, the less perceptible is its shadow. It is measured in degrees, or as a fraction:

BOOMERANG: 30°-45° or 1/1.7-1/1

INGOT: 45° " 1/1

GENTLE SLOPE 8° " 1/7

PRISM: 37° and 52° " 1/1.3 and 1/1.8

In addition, the gradient varies according to the orientation of the filter. It increases as the axis of the wedge becomes more parallel to the direction of the X-ray beam; it decreases in the reverse situation (see figures).

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
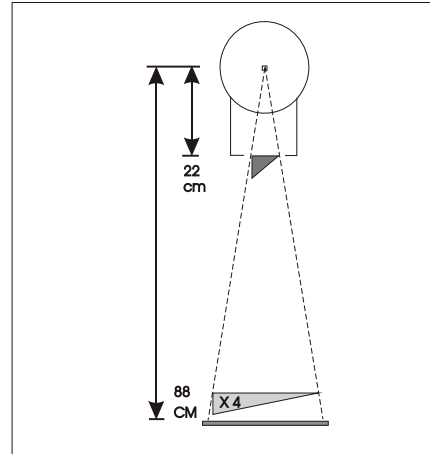
Gradient at collimator vs on patient

Bord effilé.

Tapered edge.

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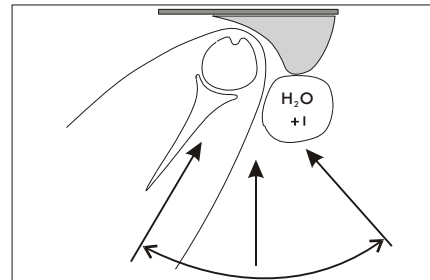Special tomography.

POSITION

1. *Behind the patient.*
2. *In front of the patient: saves radiation to the patient.*
3. *At the collimator, with Velcro[®] as an inexpensive means of attachment. Here the gradient diminishes by a factor of 4, since its width is magnified in the proportion of 88:22 cm (see diagram).*

SHAPE

1. *Curved at 90° for the BOOMERANG.*
2. *Rectilinear for other filters.*
3. *Adapted.*

TAPERED EDGE

It is fragile and must not be stretched or torn off with fingernails. It is nevertheless long lasting with reasonable care. Do not fold over, as this may create an artifact.

SPECIAL TOMOGRAPHY (axial of the shoulder)

When a superficial region is **very thick**, controlling its density is difficult: because the wide sweep of very "black" air coming from levels far from the cut, largely overlaps the anatomy, which then becomes much too dense. Nevertheless, two solutions are available:

- A. **Place a filter against the patient and add a bolus containing iodine.**
- B. **And/or use a highly absorbing filter at the collimator.**

STERILIZATION and DISINFECTION

Autoclave is recommended for sterilization; alcohol or Zephiran[®] for disinfection.

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A compensating filter to even out the density of shoulder radiographs

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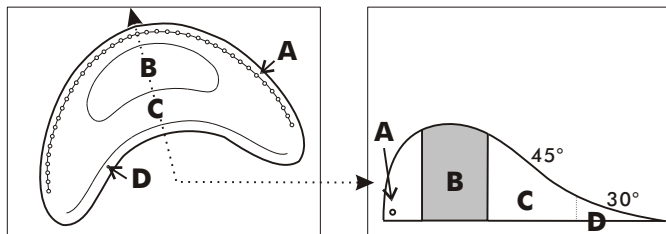
DESCRIPTION

A. **Metal chain** along the periphery indicates the presence and position of the filter on the film. It projects outside the anatomy, and reduces risks of legal litigations.

B. Area of **maximum** absorption: equivalent to +/- 9 cm of muscle. For large shoulders, cephalometry, etc.

C. Area of **intermediate** absorption: +/- 45° gradient, for smaller shoulders.

D. Area of **minimum** absorption: 30° gradient, v.g. for patella.

The shadow of the shoulder from the light localizer should fall at the desired absorption level B, C or D. The filter will stay in the vertical position with a slight pressure from the rotated trunk.

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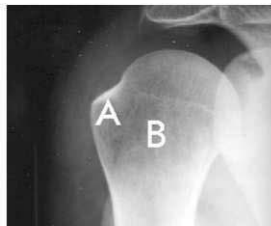
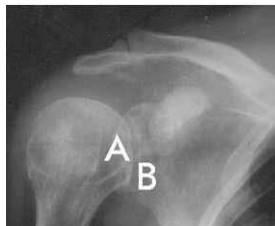
ROTATION OF THE HUMERUS

A. Greater Tuberosity
B. Lesser Tuberosity

The position of the tuberosities is controlled by the position of the elbow.

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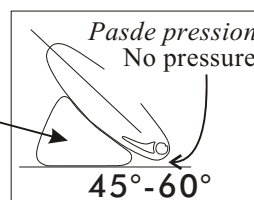
Two views at 90° are preferred for trauma cases.*

ROTATION OF THE TRUNK

The shape of the glena is controlled by the position of the trunk.

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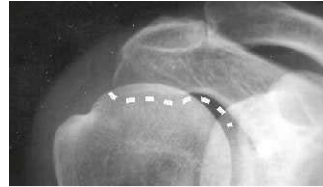
Lower border of the spine of the scapula....

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
...merges with inferior border of the acromion.

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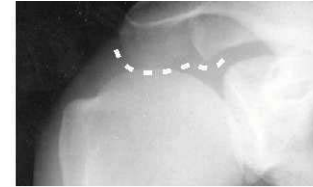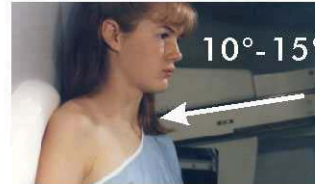
Decubitus requires more angulation than the upright.

Always position the filter according to the localizing light.

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KYPHOTIC PATIENTS

Upright requires less angulation than normally. Watch the shadow of the shoulder on the filter. Then, interpose a foam cushion.

Decubitus requires more caudad angulation than the upright.

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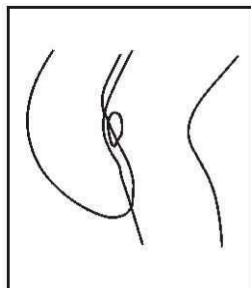
PATELLA and TIBIAL TUBEROSITY:

Use only a few cm (area D) of one of the tips. Do not cover the supra-patellar area to avoid a fluid artifact.

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CEPHALOMETRY:

Place maximum thickness (zone A) against nasion and chin. Use a high KV (90-100), a low MaS and slightly underexpose.

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D1-D3: when this portion is thinner than the rest of the thorax.

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IMPORTANT

Only the wedge portion is normally superimposed, while the periphery remains outside. Since the tips are at 90° to each other, they might occasionally be superimposed, and create an artifact. This would be recognizable by the presence of the chain. If there is a risk of a misinterpretation, make a note on the film, or take a corrected exposure.

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IMPORTANT PRECAUTIONS

1. Read this educational brochure attentively.
2. Do not filter when there is little or nothing to be gained.
At the beginning, conduct your own experiments.
3. In general, adjust technical factors for the unfiltered area.
4. If an artefact is observed, indicate it on the film;
or better, take another corrected exposure.
5. Observe local practices. Consult your radiologist.
6. Send us your suggestions.

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IMPORTANT

Must be used only by qualified personnel and under the direct supervision and responsibility of a physician. When applying traction, caution should be exercised when the following conditions are present or possible: fractures of the pelvis; spine or upper extremities; uro-genital or abdominal trauma; or all other circumstances that might be contraindicated.

OCTOSTOP[®] Inc. and its personnel do not accept any liability for how its products are used, for the indications, consequences or any situations directly or indirectly related to its products.