





SM-MYO-A01 Uterine Fibroids (Myoma) Thermal Ablation Skills Training Platform

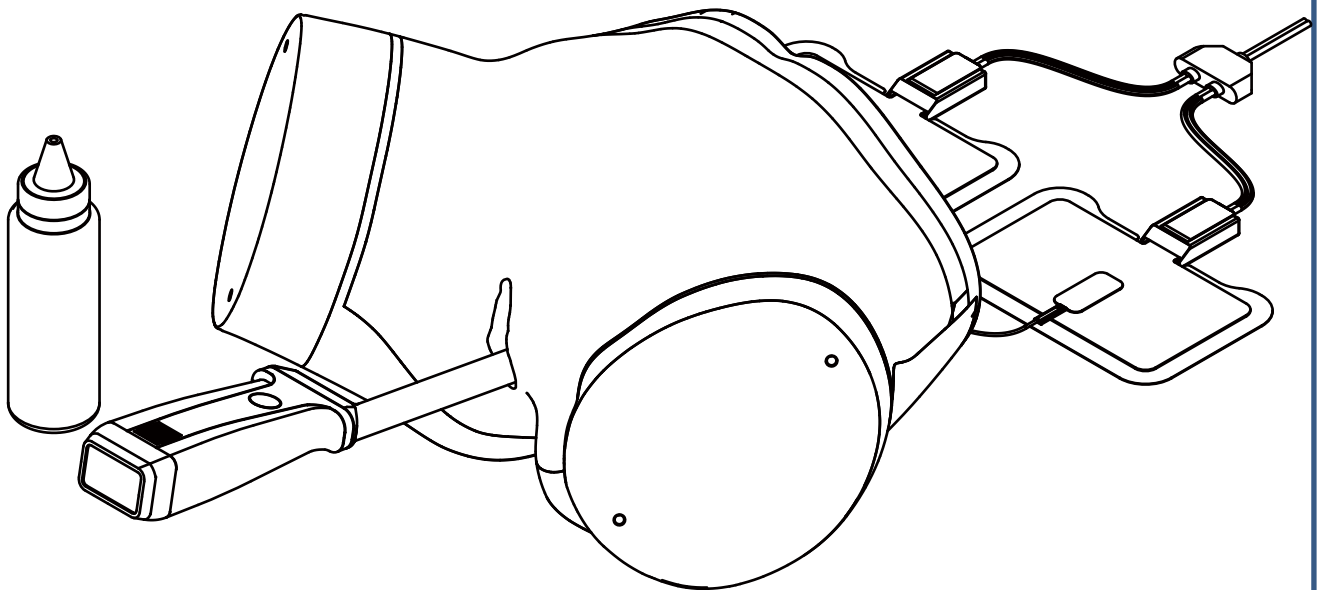


Instruction Manual

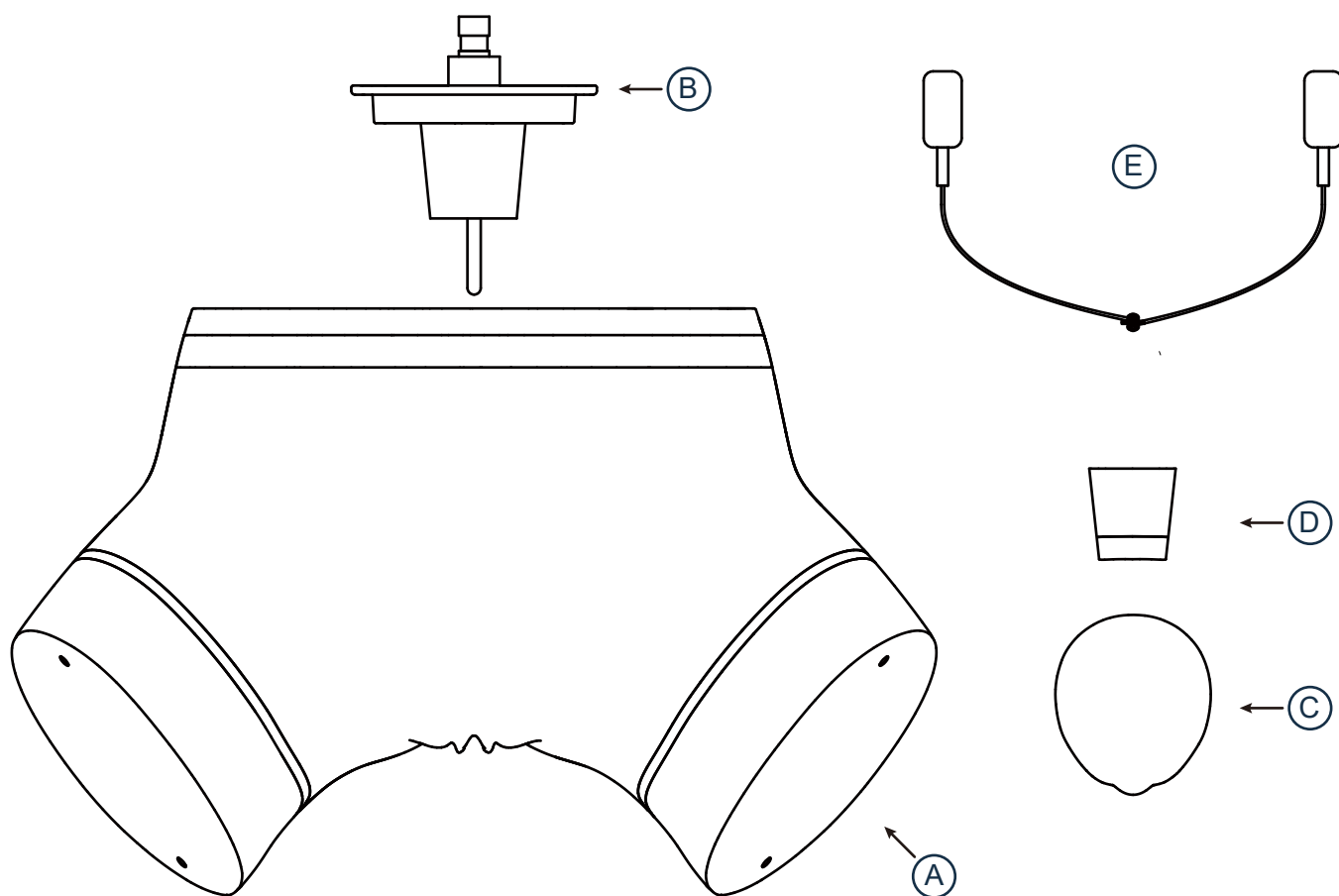


A.	<u>Body</u>	1
B.	<u>Cover</u>	1
C.	<u>Uterus</u>	1
D.	<u>Contact Block (RFA Only)</u>	1
E.	<u>Eletro Pad (RFA Only)</u>	1

● Preparation


- ① Inject gel into the Urethra and Rectum.

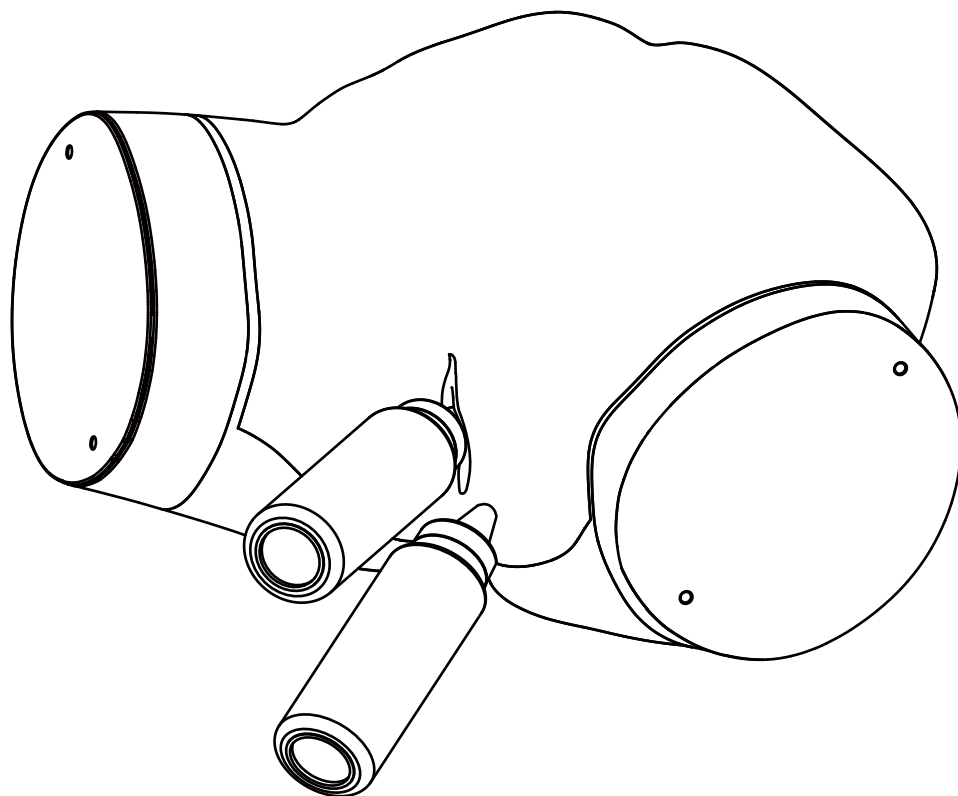


- ② Unlock and remove the Cover.

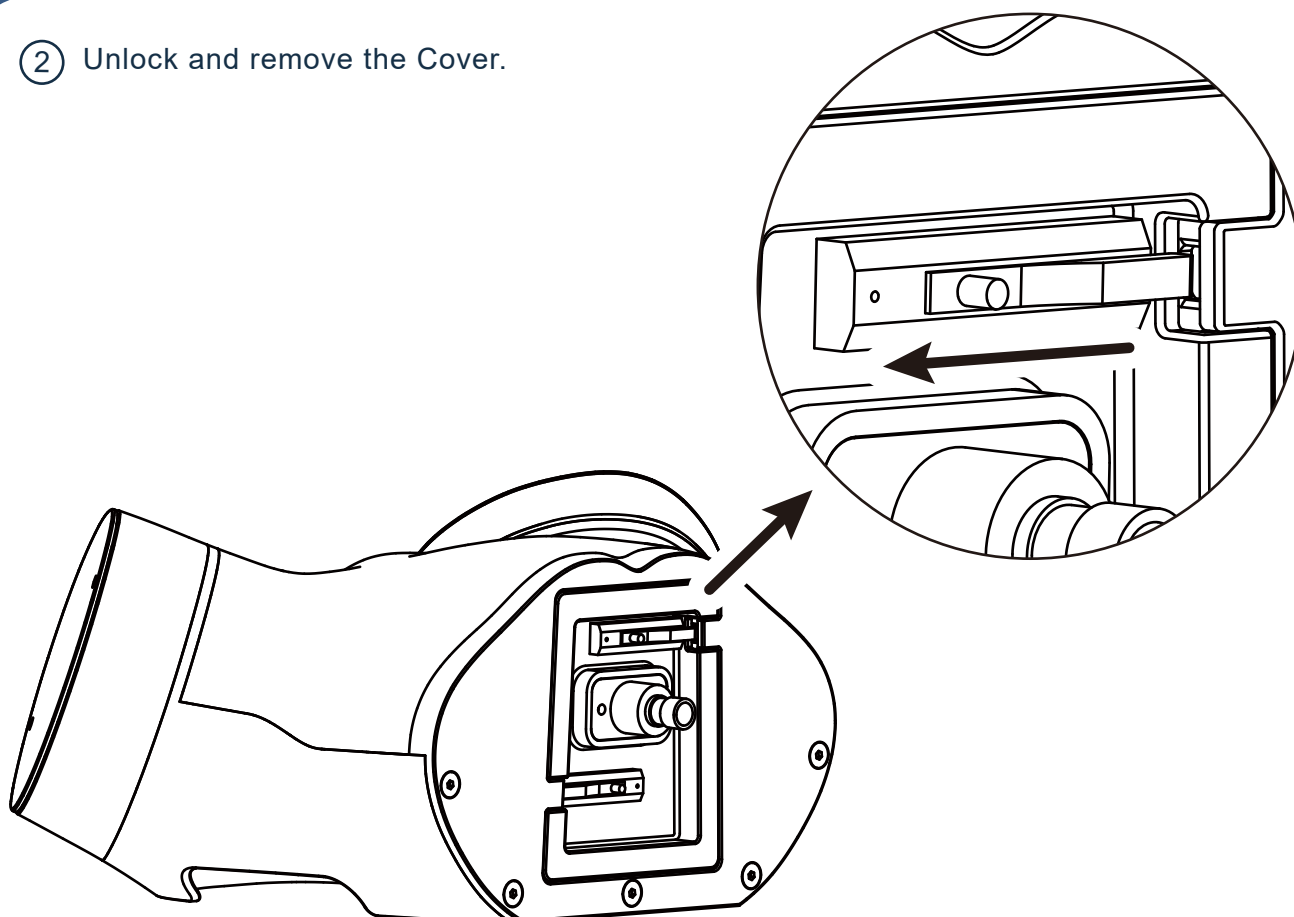


- ③ Connect the Electro Pad. (Only applicable for RFA procedure)
- ④ Establish the circuit from the Cover to the RFA console via two Electro Pads. (Only applicable for RFA procedure)

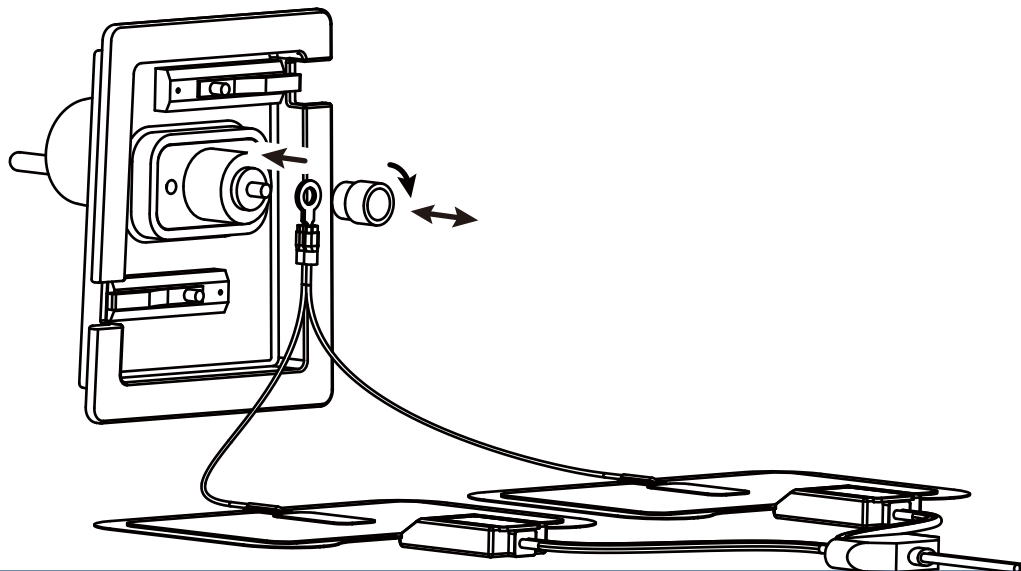

- ⑤ Insert the Uterus through the insertion tunnel.
- ⑥ Insert the Contact Block with the Cover.

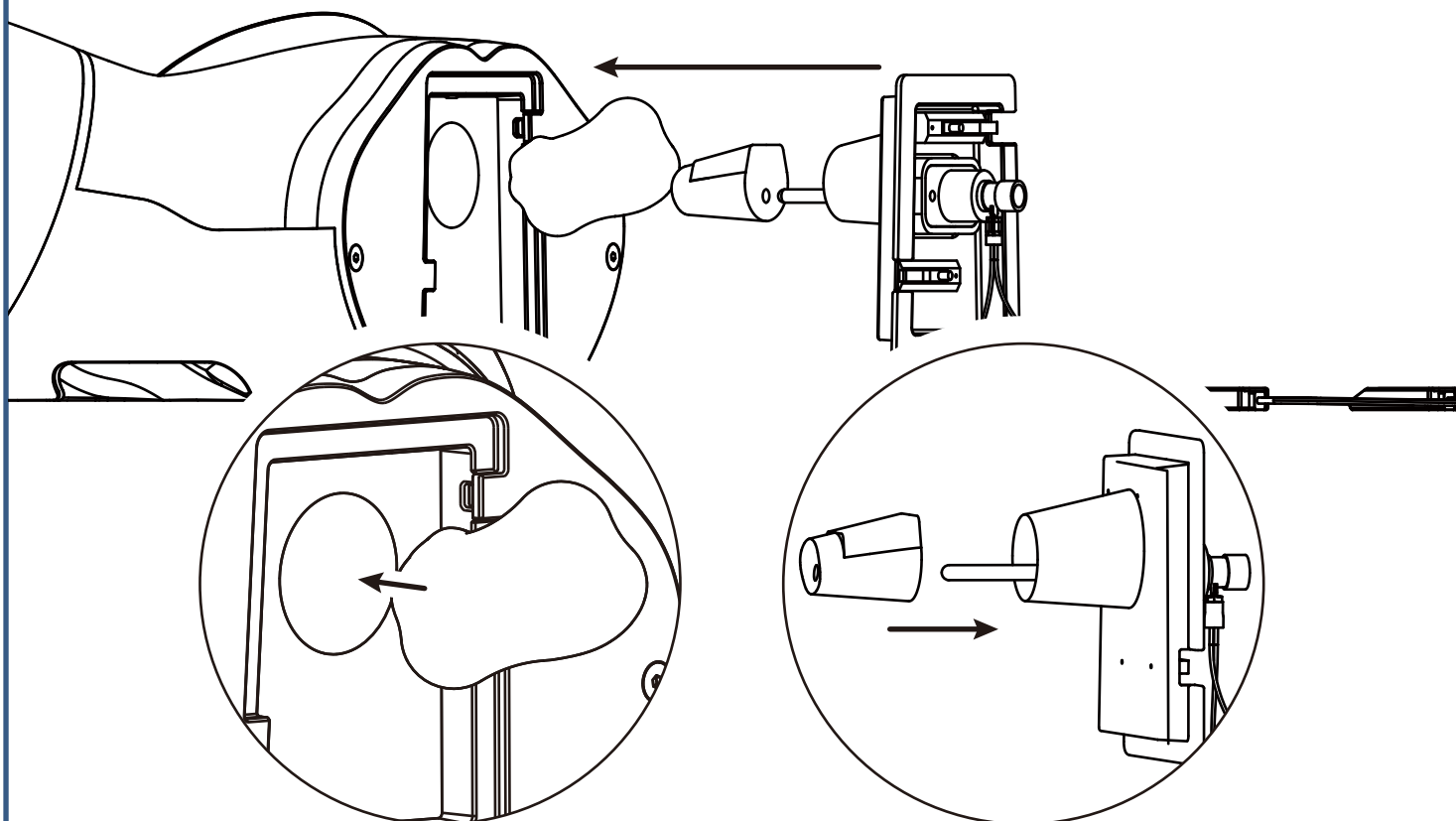

- ⑦ Slide the latches to fix the cover.

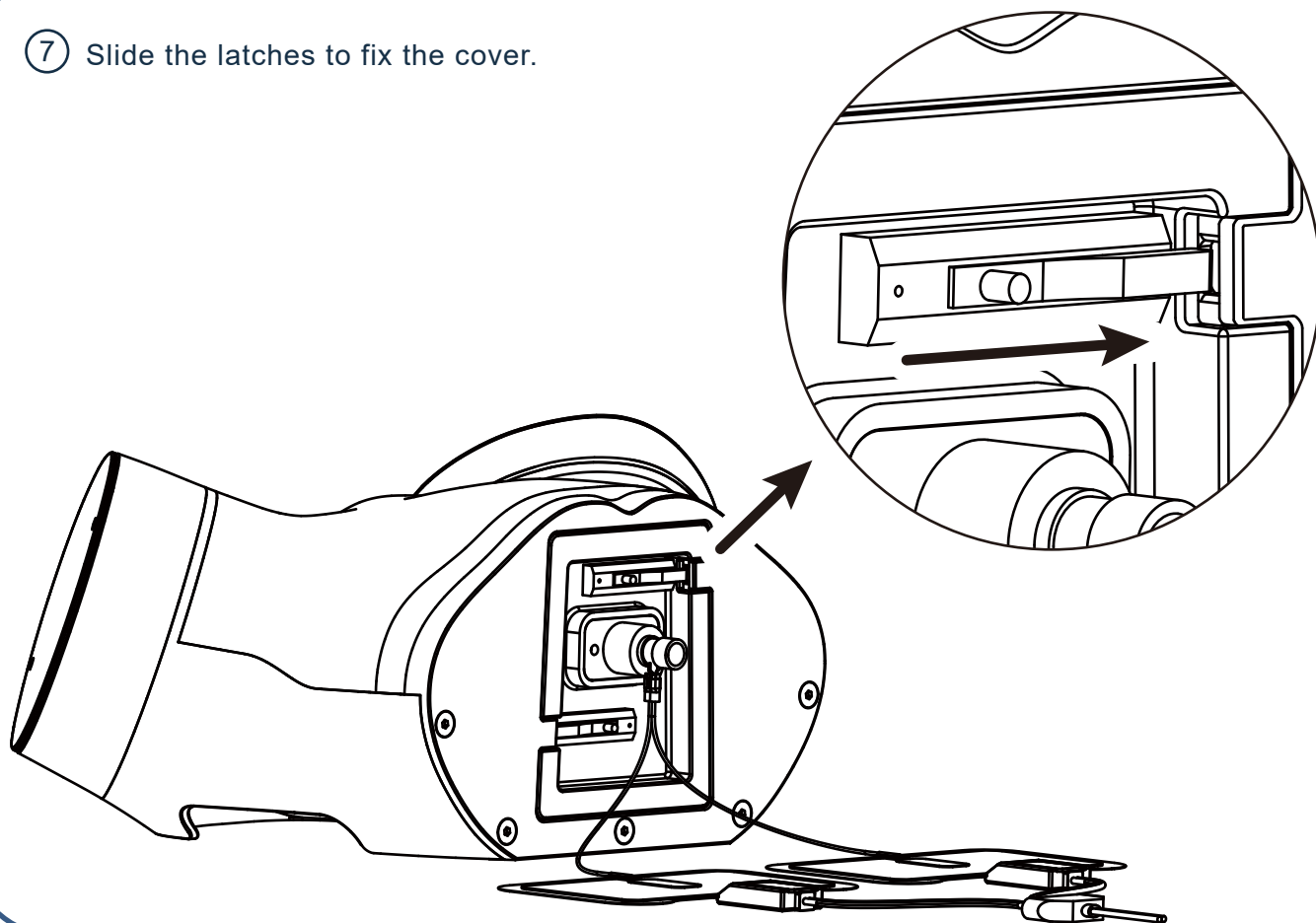

● Cleaning, Maintenance, and Storage

- ① Unlock and remove the Cover.
- ② Remove the Uterus after ablation procedures.
- ③ Use the wipes to clean the hole.
- ④ Remove the gel with in the Urethra and Rectum.